



Filtek™

Supreme
Flowable Restorative

Now our most popular flowable is easier to inject.

3M™ Filtek™ Supreme Ultra Flowable Restorative is available in a new ergonomic syringe. Designed for improved comfort and easier injection, this new product is called 3M™ Filtek™ Supreme Flowable Restorative.

Reduce bubbles and waste


Virtually no bubbles or material “run-on” during dispensing gives you better control.

No more guessing

The blue syringe barrel indicates the remaining material volume. When your syringe is empty, only the white plunger shows.


Easy to hold and inject

Triangle finger plate and plunger make it quick and easy to personalize your grip.

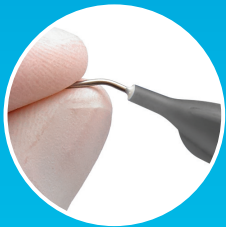

Bendable cannula

Easy-to-bend without kinking for better access.

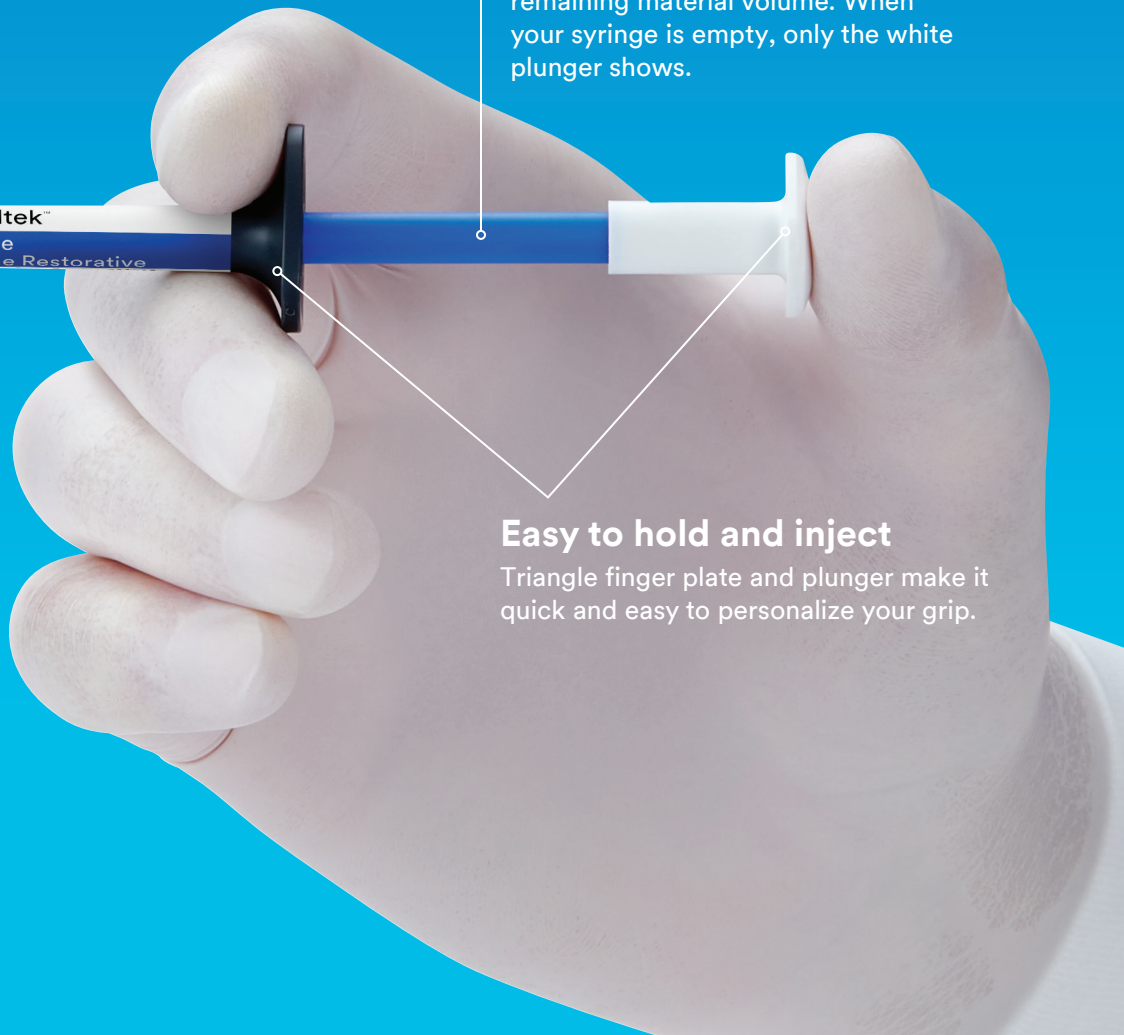

Key benefits

- Virtually no bubbles or material “run-on”
- Easy-to-bend cannula for improved access
- Ergonomic design is easy to hold and inject
- Excellent adaptation, polish retention and wear resistance
- Versatile for a wide range of indications and techniques

Can be warmed for injection molding

Warm up to 70°C (158°F) for up to one hour, up to 25 times.



Clinical cases

3M™ Filtek™ Supreme Flowable Restorative offers a smooth transition from material to tooth for esthetic restorations.



Before

Flowable placement

After

Photos courtesy of Dr. Gunnar Reich, Munich, Germany

3M™ Filtek™ Supreme Flowable Restorative adapts easily and seals the margin.



Before

Flowable placement

After

Photos courtesy of Dr. Giuseppe Chiodera, Brescia, Italy

Indications

(Incremental up to 2 mm)



Class III



Class V



Minimally Invasive Dentistry (MID)¹



Base



Liner



Repair of Indirect (small defects)



Repair²



Undercut Block-out



Pit & Fissure Sealant

Refer to Instructions for Use for full list of indications.

Ordering information



3M™ Filtek™ Supreme Flowable Restorative Refills

Syringe refills include: 2 – 2g syringes; 20 dispensing tips.

SHADE	ITEM NO.
A1	6032A1
A2	6032A2
A3	6032A3
A3.5	6032A3.5
A4	6032A4
B1	6032B1
B2	6032B2
C2	6032C2
D2	6032D2
White	6032W
Extra White	6032XW
OA3	6032OA3

ITEM NO.	PRODUCT INFO.
3700T	3M™ Filtek™ Supreme Flowable Restorative Dispensing Tips, 20-gauge, Gray (20)



3M

3M Oral Care
2510 Conway Avenue
St. Paul, MN 55144-1000 USA

Phone 1-800-634-2249
Web 3M.com/dental

3M Canada
Post Office Box 5757
London, Ontario N6A 4T1
Canada

Phone 1-888-363-3685

1. Including small, non stress-bearing occlusal restorations.
2. Repair of acrylic temporary and composite resin.

3M and Filtek are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Please recycle. Printed in U.S.A. © 3M 2020. All rights reserved. 70-2013-7231-8 (REV. 3/2020)