

Adper™ Adhesive Systems Filtek™ Z250 Universal Restorative Sof-Lex™ Discs and Strips

Preparation:

- Select shade.
- Prepare the tooth.
- Bevel enamel margins.



Adhesive Placement:

- Apply adhesive following directions for 3M™ ESPE™ Adper™ adhesive system of choice.



Place Composite:

- Place Filtek Z250 Restorative in increments. (Reference Light-Cure chart for thickness and cure time.)



Finish and Polish:

- Use the Sof-Lex Finishing and Polishing Discs (C,M,F,SF) for finishing and polishing the restoration.



Strips:

- Use Sof-Lex Strips for finishing proximal areas by gently inserting the center gapped area between teeth. Operating sequence of strips (C/M, F/SF) is the same as discs.

