

## Adper™ Adhesive Systems Filtek™ Z250 Universal Restorative Sof-Lex™ Discs and Strips

### Indication:

- Class V cavities and cervical erosion/abrasion lesions and root caries lesions.
- Class III cavities.

### Prepare the Tooth:

- Select shade using the shade guide.
- Isolate the tooth.
- Remove caries.

### Adhesive Placement:

- Apply adhesive following directions for 3M™ ESPE™ Adper™ adhesive system of choice.

### Place Composite:

- Place Filtek Z250 restorative in increments. (Reference Light-Cure chart for thickness and cure time.)

### Finish/Polish:

- Use the Sof-Lex Finishing and Polishing System (discs and strips) for finishing and polishing the restoration.

