



3M™ Cavidon™ Wipes



Save Time, Reduce Cross Contamination & Skin Safety

Bart Maene Scientific Affairs Manager EMEA

Daily Bed Bathing


Traditional Procedure (20-30 min.)

1. Collect wash materials (Towels, Wash Cloths, Soaps, Lotion/moisturiser, gloves etc)
2. Ensure privacy for patient
3. Use towel to prevent wetting the sheets
4. Apply gloves and apron
5. Use basin 1 to wash face, neck, arms, armpits, chest and abdomen. Use Basin 2 to wash legs, feet, inguinal and genital area
6. Apply additional moisturiser
7. Turn patient
8. Wash patient back using wash mitt 1 and wash basin. Wash buttocks and anal area using wash mitt 2 and wash basin 2
9. Apply skin protectant and new incontinence pad if applicable
10. Refresh wet sheets if become wet or needed
11. Turn patient onto their back
12. Dress patient
13. Dispose the water of the wash basins in the designated sink
14. Disinfect the wash basins / discard disposable basin
15. Use hand hygiene measures

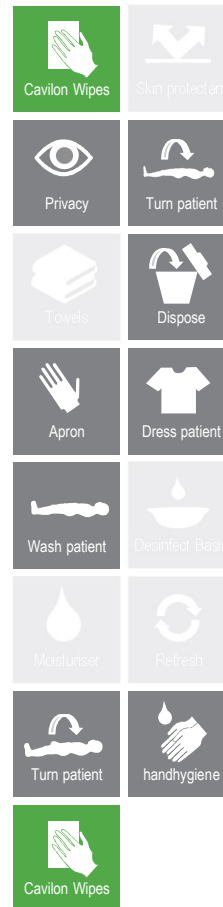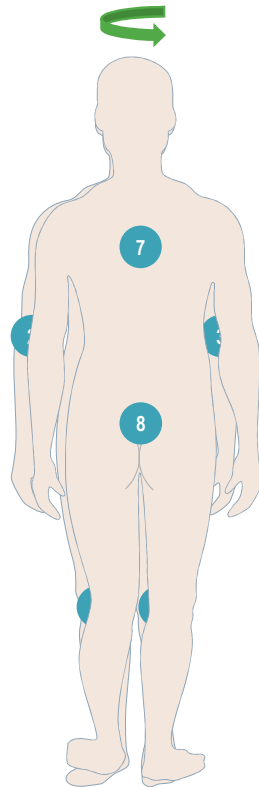

Bathing with Wipes (+/- 43% time saving)¹¹


1. Take a pack of 3M™ Cavilon™ Bathing & Cleansing Wipes
2. Ensure privacy for patient
3. Not necessary
4. Apply gloves and apron
5. Use Cavilon wipe 1 for Face, Neck and Chest, Use Cavilon wipe 2 for right arm and armpits. Use Cavilon wipe 3 for left arm and armpit. Use Cavilon wipe 4 for the perineal area. Use Cavilon wipe 5 for front of legs and feet
6. Not necessary
7. Turn patient
8. Use Cavilon wipe 6 for back and wipe 7 for buttocks and anal area. Use wipe 8 for back of legs and heels
9. Not necessary
10. Turn patient to its back
11. Dispose of the wipes
12. Dress patient
13. Not necessary
14. Not necessary
15. Use hand hygiene measures.

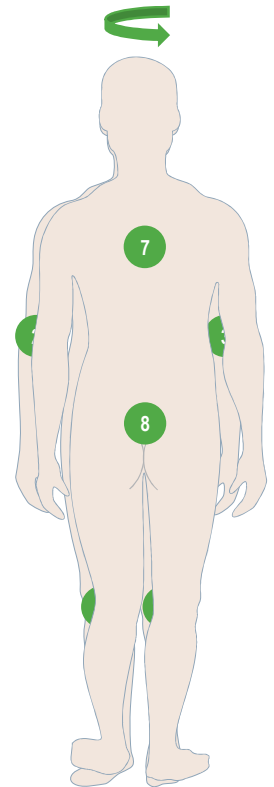

Contenance Care


Traditional Procedure (+/- 12 min.)

1. Collect wash materials
2. Fill wash basin with lukewarm water
3. Ensure privacy for patient
4. Throughout the procedure the patient should be remain covered to keep the patient warm
5. Turn patient and remove incontinence pad
6. Remove excessive faeces using wash cloth (paper) and discard appropriately
7. Wash patients buttocks and anal area
8. Cleanse wash cloth in wash basin
9. Dry patients buttocks and anal area
10. Apply designated Skin Protectant according to protocol
11. Apply new incontinence pad and turn patient
12. Change any wet sheets and re-dress patient
13. Dispose the water of the wash basins in the designated sink
14. Disinfect the wash basins or dispose of disposable wash basin
15. Dispose of other materials

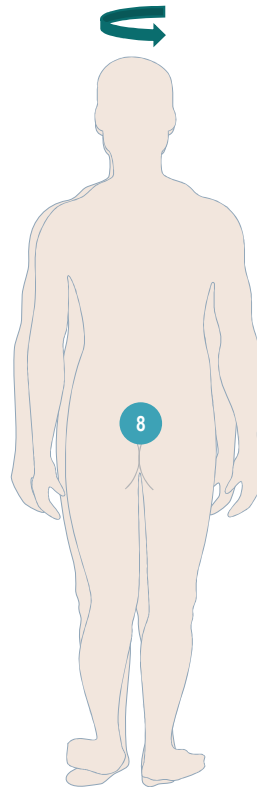

Cleansing with Wipes (+/- 7-8 min.)¹²

1. Take a pack of 3M™ Cavilon™ Contenance Care Wipes, apron and plastic bag to dispose used and soiled materials
2. **Not necessary**
3. Ensure privacy for patient
4. Throughout the procedure the patient should remain covered to keep the patient warm
5. Turn patient and remove continence care pad
6. Remove excessive faeces using wash cloth and discard appropriately
7. Use Cavilon continence care wipes to cleanse moisturise and protect
8. **Not necessary**
9. **Not necessary**
10. Follow Cavilon continence care protocol for additional protection in high risk patients
11. Apply new incontinence pad and turn patient
12. Re-dress patient
13. Dispose of the wipes
14. **Not necessary**
15. **Not necessary**

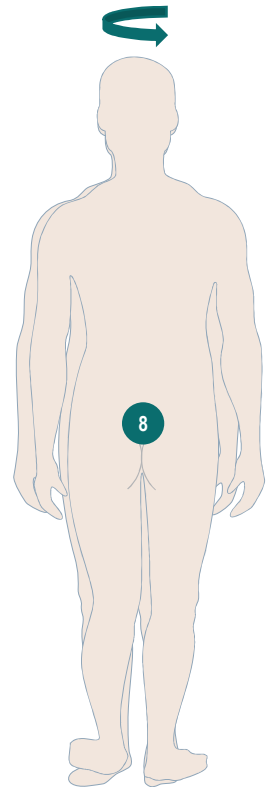

Summary: Why using Cavilon Wipes?

- Bottom Line: Wipes are better for skin, infection control and efficiency of staff!
- Decrease risk of cross contamination
- Decrease nursing time
- Increase patient comfort
- Maintain skin health
- Increase consistent use of moisturizers & protectants
- Increase staff productivity
- Decrease facility ancillary costs (laundry, soaps, basins)
- Evidence-based Care