

PROMOGRAN PRISMA™ MATRIX



What is it?

PROMOGRAN PRISMA™ Matrix is a sterile primary dressing made of freeze-dried composite of 44% oxidized regenerated cellulose (ORC), 55% collagen and 1% silver-ORC. In the presence of exudate, the PROMOGRAN PRISMA™ Matrix transforms into a soft, conformable, biodegradable gel, and thus allows contact with all areas of the wound.

What is it for?


PROMOGRAN PRISMA™ Matrix, when covered with a semi-occlusive dressing, like a foam, maintains a moist wound environment. This environment is conducive to granulation tissue formation, epithelialization and optimal wound healing. The silver in PROMOGRAN PRISMA™ Matrix helps provide a barrier to bacteria in the dressing.

How to apply

Preparing the wound


Prepare the wound according to appropriate wound management protocol as instructed by your clinician. Wash your hands before and after touching the wound. Wear gloves if you have them

Dressing preparation

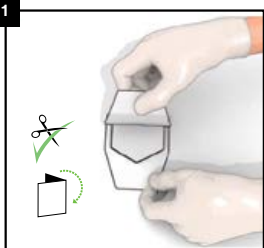

1. Peel back the tray cover and leave dressing in tray.
2. If necessary, **PROMOGRAN PRISMA™** Matrix can be cut using clean scissors, ripped or folded to fit the wound bed.
3. For a small or deep exuding wound, cut the dressing prior to application or pack dressing into the wound.

Dressing application

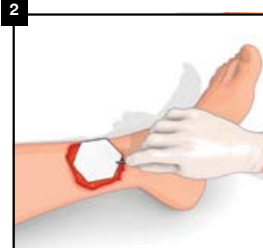

1. If the wound bed is wet apply the dressing directly.
2. If the wound is dry moisten the dressing with saline (use the dressing tray as a dish to pre-wet)

Secondary dressing application



3. Cover with an appropriate secondary dressing as directed by your clinician.

How often to apply

It is not necessary to remove any remaining PROMOGRAN PRISMA™ Matrix dressing during secondary dressing changes. Apply a new PROMOGRAN PRISMA™ Matrix as directed by your clinician..

Frequently Asked Questions

Do I have to remove PROMOGRAN PRISMA™ Matrix?

No, it is not necessary to remove PROMOGRAN PRISMA™ Matrix. You do however need to remove the secondary dressing as directed by your clinician.

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. The instructions for use are intended for healthcare professionals.

NOTE: This document is intended for patient self-care as guided by their HCP and is not intended to replace the Instructions for Use.

©Copyright 2020 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. PRA-PM-ALL-00365 (04/20)

