

**NOTE:** This form is for use by Wound Care Centers, Physician Offices, Home Health Agencies, Outpatient Centers, and Skilled Nursing Facilities staff to assist in evaluating patients. This form is not a medical prescription and is not a guarantee of product approval or reimbursement. Treating physician and device indications for use and labeling should be consulted.

## Wound management starts here

Please Answer **YES** or **NO** to These 3 Questions:

**1**  **YES**  **NO**

Is the wound aetiology one of the following:  
Chronic, acute, traumatic, subacute and dehisced,  
partial-thickness burn, ulcer (such as diabetic,  
pressure or venous insufficiency), flap or graft?

**2**  **YES**  **NO**

Is the wound bed  
clean, containing less  
than 20% slough/  
devitalized tissue?

**3**  **YES**  **NO**

Is the wound at  
least 2.25cm/sq?




**If All 3 Answers Are YES,**  
see reverse to determine if the ACTIVA.C™ Therapy System or the SNAP™ Therapy System may be a treatment option for your patient

**If Any Answers Are NO,**  
see reverse for information on 3M Advanced Wound Dressings

# Select the appropriate 3M Product

## Wound/Clinical Considerations (Non-Hospital)

CRITERIA	ACTIV.A.C.™ SYSTEM	SNAP™ SYSTEM
Wound Size	> 18cm x 18cm	≤ 18cm x ≤ 18cm
Wound Drainage	> 180mL/week	≤ 180mL/week (3) x 60cc cartridges/week
NPWT Initiation	Previously treated with Modern Wound Care; > Day 30	First line treatment; Day 1* (No 30 day requirement for modern wound care, No minimum depth) *may vary by payor

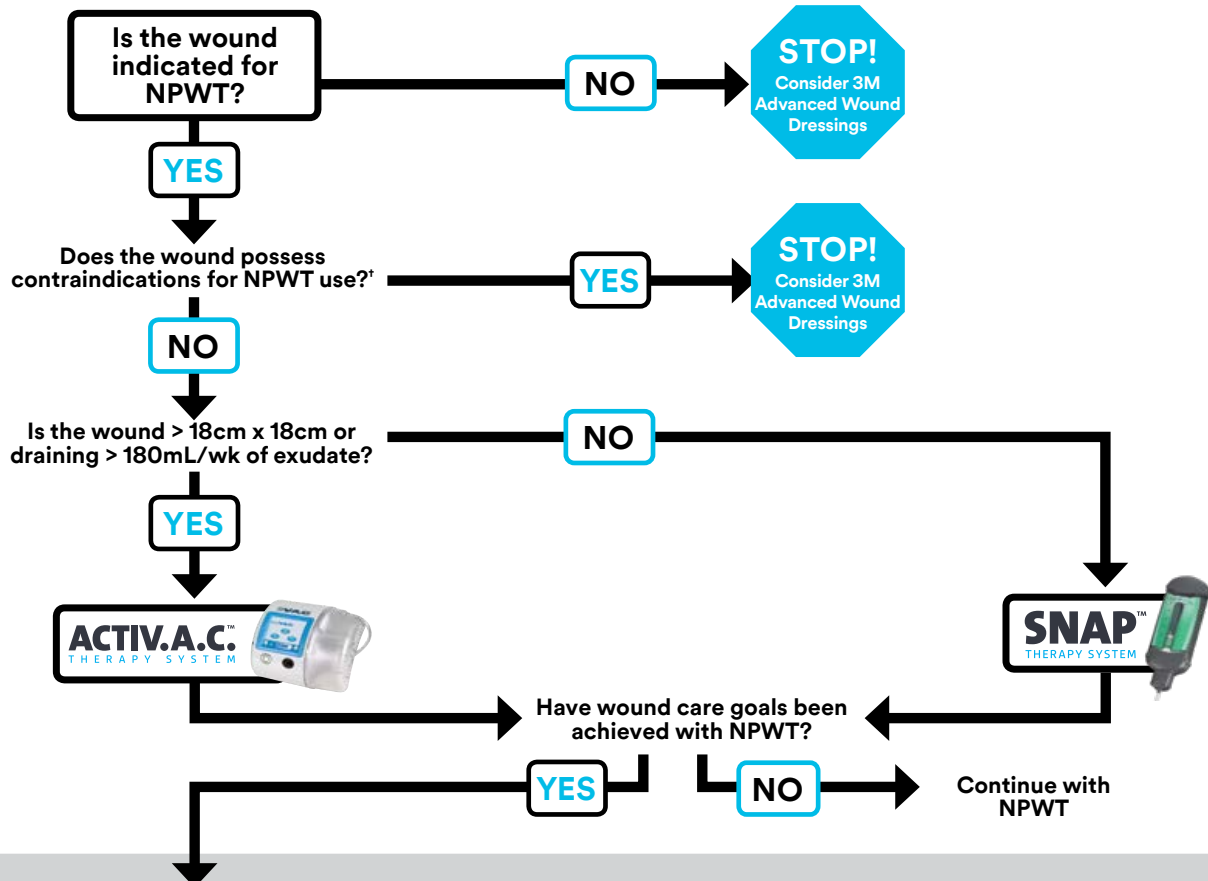

### When the Time is Right to Transition from NPWT, Consider 3M Advanced Wound Dressings

3M advanced wound dressings are appropriate when NPWT is not available or appropriate and should be selected based on clinical goal.

 <p><b>Promote Granulation</b></p> <p>PROMOGRAN PRISMA™ Wound Balancing Matrix</p>	 <p><b>Manage Bioburden</b></p> <p>SILVERCEL™ Dressing Family INADINE™ (PVP-I) Non Adherent Dressing KERRACONTACT™ Ag Dressing</p>	 <p><b>Manage Exudate</b></p> <p>KERRAMAX CARE™ Dressing Family KERRALITE COOL™ Dressing Family KERRACEL™ Gelling Fibre Dressing</p>	 <p><b>Manage Odour</b></p> <p>ACTISORB™ Dressing Family</p>
 <p><b>Minimize Adherence</b></p> <p>ADAPTIC TOUCH™ Non-Adhering Silicone Dressing ADAPTIC™ Non-Adhering Dressing</p>	 <p><b>Manage Exudate &amp; Prevention</b></p> <p>3M™ Tegaderm™ Silicone Foam Dressings</p>	 <p><b>Skin Protection</b></p> <p>3M™ Cavilon™ Skin Care Solutions 3M™ Cavilon™ No Sting Barrier Film</p>	 <p><b>Compression</b></p> <p>3M™ Coban™ 2 Layer Compression System</p>

Contact your representative \_\_\_\_\_

\*When used in conjunction with good clinical practice; utilise appropriate debridement and/or antibiotics where necessary. Untreated osteomyelitis is contraindicated for use with V.A.C.® Therapy.

†NOTE: Specific indications, contraindications, warnings, precautions, and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application.

©Copyright 2020 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. PRA-PM-EU-00278 (04/20)