

Protection through prevention

KerraPro™ helps protect at-risk patients from pressure damage to their skin

Torso

Arms

Legs

1
Strip provides protection when using face masks or other healthcare equipment

2
Sheet cushions the head or shoulder blades and can be used flat or folded

3
Sacrum relieves pressure on the shoulder and other bony prominences

4
Sacrum is perfectly shaped to reduce pressure in the sacral area

5
Heel can be used in different locations, including on the elbows and knees

6
Sheet can be cut to size and provides protection to wrists and hands

7
Versatile and flexible, wrap **Strip** around the wrist

8
Cut **Strip** to size to protect at-risk areas, including fingers, toes, hands, feet and ankles

9
Sheet reduces pressure between knees or ankles

10
Sacrum can also reduce pressure on the ankles

11
Heel can protect the back of the heel on patients lying down

12
Heel can also protect underneath the foot on patients sitting up or walking

The KerraPro range

KerraPro Pressure Reducing Pads must only be used on healthy, intact or recently healed skin.



Sheet
10x10cm
0.3 or 1.2cm thick



Heel



Sacrum/Ankle



Strip
50x2.5cm or 30x5cm
0.3cm thick



The superpowers of silicone

Silicone is incredibly hardwearing and is odourless, making it the ideal material.

Care and cleaning instructions

- Simply wash with soap and water
- Pad must be completely dry before re-application
- KerraPro can be placed in an autoclave, if this level of infection control is required
- Can be easily cut to required size or shape

KerraPro™
Pressure Reducing Pads