

Inspection and Redonning (putting on) of a Cup Style Filtering Facepiece Respirator for Reuse

Inspection



1. Perform hand hygiene per facility protocol.




2. Apply clean gloves.




3. Take care not to touch the inside of the respirator. Begin inspection according to the specific instructions for your model.




4. Inspect body for holes, tears, or other damage or soiling.

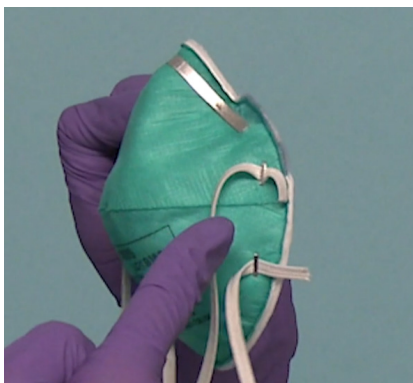


5. Inspect attachment points.

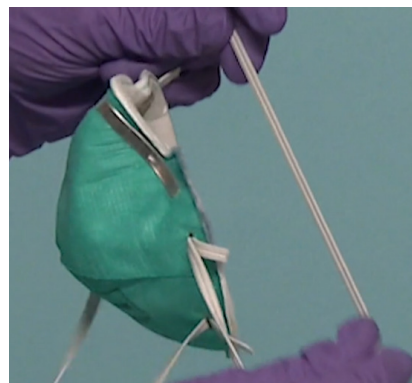

6. Inspect straps for damage and to ensure they maintain original elasticity.

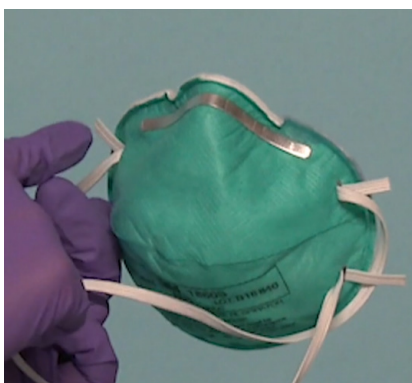

7. Inspect the noseclip to ensure it remains in original condition and is not damaged.

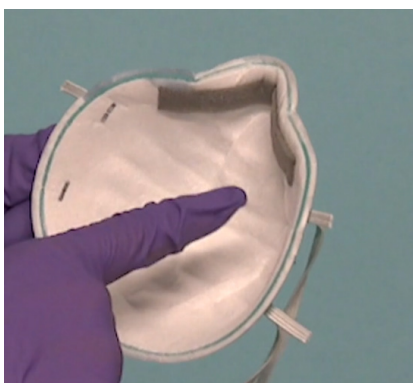

8. Inspect the nosefoam to ensure it remains in original condition and is not compressed, detached or otherwise damaged.

NOTE: You can view the video *Inspection and Redonning (putting on) of a Filtering Facepiece Respirator* [here](#).

DISCLAIMER: Always read and follow *User Instructions*.

Redonning



1. Hold the respirator in palm of the hand, with straps facing the floor.



2. Place respirator on your face, covering your nose and mouth.



3. Pull upper strap up and over your head. Place on back of head.



4. Pull bottom strap up and over your head. Place behind your neck below ears.



5. Use both hands to mold the nosepiece snugly to the face.



6. Perform user seal check according to manufacturer's instructions.



7. Remove and discard gloves according to facility protocol.



8. Perform hand hygiene per facility protocol.

NOTE: You can view the video *Inspection and Redonning (putting on) of a Filtering Facepiece Respirator* [here](#).

DISCLAIMER: Always read and follow *User Instructions*.

3M Personal Safety Division Technical Service 1 (800) 243-4630