

Doffing (Removing) a Filtering Facepiece Respirator (FFR) for Reuse and Storage

Doffing



1. Remove other personal protective equipment, including gloves.



2. Leave respirator in place.



3. Perform hand hygiene per facility protocol.



4. Apply clean gloves.



5. Tilt head forward to remove the respirator.



6. Use both hands to grab bottom strap. Pull strap out and away slowly over head.



7. Use both hands to grab the top strap. Pull strap out and away slowly over head. Keep tension on upper strap as you remove it.




8. Allow respirator to lower from face.

NOTE: Discard the respirator if damaged or soiled.

NOTE: You can view the video *Doffing (Removing) a Filtering Facepiece Respirator (FFR) for Reuse and Storage* [here](#).

DISCLAIMER: Always read and follow *User Instructions*.

Storage

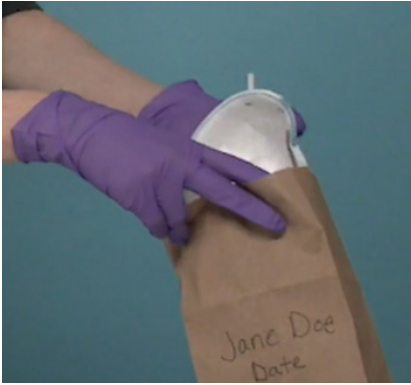

1. Taking care not to touch the inside of the respirator, place in a breathable facility-approved storage (CDC recommends hanging on a hook or placing in a paper bag—write name and date on bag).



2. Remove and dispose of gloves.



3. Perform hand hygiene per facility protocol.

NOTE: You can view the video *Doffing (Removing) a Filtering Facepiece Respirator (FFR) for Reuse and Storage* [here](#).

DISCLAIMER: Always read and follow *User Instructions*.

3M Personal Safety Division Technical Service 1 (800) 243-4630