



Basic application guide for V.A.C.[®] Dressings

For wounds without exposed vessels, organs, tendons and nerves




SENSAT.R.A.C.[™] Pad



V.A.C.[®] Drape




V.A.C.[®] GRANUFOAM[™] Dressing

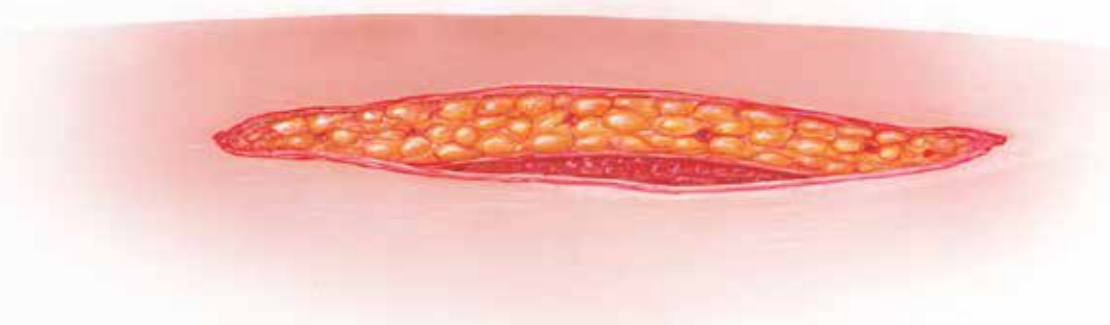

Wound Bed


Basic application guide for V.A.C.[®] Dressings

Step 1: Assess wound size and type

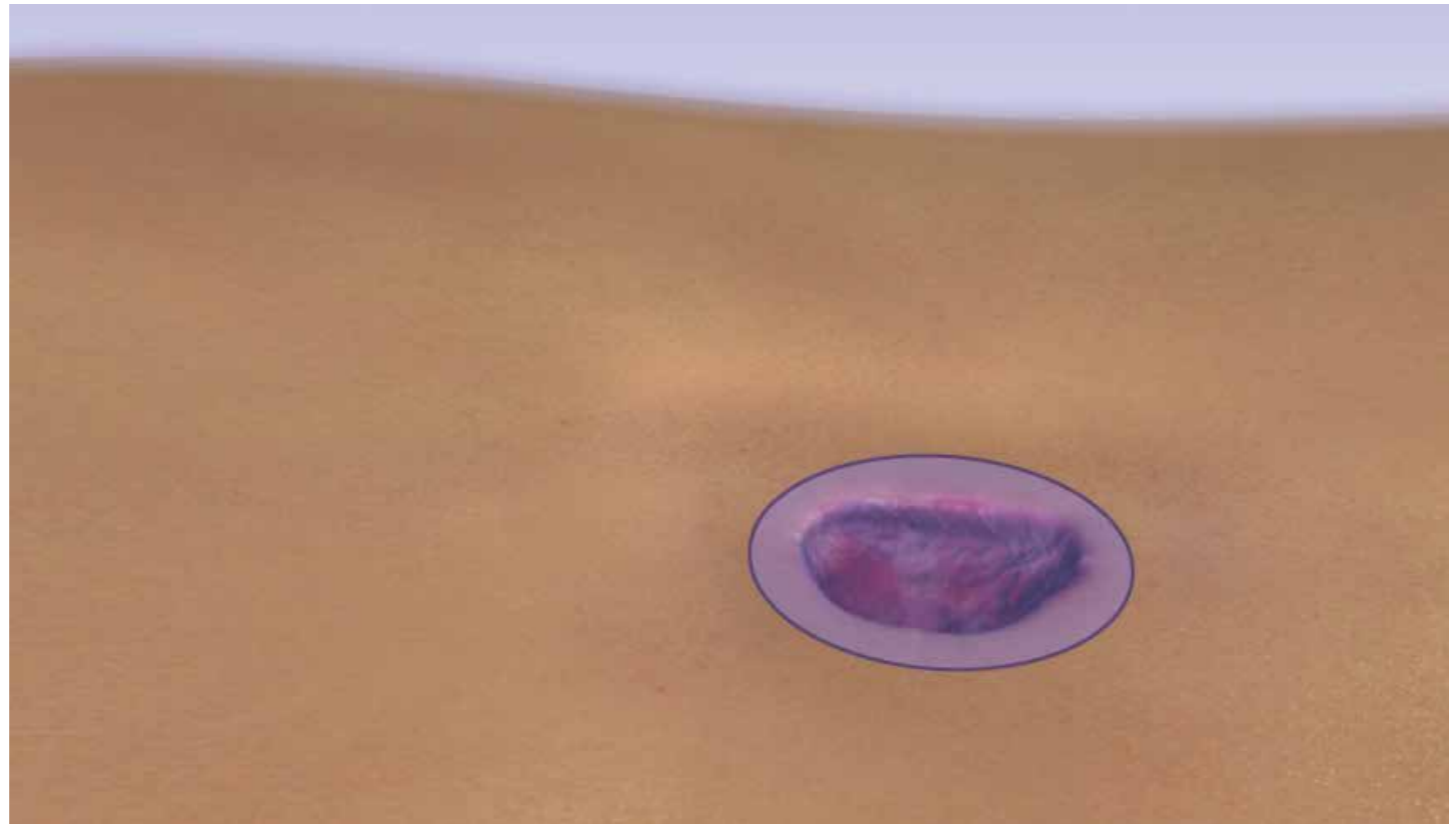

Prepare wound bed appropriately.


Step 2: Cut appropriate dressing to size

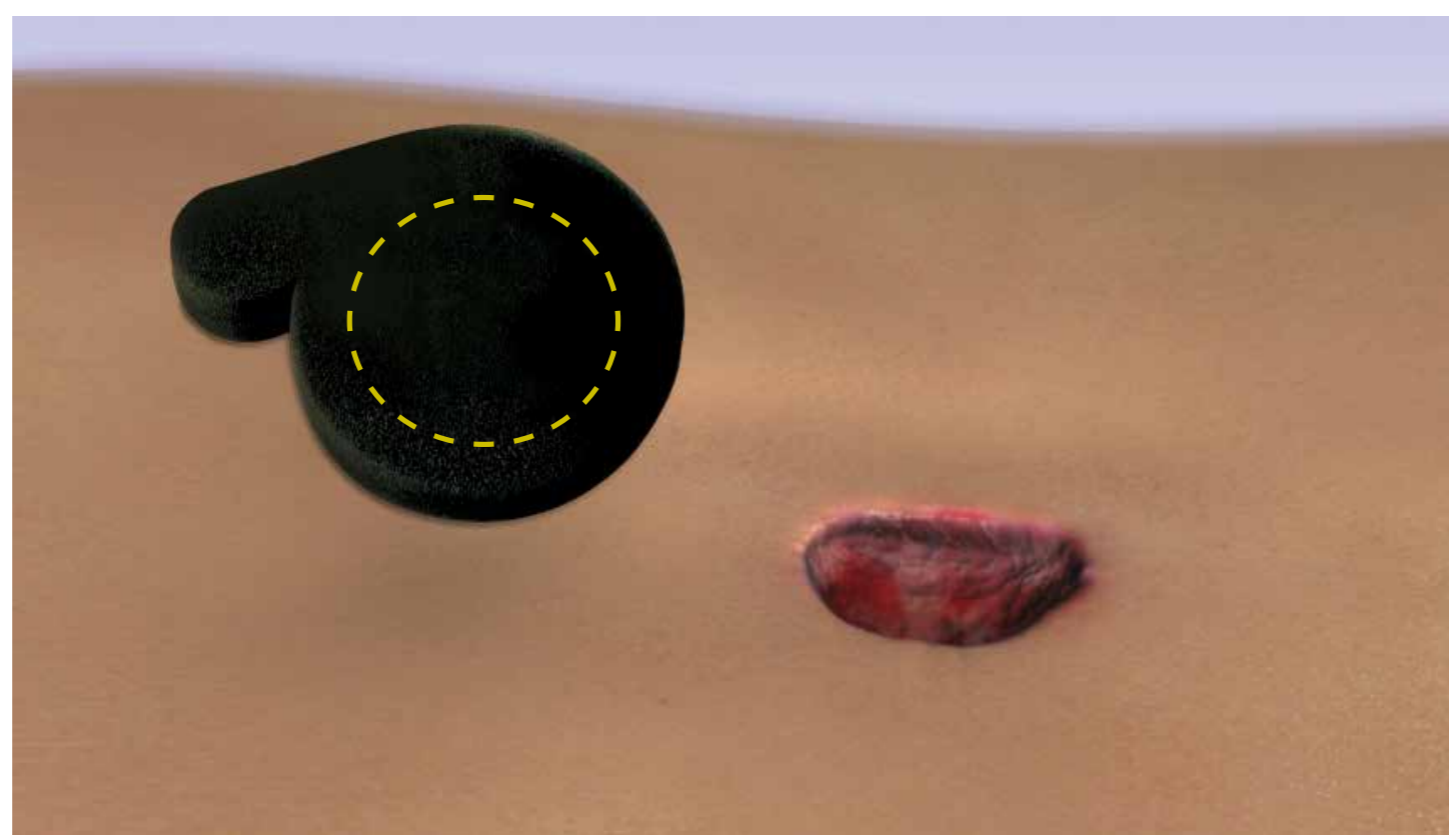

Do not cut dressing over wound as this could result in foam particles falling into the wound.


Basic application guide for V.A.C.[®] Dressings

Step 3: Place dressing in wound

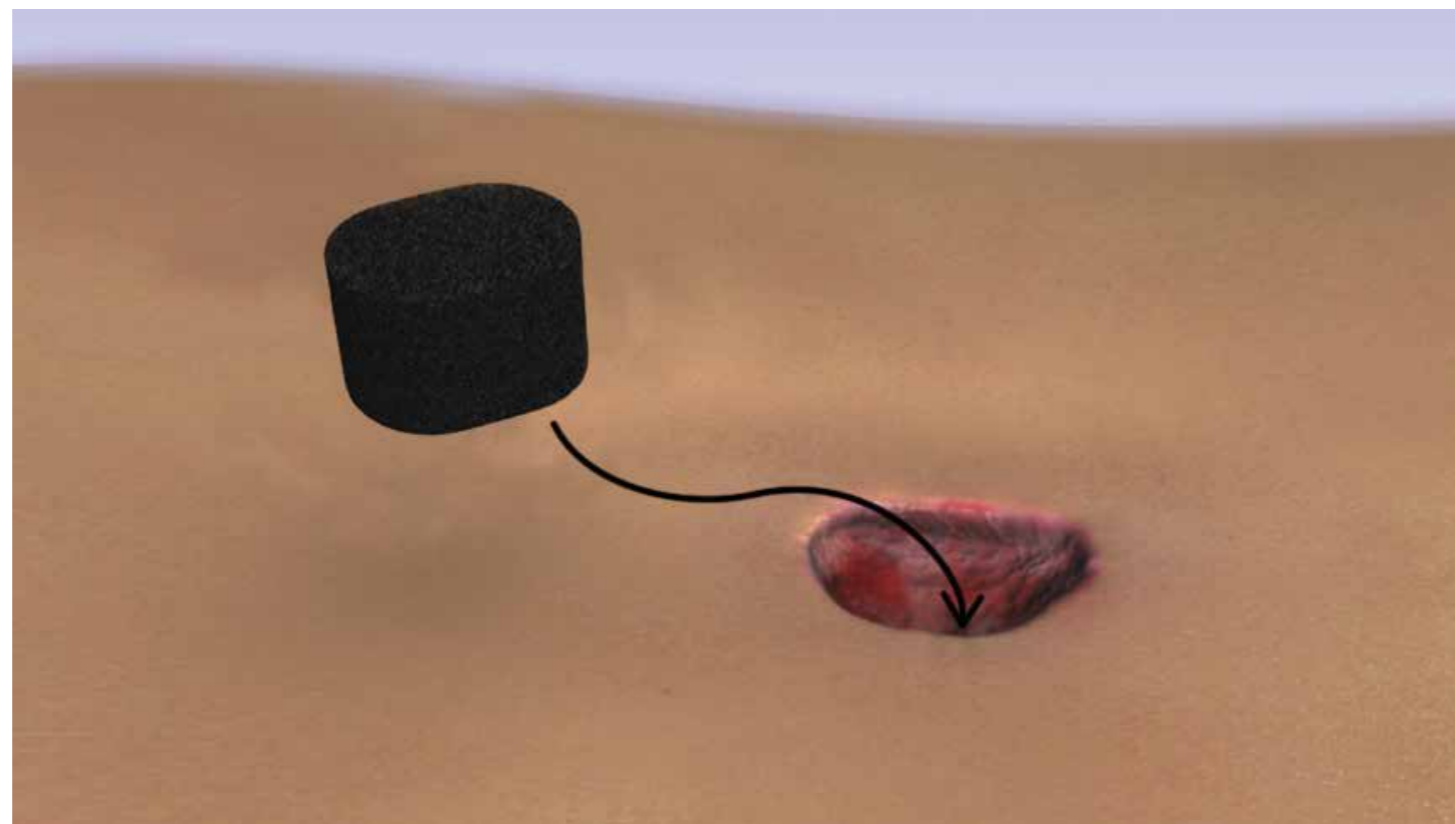

Note the total number of pieces of foam used in the wound.


Step 4: Trim V.A.C.[®] Drape to size

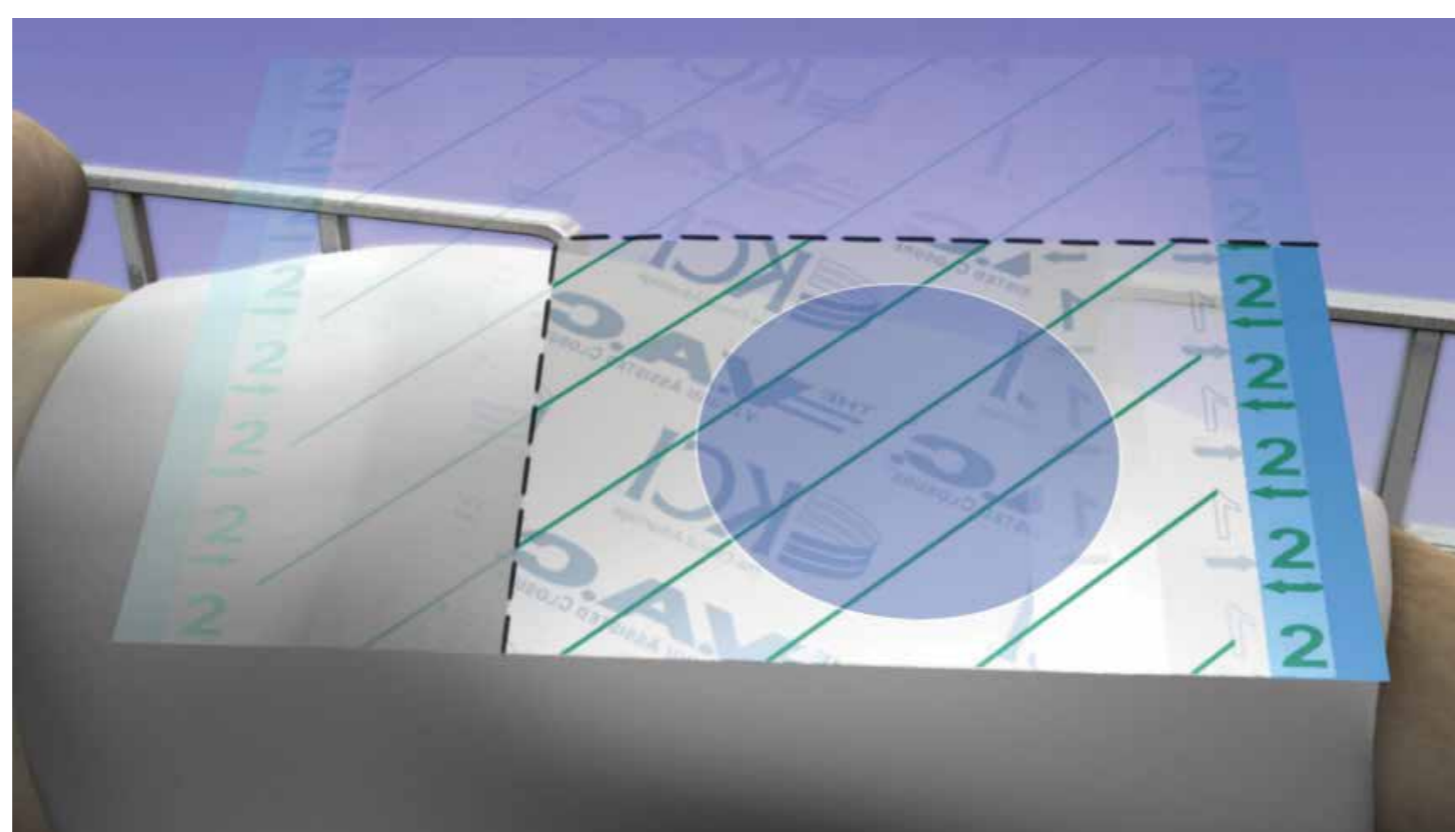

Peel back one side of Layer 1 and place adhesive side down over wound. Remove the remaining side of Layer 1, the green striped stabilization Layer 2 and the perforated Blue Handling Tab(s)


Basic application guide for V.A.C.[®] Dressings

Step 5: SensaT.R.A.C.[™] Pad Application

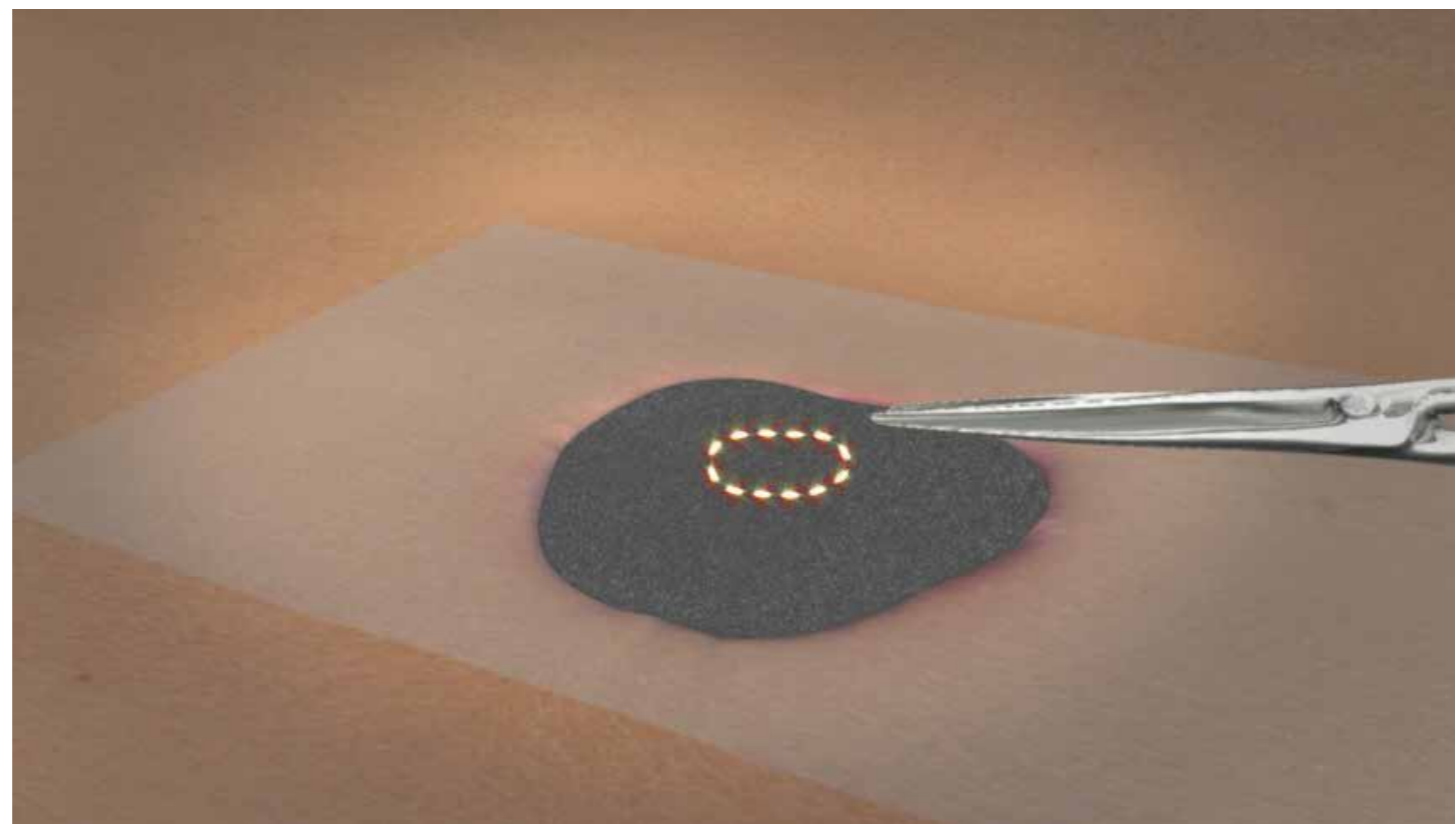

Identify application site for SensaT.R.A.C.[™] Pad.


Step 6: Pinch drape and cut a 2.5cm round hole

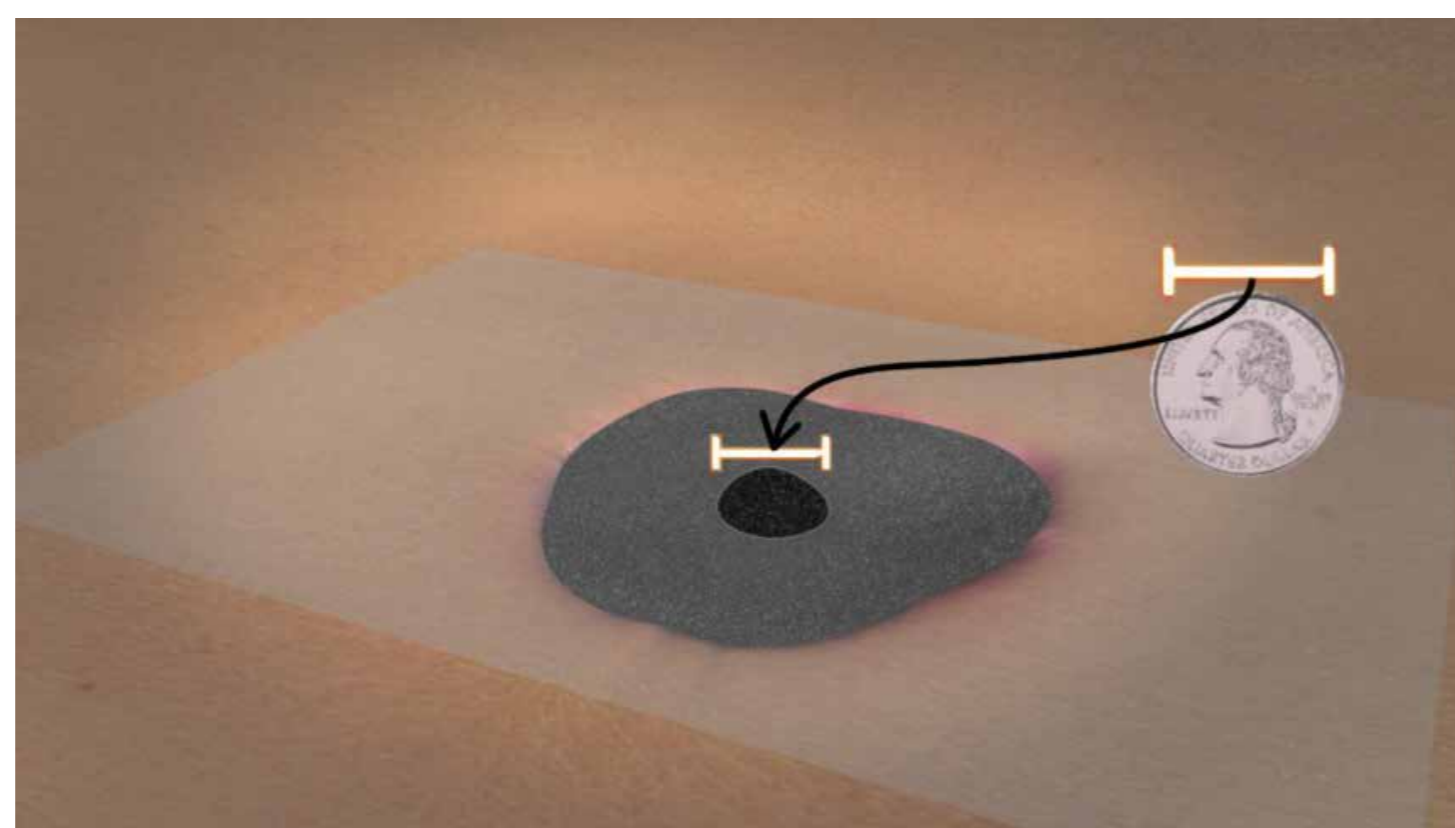


Consider the diameter of a Quarter for size reference.

Basic application guide for V.A.C.[®] Dressings

Step 7: Remove backing Layers 1 and 2 from SENSAT.R.A.C.[™] Pad

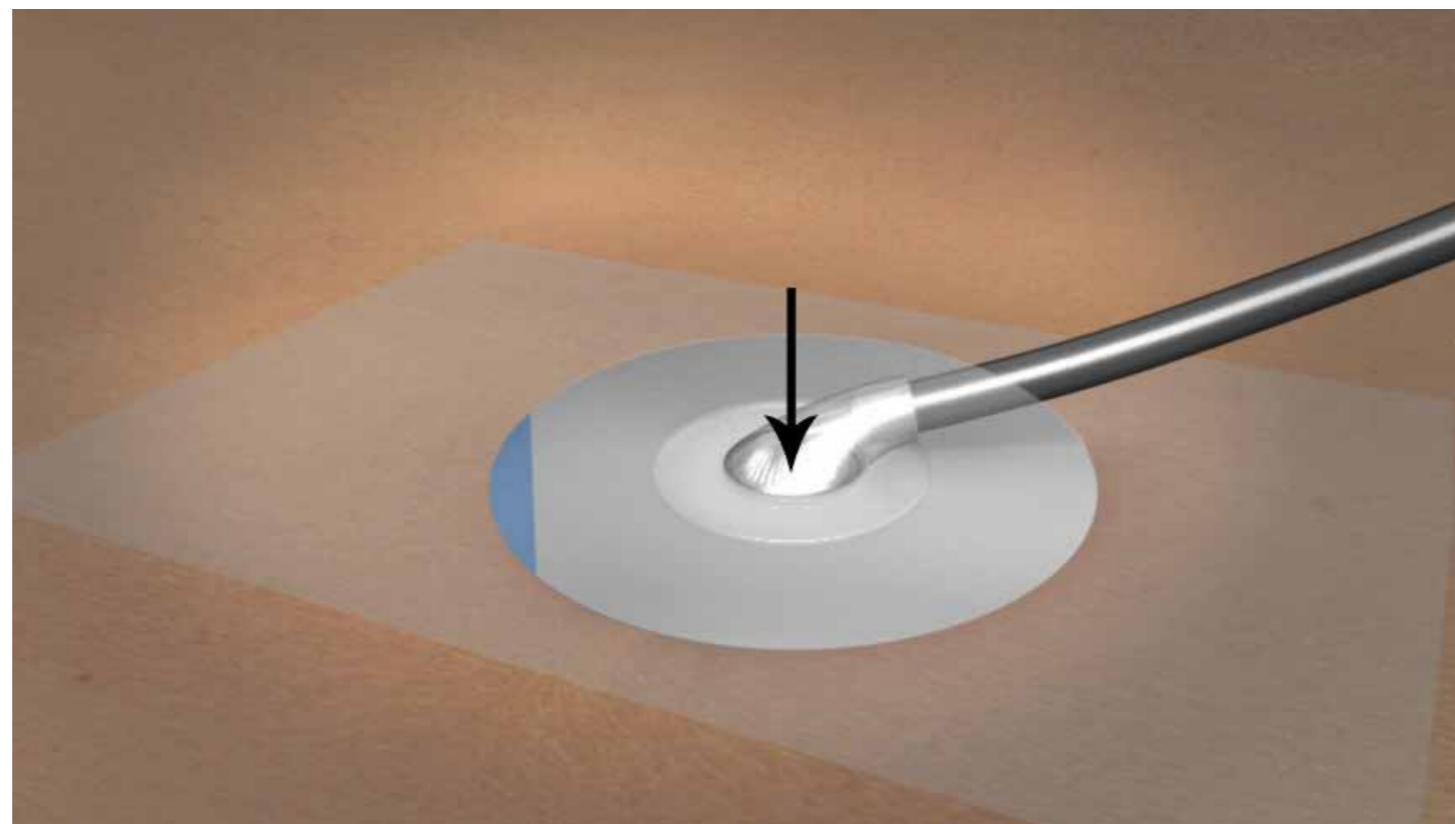

Step 8: Place SENSAT.R.A.C.[™] Pad opening directly over hole

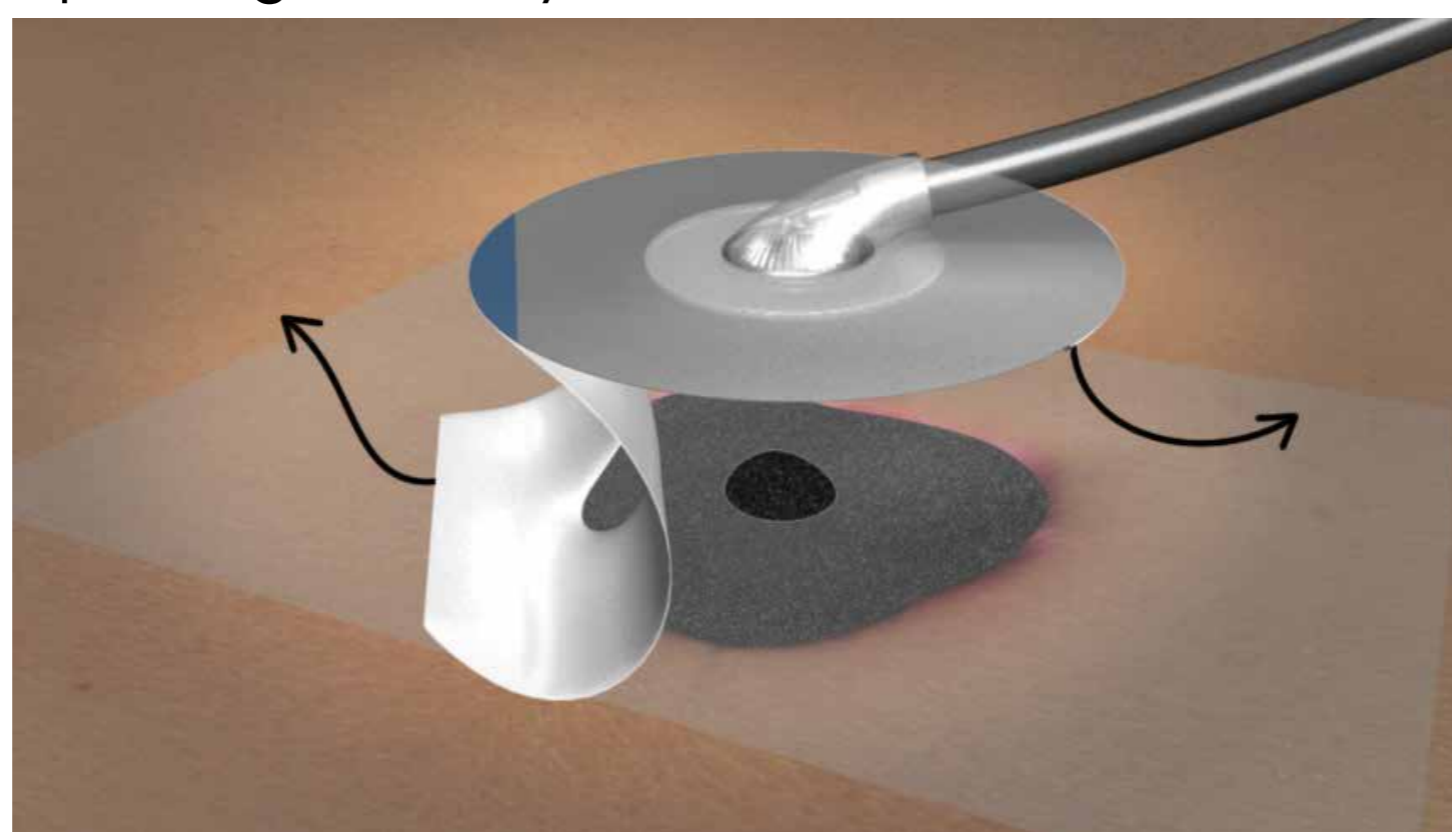


Pull blue tab to remove stabilization layer. Record date and number of foam pieces used on the Foam Quantity Label and place in the patient record.

Basic application guide for V.A.C.[®] Dressings

Step 9: Connect SENSAT.R.A.C.[™] Pad tubing to canister tubing



Step 10: Turn on power to V.A.C.[®] Therapy Unit

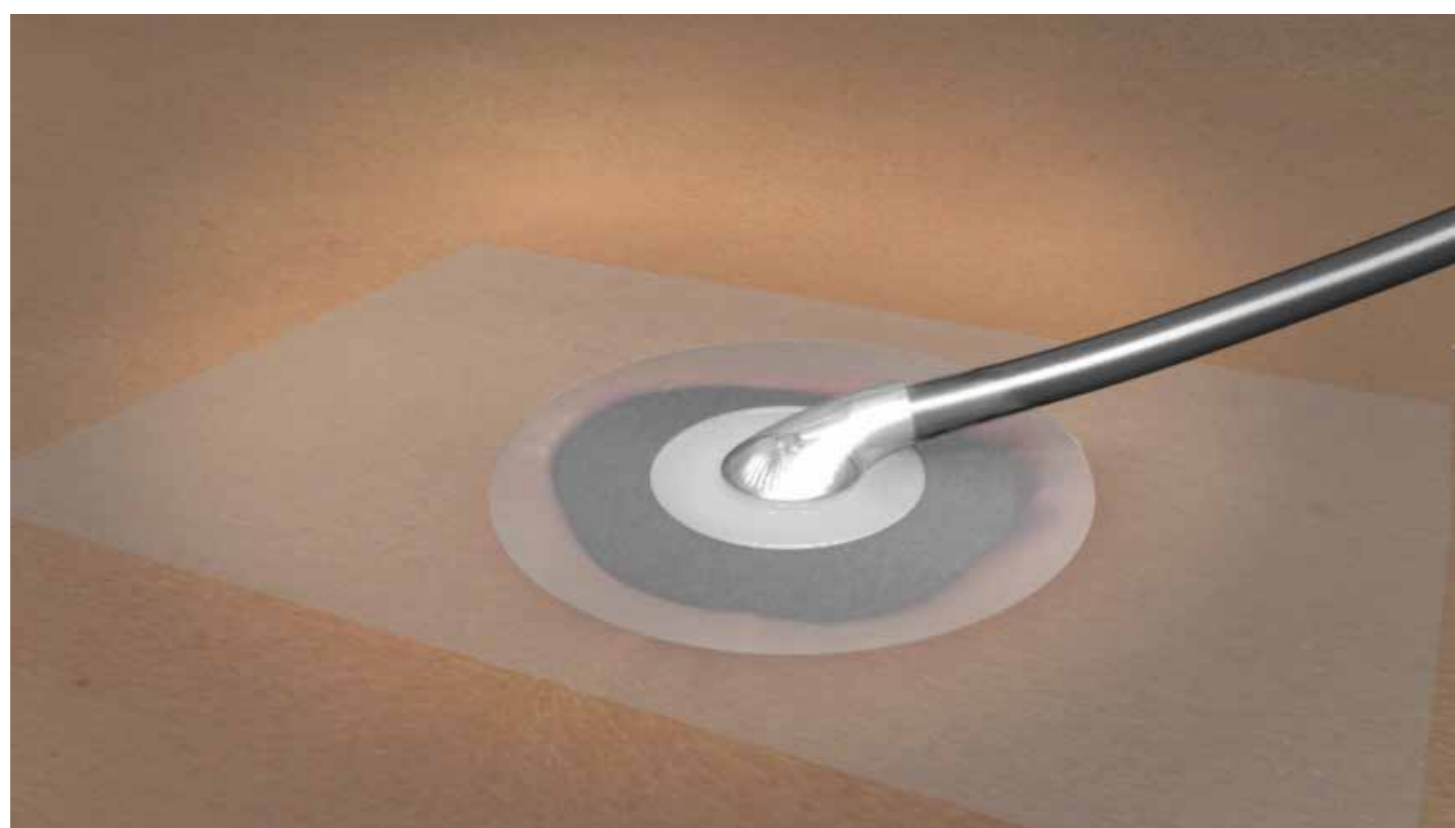

Set to the physician prescribed therapy settings to initiate therapy.

Instructions for Use and Safety Information: This is a quick reference showing the basic Steps in a typical dressing application. As with any device, it is important to read and understand the detailed instructions for use and safety information (including information on exposed vessels, organs, tendons, nerves, and infection) applicable to your V.A.C.[®] Therapy Unit and dressing application that can be found with the therapy unit, disposables carton or in the V.A.C.[®] Therapy Clinical Guidelines.

Questions and Information: For additional, detailed information, please refer to specific dressing instructions for use (contained in disposables cartons) and the V.A.C.[®] Therapy Clinical Guidelines. These sources provide information concerning the proper application of all V.A.C.[®] Therapy Dressings including advanced applications and techniques. If you have product questions, please call **800-275-4524**.

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for KCI products and therapies. Please consult a physician and product instructions for use prior to application. Rx only.

©Copyright 2020 3M. All rights reserved. 3M is a mark and/or registered mark of 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. Please recycle. Printed in USA. PRA-PM-US-02523 (06/20)

