



Easy-to-Read 3M Steam Chemical Integrators.

3M™ Attest™ Steam Chemical Integrators, Type 5


Unprocessed

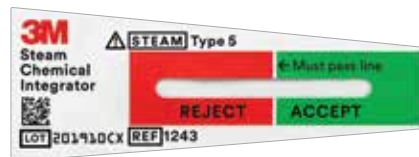

Processed

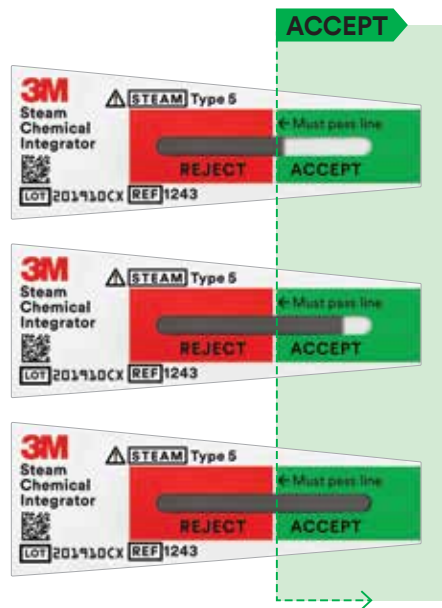

Processed

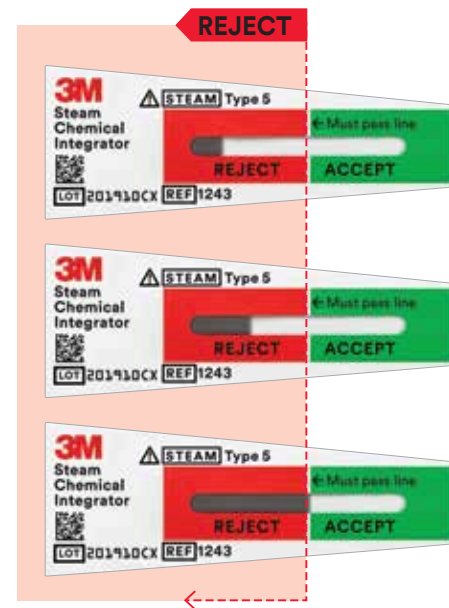


3M™ Attest™ Steam Chemical Integrators, Type 5 comply with ISO 11140-1:2014.

The color bar has reached the **ACCEPT** window in all three samples shown above, indicating that the necessary conditions for sterilization have been met. **If the color bar crosses into the ACCEPT region, it is considered a pass.**


The color bar is in the **REJECT** window, indicating that the necessary conditions for sterilization have not been met. The pack should be reprocessed and the cause of sterilization failure should be investigated. **If the color bar is in the REJECT region or on the line, it is considered a fail.**

3M™ Comply™ (SteriGage™) Steam Chemical Integrator


Unprocessed

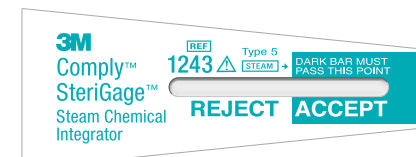

Processed

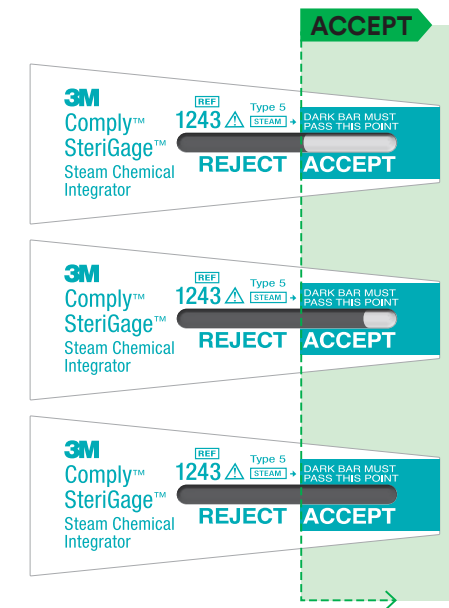

Processed

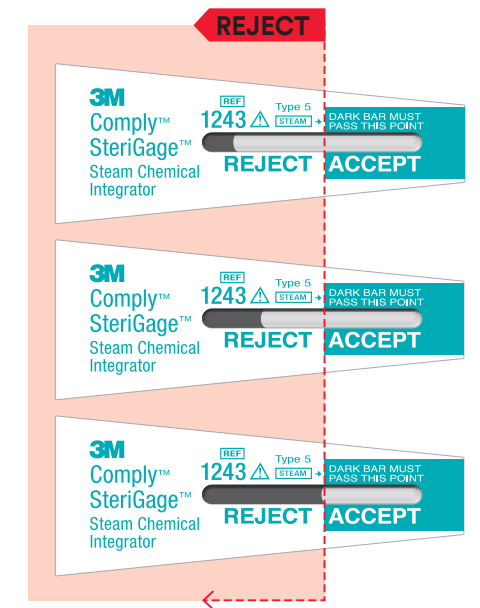

3M™ Comply™ SteriGage™ Steam Chemical Integrators are Type 5 (Category i5) Integrating Indicators as categorized by ISO 11140-1:2014.

The color bar has reached the **ACCEPT** window in all three samples shown above, indicating that the necessary conditions for sterilization have been met. **If the color bar crosses into the ACCEPT region, it is considered a pass.**

The color bar is in the **REJECT** window, indicating that the necessary conditions for sterilization have not been met. The pack should be reprocessed and the cause of sterilization failure should be investigated. **If the color bar is in the REJECT region or right on the line, it is considered a fail.**