

**3M** Science.  
Applied to Life.™



3M™ Incognito™ Appliance System  
**Creating confidence for  
you and your patients.**

---


**Incognito**™  
Appliance System

# We believe that no one should have to compromise on their smile.

**That's why the 3M™ Incognito™ Appliance System is 100% customised for each patient.**

- Created with a combination of digital workflows.
- Gold alloy brackets, designed for optimal conformance to teeth.
- Innovative custom archwires.

**Giving you the control and confidence to do what you do best – create smiles you know your patients will love.**

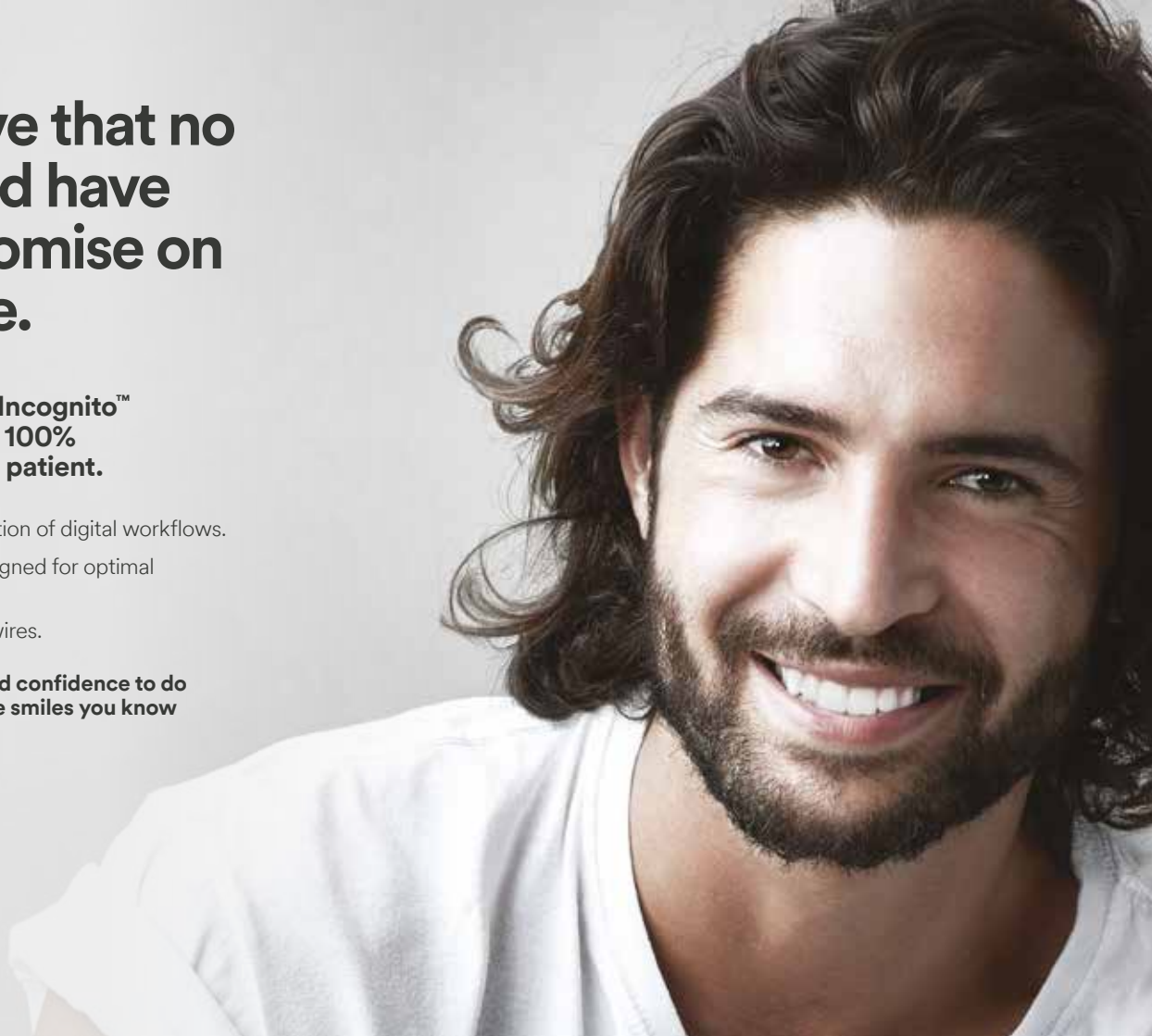



# Aesthetics without sacrificing control.

**As patient demand for aesthetic treatment continues to grow, so does your need for uncompromised control.**

You are the expert, you recommend the right treatment plan to a patient, taking in consideration patient's needs for aesthetics, treatment time and financial investment.

Combining the mechanics of traditional brackets with the majority of appliances and attachments on the lingual side of the teeth, the Incognito System brings together the control that doctors require for treatment with the aesthetics of hidden braces that patients seek, delivering an effective, aesthetic orthodontic experience.

**Expand your aesthetic orthodontic treatment options by adopting Incognito appliances along with your ceramic labial brackets or aligners.**

# Digital workflow for precise treatment results.

## Scanner

Take your impressions for the Incognito appliance system digitally with a variety of intraoral scanners.



## 3M™ Unitek™ Treatment Management Portal

Ordering software leads you conveniently through the process.

View, evaluate and approve setups in 3D.

## Digital Setup Lab

Precise digital setups are produced by 3M.



## Computer-Aided Bracket Design

Fully customised bracket bonding pad and bracket slots are designed on the lingual surface of the digital model.



## Bracket Manufacturing

The brackets are precision made from a dental gold alloy, using a multi-step process.





### 3M™ Incognito™ Clear Precision Tray

This precise bonding tray is produced from digital data, increasing bracket positioning precision.

### Archwire Bending

Custom archwire geometry is determined by the CAD/CAM software.

### Self-3D Print Models Service

Practitioners can now request, download and 3D print the setup files with morphed gingiva for review or to make retainers and positioners.

### Quality Assurance and Shipping

Strict quality control measures are taken at each stage of the manufacturing process to ensure the highest product quality.





## Digital treatment management.

The 3M™ Unitek™ Treatment Management Portal is an advanced orthodontic software platform for entering your treatment plan and efficiently ordering the 100% customised Incognito system.

### Featuring:

- Precise 3D setup review and overlay capabilities.
- Easy customised ordering.
- Helpful communications and messaging with the Incognito appliance lab.
- Robust measurement and analysis tools:
  - Overlay views
  - Biteplane analysis
  - Bolton analysis

**Order your files for Self-3D Models Print.**

3M™ Incognito™  
Appliance System



3M™ Incognito™ Lite  
Appliance System



3M™ Incognito™  
Brackets with Tip Bar

# 100% customised.

**The 100% customised, invisible, fixed orthodontic appliance from setup to bracket and wire.**

**Available as full appliance from 7-7 or Lite version from 3-3.**

Customised to the unique prescription of each patient, it can be used to treat a wide range of orthodontic case types.

The Incognito lite system may be a great option for mild to moderate anterior crowding and relapse cases.

The tip bar bracket in combination with 3M™ AlastiK™ Lingual Ligatures brings 75% improvement in tipping efficiency, enhancing angulation and tip control for better outcomes.\*

\*Source: Data on file.



## Next level precision.

**Successful outcomes begin with a precise start. The Incognito system relies on the brackets being placed in the correct position on the tooth.**

We have developed a way to build the indirect bonding tray using the same digital data used for designing the brackets. The result is accurate placement of the brackets.

The 3M™ Incognito™ Clear Precision Tray combines a hard outer shell with a soft inlay manufactured from digital data. The brackets are inserted into the tray, which can be cut and used for accurately replacing brackets.



**Easy to see on gold brackets.**

**Optimised cross section delivers ideal forces.**

**Innovative design for easy ligating.**

**Proprietary design for easy application.**

**Recommended by 90% of practitioners.**

**Replaces overties.**

**Convenient tab enables secure grip.**



## As strong as an overtie.

**3M™ AlastiK™ Lingual Ligatures, used as plain ties, apply similar vertical and tip control when compared to overties used with powerchain.\***

Due to the design of the ligatures, the application is easier compared to current ligation techniques and provides patient comfort at the same time.

\*Source: 3M internal data

# Precision at your fingertips.

3M™ Unitek™ Lingual Instruments have been developed to provide the versatility and precision needed for today's treatment requirements.

Designed with input from leading lingual orthodontists, Unitek instruments are a perfect complement to the Incognito appliances.

Defining features:

- **Long handles for access to difficult to reach areas**
- **Small instrument heads to accommodate the tight interproximal space on the lingual side**
- **Smooth contours designed for patient comfort**

## Lingual Instruments



**Unitek Lingual Anterior  
Debonding Pliers**



**Unitek Lingual Weingart Utility Pliers  
with 60° Bend**

## Adhesive



**3M™ RelyX™ Unicem 2  
Automix Self-Adhesive Universal  
Resin Cement**

Reduced bond failure

Moisture tolerance

Metal primer application  
not needed

Automatic mixing

Filled bonding material

One adhesive for bonding  
and rebonding



## Supporting you with certification and education.

Contact us today to find out how to become a 3M certified Incognito system practitioner. Once you are certified, you'll have access to the TMP and be able to place Incognito system orders. 3M also provides continuing education to help deepen your expertise with the system.

The 3M™ Incognito™ System Certification Course combines a mixture of online and hands-on practical learning, providing a modern and flexible approach to our certifying Incognito users.

We offer a range of Incognito Courses via the 3M<sup>SM</sup> Health Care Academy, you will need to register for an account to view these courses.

Contact your local 3M representative for more information.

# We are here to support you from start to finish.

For more information please contact your 3M Oral Care Representative.



**3M Australia Pty Ltd  
Oral Care Division**

Building A, 1 Rivett Road,  
North Ryde, NSW 2113

Phone 1300 363 484  
Web [3M.com.au/ortho](http://3M.com.au/ortho)

**3M New Zealand Ltd  
Oral Care Division**

94 Apollo Drive,  
Rosedale, Auckland 0632

Phone 0800 441 622  
Web [3M.co.nz/ortho](http://3M.co.nz/ortho)

3M, Incognito, AlastiK, RelyX and Unitek are trademarks of 3M.  
© 3M 2020. All rights reserved.  
AV011496617

**Incognito**<sup>™</sup>  
Appliance System