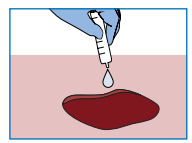
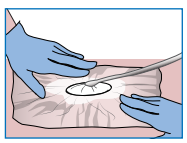
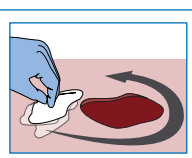
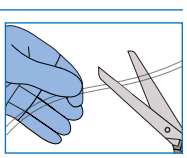
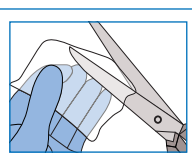
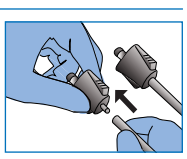
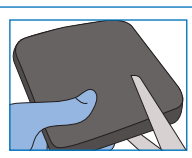
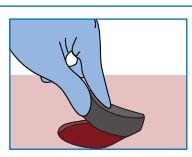
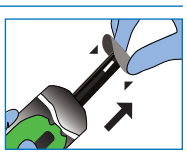
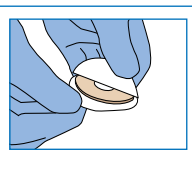
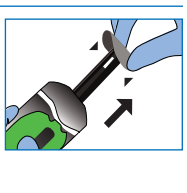

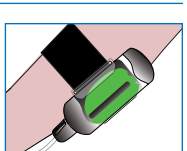


## 3M™ Snap™ Therapy System Dressing Application Guide\*

- |   |   |   |   |
|---|---|---|---|
| <p><b>1</b> Prepare the wound bed and periwound skin per institutional protocol and irrigate wound bed thoroughly with normal saline.</p>   |    | <p><b>8</b> Remove backing and place the 3M™ Snap™ Advanced Dressing over the wound and seal.</p>   |    |
| <p><b>2</b> If necessary, apply a skin protectant to the surrounding skin and allow to dry.</p>   |    | <p><b>9</b> At a right angle, cut the dressing tubing to the desired length, if required.</p>   |    |
| <p><b>3</b> If necessary, cut a single layer of wide-mesh, non-adherent dressing to the size of the wound and place on the wound bed.</p>   |    | <p><b>10</b> Fully insert the tube fitting into the tubing.</p>   |    |
| <p><b>4</b> Cut the foam interface to fit the size and shape of the wound.</p>  |    | <p><b>11</b> Connect the 3M™ Snap™ Therapy Cartridge to the tube fitting using both hands.</p>  |    |
| <p><b>5</b> Place foam interface into wound cavity. Foam interface should fill the wound cavity and extend above the wound margins.</p>   |   | <p><b>12</b> To activate the Snap Therapy Cartridge, push down the Activation/Reset Key to release and remove the key from the Snap Therapy Cartridge. Cartridge may need to be primed multiple times depending on size of the dressing.</p>  |   |
| <p><b>6</b> If necessary, remove the 3M™ Snap™ SecurRing™ Hydrocolloid Skin Barrier from packaging. Remove release liner from both surfaces.</p>  |  | <p><b>13</b> Check negative pressure operation. The 3M™ Snap™ Therapy System is working properly if:</p> <ul style="list-style-type: none"> <li>• Green capacity indicator is both visible and stationary in the chamber window</li> <li>• Dressing has a “sucked down” appearance</li> <li>• Dressing feels hard to the touch</li> </ul> |  |
| <p><b>7</b> If necessary, gently mold, stretch, compress or roll the Snap SecurRing Hydrocolloid Skin Barrier to any shape to fit around the outside of the wound, leaving approx 1cm of intact skin between the Snap SecurRing Hydrocolloid Skin Barrier and the wound margin.</p> |  | <p><b>14</b> Secure the Snap Therapy Cartridge to the patient's extremity or belt using the 3M™ Snap™ Therapy Strap (if being utilised).</p>  |  |

Problem	Possible Causes	Possible Solutions
<p>Airtight seal is <b>not</b> present at the dressing, and <b>red</b> Pressure Discharge Indicator is visible <b>or green</b> Capacity Indicator does not remain stationary in the chamber window</p>	<p>Tubing connections may not be secure <b>and/or</b> Leak may be present at the hydrocolloid-to-skin interface</p>	<ul style="list-style-type: none"> <li>• Check that tubing connections are secure</li> <li>• Smooth dressing with fingers to flatten wrinkles</li> <li>• Seal dressing edges with additional adhesive drapes</li> </ul>
<p>Airtight seal is <b>not</b> present at the dressing (dressing is not drawn down in appearance) and <b>green</b> Capacity Indicator is stationary</p>	<p>Tubing may be occluded</p>	<ul style="list-style-type: none"> <li>• Examine tubing for possible occlusion or kinks. If found, straighten tubing or replace Snap Advanced Dressing.</li> <li>• Occlusion may reside in dressing; replace wound dressing</li> </ul>

\*For safety information and complete instructions for use, please refer to the IFU.

For additional information, please contact your local representative.

**NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a physician and product instructions for use prior to application. This material is intended for healthcare professionals.**