




3M™ Attest™ Vaporized Hydrogen Peroxide Tri-Metric Chemical Indicator, Type 4


Unprocessed




Processed

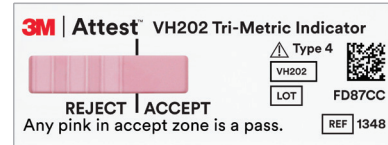

ACCEPT

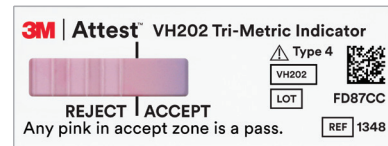

ACCEPT

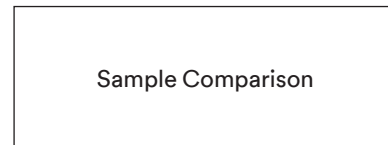

If a color change toward pink has entered the ACCEPT zone it is considered a pass.

Please reference the instructions for use for additional information.


Processed



REJECT



REJECT

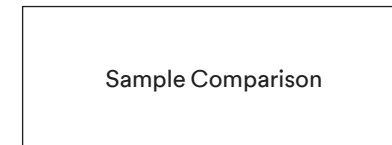

If a color change toward pink has not entered the ACCEPT zone, it is considered a fail.

Please reference the instructions for use for additional information.

3M™ Attest™ Vaporized Hydrogen Peroxide Tri-Metric Chemical Indicators, Type 4 comply with ISO 11140-1:2014.

© 2020 3M. All rights reserved. 3M and Attest are marks and/or registered marks of 3M. Unauthorized use prohibited. Used under license in Canada.