



12 CLINICAL EVALUATORS

425 TOTAL USES

99% CLINICAL RATING

**Key features:** Dual-cure cement • Both adhesive and self-adhesive modes  
• Four shades • Innovative, waste reducing syringe design

## Description

**3M™ RelyX™ Universal Resin Cement** is a dual-cure universal cement with an innovative syringe design.

- It can be used in either the adhesive or self-adhesive mode for cementation
- Kit contains **3M™ Scotchbond™ Universal Plus Adhesive** for adhesive cementation
- Available in shades A1, WO, A30, and TR, which coordinate with the **3M™ RelyX™ Try-In Pastes**
- Adhesive strength can be enhanced further with the use of 35% phosphoric acid

## Indications

Use of **3M™ RelyX™ Universal Resin Cement** in Self-Adhesive Mode or Adhesive Mode with **3M™ Scotchbond™ Universal Plus Adhesive** for final cementation of:

- All-ceramic, composite, or metal inlays, onlays, crowns, and bridges
- Ceramic, glass fiber-reinforced composite or metal posts, as well as screws
- All-ceramic, composite, or metal restorations on implant abutments

Use of **3M™ RelyX™ Universal Resin Cement** in Adhesive Mode with **3M™ Scotchbond™ Universal Plus Adhesive** for final cementation of:

- All-ceramic or metal Maryland bridges and 3-unit inlay/onlay bridges
- All-ceramic or composite veneers and occlusal veneers

## Clinical Tips

- Do not allow cement to cure too much when tack curing or it will be very difficult to remove excess
- Extrude from tip slowly with light pressure
- The most difficult thing about resin cements is cleaning up interproximally. Clean interproximal areas first, whenever possible



## Unique Attributes

- Redesigned tip significantly reduces wasted material in the mixing tip
- Truly a universal cement as it can be used in either Adhesive or Self-Adhesive mode and used for any substrate

**“THE NEW SYRINGE AND TIP ARE A WONDERFUL COMBINATION! EXCELLENT ENGINEERING.”**

## Evaluators' Comments

*“The syringe is super comfortable to hold and has an indicator on the plunger to show when it is empty.”*

*“It's great to have one cement with options for for self-adhesive and adhesive bonding.”*

*“The unique mixing tip allows for minimal waste.”*

*“Having one material to use for all cases is great.”*

*“The 6-minute wait for self-cure was my only complaint.”*

*“The choice of shades was helpful.”*




Bond Strength (MPa)


**3M RELYX UNIVERSAL RESIN CEMENT**

- Self-adhesive bond strengths of the 3M cement to dentin, enamel and zirconia substrates are the highest of any self-adhesive cements tested with this method by DENTAL ADVISOR.
- Adhesive bond strength to dentin and enamel was excellent, and in particular, the zirconia bond strengths are the highest among the universal adhesives tested.

### Evaluation Summary:

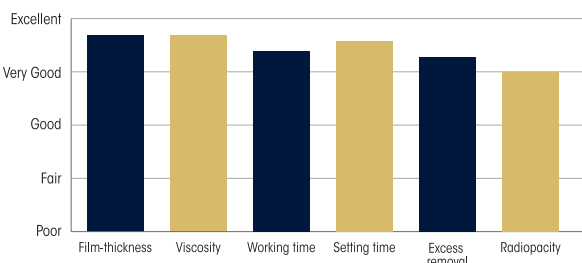

### Compared to Competitive Products:

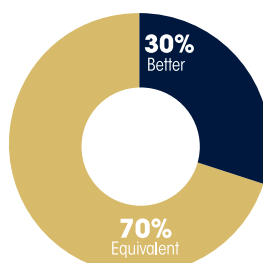

### Consultants who would:

**100%** Recommend to a colleague

**Consultants who would want to stock in office:**

**44%** Yes, instead of current product

**56%** Yes, in addition to current product