



39 CLINICAL EVALUATORS
1191 TOTAL USES
96% CLINICAL RATING

Key features: Next generation of 3M™ Scotchbond™ Universal Adhesive

- Single bottle universal adhesive and primer for all dental materials; covers all direct and indirect bonding indications

Description

3M™ Scotchbond™ Universal Plus Adhesive is a light-cured, single-component dental adhesive:

- Next generation of 3M™ Scotchbond™ Universal Adhesive
- The first radiopaque, universal adhesive available on the market
- Compatible with light-, dual-, and self-cured composite materials, cements, and core-build-up materials
- Can be used with self-etch, selective-etch, and total-etch techniques

Indications

- Bonding of light-, dual-, and self-cured composite or compomer restorations
- Root surface desensitization
- Bonding of pit and fissure sealants
- Protective varnish for glass ionomer restorations
- Repair of composite and compomer restorations
- Sealing of cavity preparations prior to amalgam placement

Unique Attributes

- First radiopaque universal adhesive
- Bonds and seals caries-affected dentin, supporting minimally invasive preparations
- BPA and BisGMA free
- High bond strength to etched glass ceramic; equivalent to silane
- No dual cure activator needed



Clinical Tips


- Gently air dry until adhesive layer appears glossy and uniform
- Treat it similar to 3M™ Scotchbond™ Universal Adhesive

“IT HANDLED JUST LIKE SCOTCHBOND UNIVERSAL AND I LOVE THE RADIOPAQUE FEATURE.”


Evaluators' Comments

- “It did not run all over the tooth when being air dried - it stayed where I put it.”
- “After drying, it was easy to tell if you have a nice layer of material over the entire surface to be bonded.”
- “The re-designed bottle delivers a very small amount at a time.”
- “I did not experience any post-op sensitivity.”
- “You can use it with many more bonding procedures without the need for a separate activator.”
- “I like its slightly thicker consistency. This gives me the feeling that I am properly coating the cavity preparation.”
- “Can pool at line angles if you don't thoroughly air dry.”


DENTAL ADVISOR RESEARCH RESULTS

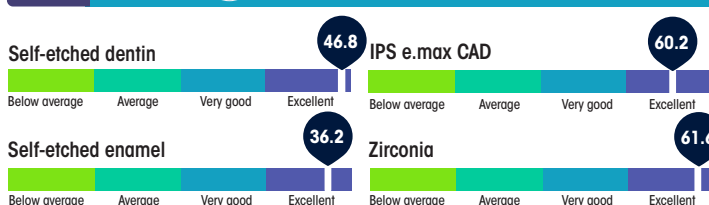

Evaluation Summary:

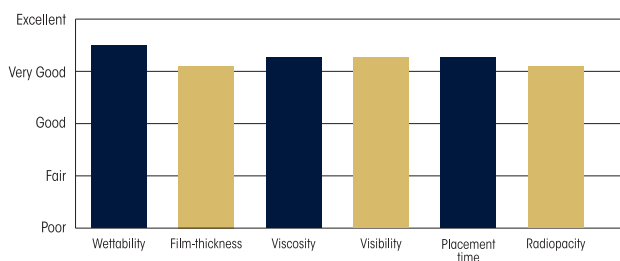

Compared to Competitive Products:

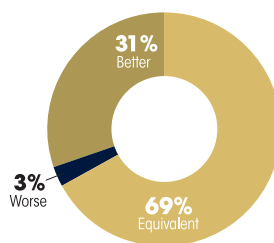

Consultants who would:

- 100%** Recommend to a colleague
- Consultants who would want to stock in office:**
 - 41%** Yes, instead of current product
 - 43%** Yes, in addition to current product
 - 14%** No, however I might want to order it for certain cases