

3M Science.
Applied to Life.™

The difference is real with the 3M™ Snap™ Therapy System

Consider early intervention with the Snap Therapy System - disposable negative pressure wound therapy (dNPWT) that makes a real difference for more patients.



The 3M™ Snap™ Therapy System offers real patient customization and real-world compatibility



The Snap Therapy System adapts to each patient and wound


- **Manage more wounds:** supports an environment to promote healing of small, low-exuding wounds with customizable dressings that fit even the most difficult anatomies such as:



Foot
Abscess



Calcaneal Diabetic
Foot Ulcer

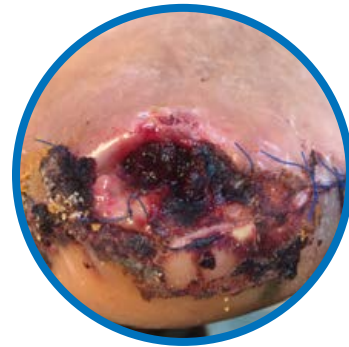

Metatarsal
Amputation

The Snap Therapy System seamlessly fits into patients' lives for high acceptance

- **Fast application:** up and running in under 10 minutes¹
- **No interference:** mechanically-powered without the need for batteries, recharge, or noise, it minimally interferes with daily activities such as sleep, social interactions, and mobility¹
- **Friendly fit:** discreet and comfortable placement (under clothing) anywhere on the body lets patients shower and sleep with it in place¹
 - Compact (fits in the palm of your hand)
 - Silent (no audible alarms)
 - Lightweight (weighs less than a cell phone)

The 3M™ Snap™ Therapy System provides real NPWT results

Clinical evidence from two multicenter Randomized Controlled Trials (RCTs) demonstrated similar healing outcomes as 3M™ V.A.C® Therapy.^{1,2a}



The average difference in time to healing was significantly ($p < 0.0001$) shorter (54.27 +/- 28.1 days), when individual Snap Therapy System patients were compared to patients treated with modern wound care protocols.³

The Snap Therapy System provides features and versatility the competition can't deliver.^{1,2a}

Feature	3M™ Snap™ Therapy System	PICO® 7 / PICO® 14 Systems
Negative Pressure Level	Multiple Options: -75, -100, or -125mmHg	-80mmHg ⁴
Exudate Containment	Visual Proof: Cartridge with transparent volume viewing window (60 or 150ml)	Dressing
Dressing Composition	Wound Management: Hydrocolloid drape and reticulated open-cell foam support macrostrain and periwound protection without the need for additional fixation or skin protection ²	4-layer dressing with high MVTR drape and silicone contact layer
Cut-to-Fit Dressings	✓	✗
Cut-to-Fit Tubing	✓	✗
Bridge Dressing	✓	✗
Wound Depth Restriction	None	PICO® 7: a filler is required if $\geq 2\text{cm}$ PICO® 14: a filler is required if $\geq 4.5\text{cm}$
Weight	2.2 _{oz.}	4.2 _{oz.}
Full Showering (with device connected)	✓	✗
Power Source	None: Mechanical	2 AA lithium batteries
Visual alarms	✓	✓
Off-the-shelf ready	✓	✓

^aDespite randomization in a multicenter RCT of 115 patients with lower extremity diabetic and venous wounds, the initial wound size was significantly greater in V.A.C.® Therapy patients (n=56) vs Snap Therapy patients (n=59) with a mean of 9.95 cm² vs 5.37 cm², respectively; $p=0.0093$, and no significant difference was seen in promoting wound closure; $p=0.9620$.^{1,2} ^aA foam or gauze filler is recommended if wound is over 0.5cm in depth.

The 3M™ Snap™ Therapy System is part of a real support system

The Snap Therapy System is a key component of the 3M continuum of specialized wound care from acute to out-of-hospital care settings

Ordering Information

SKU	Description	Size	Unit of Measure
SNPA125US	3M™ Snap™ -125mmHg Therapy Cartridge	60mL	Eaches
SNPA125US/10	3M™ Snap™ -125mmHg Therapy Cartridge	60mL	Case of 10
SNPA125PLUS	3M™ Snap™ Plus -125mmHg Therapy Cartridge	150mL	Eaches
SNPA125PLUS/10	3M™ Snap™ Plus -125mmHg Therapy Cartridge	150mL	Case of 10
BKTF14X11	3M™ Snap™ Bridge Dressing Kit	14cm x 11cm	Eaches
BKTF14X11/10	3M™ Snap™ Bridge Dressing Kit	14cm x 11cm	Case of 10
BKTF14X11S	3M™ Snap™ Bridge Dressing Kit with SecurRing™ Hydrocolloid Skin Barrier	14cm x 11cm	Eaches
BKTF14X11S/10	3M™ Snap™ Bridge Dressing Kit with SecurRing™ Hydrocolloid Skin Barrier	14cm x 11cm	Case of 10
SKTF10X10	3M™ Snap™ Advanced Dressing Kit	10cm x 10cm	Eaches
SKTF10X10/10	3M™ Snap™ Advanced Dressing Kit	10cm x 10cm	Case of 10
SKTF15X15	3M™ Snap™ Advanced Dressing Kit	15cm x 15cm	Eaches
SKTF15X15/10	3M™ Snap™ Advanced Dressing Kit	15cm x 15cm	Case of 10
STPAS	3M™ Snap™ Therapy Strap - Small	18"	Eaches
STPAM	3M™ Snap™ Therapy Strap - Medium	21"	Eaches
STPAL	3M™ Snap™ Therapy Strap - Large	24"	Eaches
STPASP	3M™ Snap™ Plus Therapy Strap - Small	18"	Eaches
STPASP	3M™ Snap™ Plus Therapy Strap - Medium	21"	Eaches
STPASP	3M™ Snap™ Plus Therapy Strap - Large	24"	Eaches
SRNG10	3M™ Snap™ SecurRing™ Hydrocolloid Skin Barrier	2" diameter	Case of 10

If you have product questions, please contact your local representative or call 1-800-275-4524 option #3 or visit 3M.com/medical.

References: **1.** Armstrong DEG, Marston WA, Reyzelman AM, Kirsner RS. Comparative effectiveness of mechanically and electrically powered negative pressure wound therapy devices: a multicenter randomized controlled trial. *Wound Rep Reg* 2012; 20(3):332-341. **2.** Marston WA, Armstrong DG, Reyzelman AM, Kirsner RS. A Multicenter Randomized Controlled Trial Comparing Treatment of Venous Leg Ulcers Using Mechanically Versus Electrically Powered Negative Pressure Wound Therapy. *Advances in Wound Care*. 2015;4(2):75-82. doi:10.1089/wound.2014.0575. **3.** Lerman B, Oldenbrook L, Eichstadt SL, Ryu J, Fong KD, Schubart PJ. Evaluation of chronic wound treatment with the SNAP™ wound care system versus modern dressing protocols. *Plastic and Reconstructive Surgery*. 2010 Oct;126(4):1253-61. **4.** PICO™ 7 Single Use Negative Pressure Wound Therapy System [user manual]. Hull, England. Smith & Nephew, Inc.; 2018.marks.

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals. Rx only.

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. All other marks are property of their respective owners. 70-2011-8073-7 PRA-PM-US-02812 (04/21)



3M Company
2510 Conway Ave
St. Paul, MN 55144 USA

Phone 1-800-275-4524 (NPWT products)
1-800-228-3957
Web 3M.com/medical