









3M™ V.A.C. Veraflo™ Dressing selection guide

This guide may be used in deciding which wound dressing to use in conjunction with 3M™ Veraflo™ Therapy.

| | 3M™ V.A.C. Veraflo™ Dressing - Small/Medium | 3M™ V.A.C. Veraflo™ Dressing - Large | 3M™ V.A.C. Veraflo Cleanse™ Dressing | 3M™ V.A.C. Veraflo Cleanse Choice™ Dressing |
|-------------------------------|---|---|---|---|
| Wound types |  |  |  |  |
| Wound characteristics | Open wounds, including wounds with shallow undermining or tunnel areas where the distal aspect is visible. | Large open wounds, including wounds with shallow undermining or tunnel areas where the distal aspect is visible. | Cavity wounds or wounds with complex geometries. | Wounds with thick fibrinous exudate, slough, infectious material and other wound bioburden. |
| Key goal(s) of therapy | <ul style="list-style-type: none"> • When used in conjunction with Veraflo Therapy, to help facilitate the removal of infectious material and other wound bioburden • Generation of robust granulation tissue | <ul style="list-style-type: none"> • When used in conjunction with Veraflo Therapy, to help facilitate the removal of infectious material and other wound bioburden • Generation of robust granulation tissue in large wounds | <ul style="list-style-type: none"> • When used in conjunction with Veraflo Therapy, to initiate therapy and to help facilitate the removal of infectious material and other wound bioburden • Easy application into tunneling and undermining | <ul style="list-style-type: none"> • When used in conjunction with Veraflo Therapy, to initiate therapy and to help facilitate the removal of infectious material such as thick fibrinous exudate, slough, and other wound bioburden • Provides a wound cleansing option for clinicians when surgical debridement must be delayed or is not possible or appropriate¹ |

3M™ V.A.C. Veraflo™ Dressing selection guide

| | 3M™ V.A.C. Veraflo™ Dressing - Sm/Md | 3M™ V.A.C. Veraflo™ Dressing - Large | 3M™ V.A.C. Veraflo Cleanse™ Dressing | 3M™ V.A.C. Veraflo Cleanse Choice™ Dressing |
|-----------------------------|--|---|--|---|
| SKU | Small: ULTVFL05SM Medium: ULTVFL05MD  | ULTVFL05LG  | ULTVCL05MD  | Medium: ULTVCC05MD Large: ULTVCC05LG  |
| Shape | Spiral-cut foam. | Block foam pre-slit into two layers. | Tubular shape. | Block foam pre-slit into three layers. |
| Application characteristics | Easy application: <ul style="list-style-type: none"> • Size without scissors • Precut area for pad application when used for bridging • Single pad application | Easy application: <ul style="list-style-type: none"> • Ideal for large surfaces areas with shallow depths • Provided with 3M™ V.A.C. VeraT.R.A.C. Duo™ Tube Set for extended surface area coverage | Application flexibility: <ul style="list-style-type: none"> • Ideal for addressing wounds with complex geometries (eg, tunnels, undermining and circumferential applications) • Single pad application | Easy application with flexibility: <ul style="list-style-type: none"> • Thin layers for improved conformability • Multiple layers provide application options for wounds with varying depths • Ideal for dirty wounds needing active therapy • Single or duo pad application |
| Dressing characteristics | Small: 11.2 × 7.7 × 1.8cm per piece (2 pieces spiral) <ul style="list-style-type: none"> • Each piece 24cm uncoiled • Each piece 18cm long bridge - 2 sheets 3M™ V.A.C.® Advanced Drape - 1 3M™ V.A.C. VeraT.R.A.C.™ Pad - 1pk 3M™ Cavilon™ No Sting Barrier Film - 1 disposable ruler/foam quantity label Medium: 17.4 × 14.7 × 1.8cm per piece (2 pieces spiral) <ul style="list-style-type: none"> • Each piece 80cm uncoiled • Each piece 74cm long bridge - 3 sheets 3M™ V.A.C.® Advanced Drape - 1 3M™ V.A.C. VeraT.R.A.C.™ Pad - 2pk 3M™ Cavilon™ No Sting Barrier Film - 1 disposable ruler/foam quantity label Fill assist set point range Small: 12-80mL (1 piece) 26-160mL (2 pieces) Medium: 38-240mL (1 piece) 80-450mL (2 pieces) | 25.6 × 15.0 × 1.6 cm per piece (oval - 2 pieces) - 5 sheets 3M™ V.A.C.® Advanced Drape - 1 3M™ V.A.C. VeraT.R.A.C. Duo™ Pad - 4pk 3M™ Cavilon™ No Sting Barrier Film - 1 disposable ruler/foam quantity label Fill assist set point range 55-350mL (1 piece) 110-500mL (2 pieces) | 1 each coiled rod (61 × 3.2cm diameter) - 3 sheets 3M™ V.A.C.® Advanced Drape - 1 3M™ V.A.C. VeraT.R.A.C.™ Pad - 2pk 3M™ Cavilon™ No Sting Barrier Film - 1 disposable ruler/foam quantity label Fill assist set point range 65-130mL (1/2 rod) 130-250mL (full rod) | Medium: 18.0cm x 12.5cm x Varying Thicknesses (0.8cm – 1.6cm) - 4 sheets 3M™ V.A.C.® Advanced Drape - 1 3M™ V.A.C. VeraT.R.A.C.™ Pad - 4pk 3M™ Cavilon™ No Sting Barrier Film - 1 disposable ruler/foam quantity label Large: 25.6cm x 15.0cm x Varying Thicknesses (0.8cm – 1.6cm) - 6 sheets 3M™ V.A.C.® Advanced Drape - 1 3M™ V.A.C. VeraT.R.A.C. Duo™ Pad - 5pk 3M™ Cavilon™ No Sting Barrier Film - 1 disposable ruler/foam quantity label Fill volume start points Medium: 85mL (1.6cm Cover Layer); 42mL (0.8cm Cover Layer); 24mL (0.8cm Wound Contact Layer) Large: 150 mL (1.6cm Cover Layer); 75mL (0.8cm Cover Layer); 42mL (0.8cm Wound Contact Layer) |

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. Used under license in Canada.
 70-2011-8204-8 PRA-PM-EU-00507 (11/21)

