



Filtek™

**Bulk Fill
Flowable Restorative**

The bulk fill flowable that's easier to inject.

3M™ Filtek™ Bulk Fill Flowable Restorative is available in a new ergonomic syringe designed for improved comfort and easier injecting. This gives you more control and visibility for precise placement in deep restorations.

Reduce bubbles and waste

Virtually no bubbles or material "run-on" during dispensing gives you better control.

No more guessing

The green syringe barrel indicates the remaining material volume. When your syringe is empty, only the white plunger shows.

Easy to hold and inject

Triangle finger plate and plunger make it quick and easy to personalize your grip.



Bendable cannula

Easy-to-bend without kinking for better access.



Key benefits

- Virtually no bubbles or material “run-on”
- Easy-to-bend cannula for improved access
- Ergonomic design is easy to hold and inject
- Injectable, one-step placement up to 4 mm
- Excellent adaptation and low shrinkage stress

Can be warmed for injection molding

Warm up to 70°C (158°F) for up to one hour, up to 25 times.



Posterior restoration made simple



Prepare the tooth and apply an adhesive e.g. 3M™ Single Bond Universal Adhesive.



Dispense up to 4 mm of 3M™ Filtek™ Bulk Fill Flowable Restorative into the prepared cavity and cure, e.g. with 3M™ Elipar™ DeepCure-L LED Curing Light.




Place a 2 mm (minimum) capping layer of an aesthetic composite, such as a 3M™ Filtek™ Z350 XT Universal Restorative.

Clinical photos courtesy of Dr. Reich, Seefeld, Germany

Indications

(Incremental up to 4 mm)

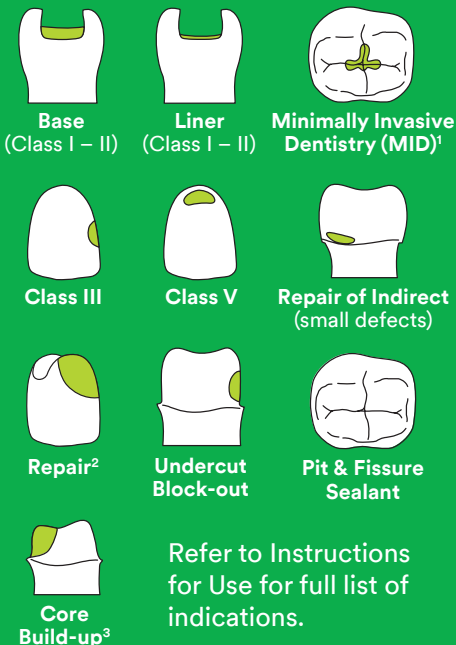

Ordering information



3M™ Filtek™ Bulk Fill Flowable Restorative Refills

Syringe refills include: 2 – 2g syringes; 20 dispensing tips.

SHADE	ITEM NO.
A2	4862A2
A3	4862A3

ITEM NO.	PRODUCT INFO.
3740T	3M™ Filtek™ Bulk Fill Flowable Restorative Dispensing Tips, 19-gauge, Green (20)



3M Oral Care Indonesia
Perkantoran Hijau Arkadia, Menara F Lt. 8
Jalan TB Simatupang Kav. 88, Jakarta Selatan - 12520

Office +621 2997 4000
Phone +62 813 1554 0844
Web 3M.co.id/3M/en_ID/dental-id

1. Including small, non stress-bearing occlusal restorations.
2. Repair of acrylic temporary and composite resin.
3. Where at least half the coronal tooth structure is remaining to provide structural support for the crown

3M and Filtek are trademarks of 3M Company. © 2019. All rights reserved.