



# Filtek™

Supreme  
Flowable Restorative

## Now our most popular flowable is easier to inject.


3M™ Filtek™ Supreme Ultra Flowable Restorative is available in a new ergonomic syringe. Designed for improved comfort and easier injection, this new product is called 3M™ Filtek™ Supreme Flowable Restorative.

### Reduce bubbles and waste

Virtually no bubbles or material “run-on” during dispensing gives you better control.

### No more guessing

The blue syringe barrel indicates the remaining material volume. When your syringe is empty, only the white plunger shows.

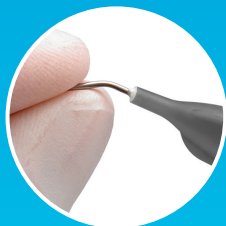


### Bendable cannula

Easy-to-bend without kinking for better access.

### Easy to hold and inject

Triangle finger plate and plunger make it quick and easy to personalize your grip.

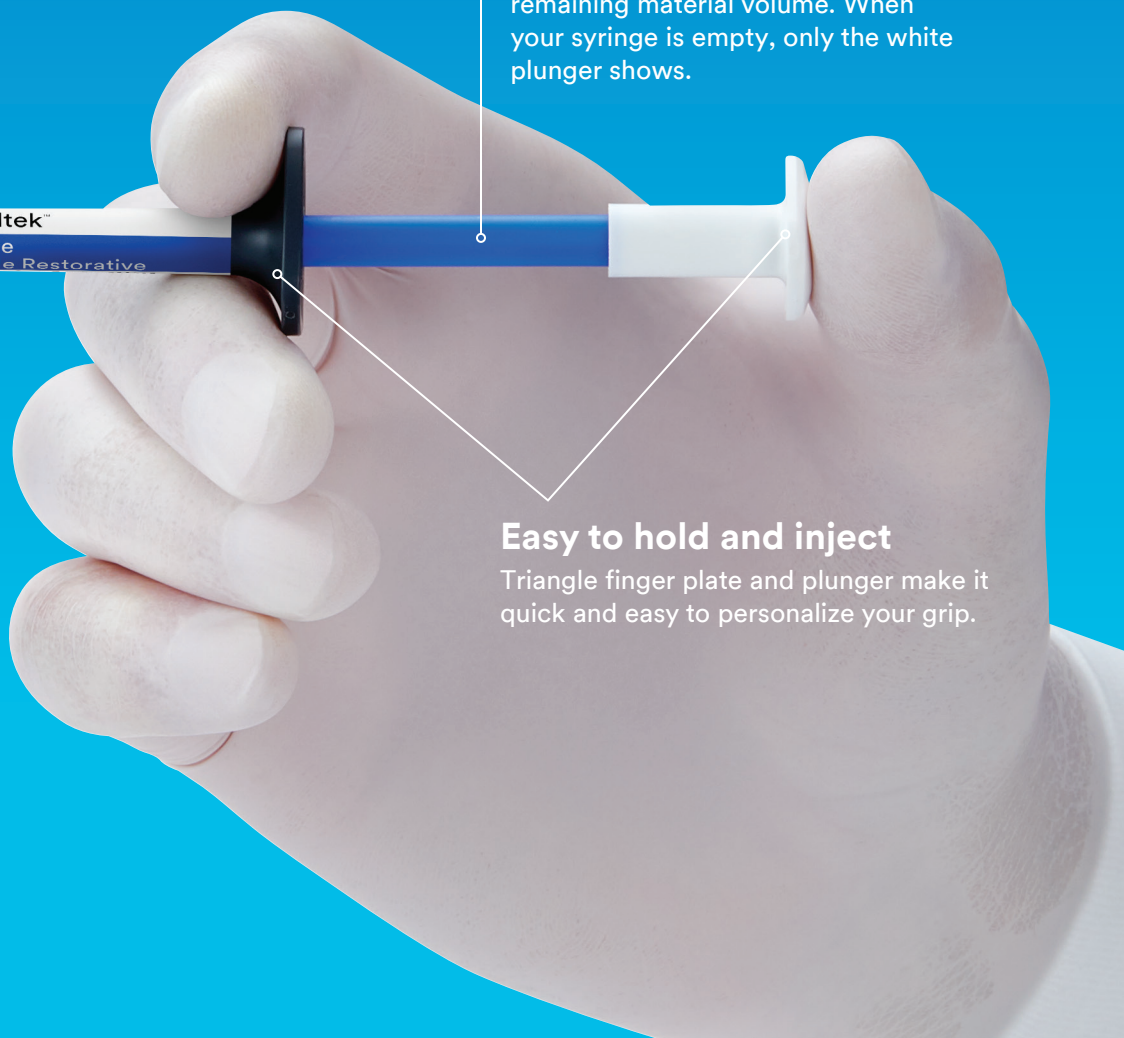

## Key benefits

- Virtually no bubbles or material “run-on”
- Easy-to-bend cannula for improved access
- Ergonomic design is easy to hold and inject
- Excellent adaptation, polish retention and wear resistance
- Versatile for a wide range of indications and techniques

## Can be warmed for injection molding

Warm up to 70°C (158°F) for up to one hour, up to 25 times.



## Clinical cases

3M™ Filtek™ Supreme Flowable Restorative offers a smooth transition from material to tooth for esthetic restorations.



Before



Flowable placement



After

Photos courtesy of Dr. Gunnar Reich, Munich, Germany

3M™ Filtek™ Supreme Flowable Restorative adapts easily and seals the margin.



Before



Flowable placement



After

Photos courtesy of Dr. Giuseppe Chiodera, Brescia, Italy



3M Oral Care Indonesia  
Perkantoran Hijau Arkadia, Menara F Lt. 8  
Jalan TB Simatupang Kav. 88, Jakarta Selatan - 12520

Office +621 2997 4000  
Phone +62 813 1554 0844  
Web 3M.co.id/3M/en\_ID/dental-id

## Indications

(Incremental up to 2 mm)



Class III



Class V



Minimally Invasive Dentistry (MID)<sup>1</sup>



Base



Liner



Repair of Indirect (small defects)



Repair<sup>2</sup>



Undercut Block-out



Pit & Fissure Sealant

Refer to Instructions for Use for full list of indications.

## Ordering information



### 3M™ Filtek™ Supreme Flowable Restorative Refills

Syringe refills include: 2 – 2g syringes; 20 dispensing tips.

SHADE	ITEM NO.
A2	7032A2
A3	7032A3
A4	7032A4
OA3	6032OA3

ITEM NO.	PRODUCT INFO.
3700T	3M™ Filtek™ Supreme Flowable Restorative Dispensing Tips, 20-gauge, Gray (20)



1. Including small, non stress-bearing occlusal restorations.  
2. Repair of acrylic temporary and composite resin.

3M and Filtek are trademarks of 3M Company. © 2019. All rights reserved.