



CuROS™

Disinfecting Port
Protectors

Cap it with the right colour.

Use this CuROS Cap:

Only on:


Do not use on:



**3M™ CuROS™ Disinfecting Cap
for needleless connectors**



**Needleless
connectors**

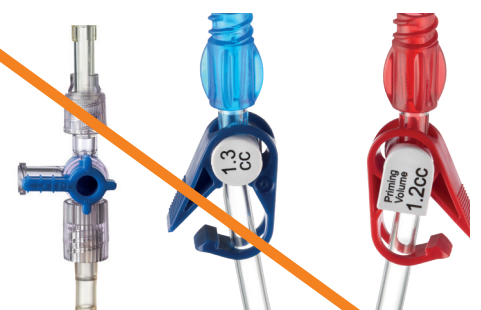

**Open female luers
(stopcocks or catheter hubs)**



**3M™ CuROS™ Stopper Disinfecting Cap
for open female luers**



**Open female luers
(stopcocks or catheter hubs)**




**Needleless
connectors**



**3M™ CuROS™ Disinfecting Cap
for Tego® Hemodialysis Connectors**



Tego® Hemodialysis Connectors

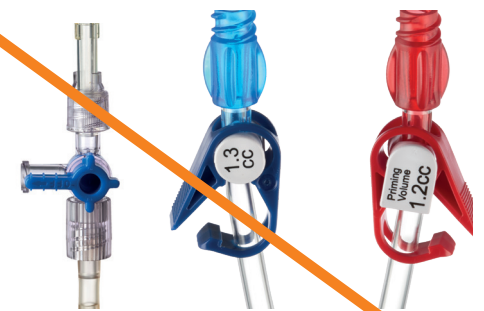

**Open female luers
(stopcocks or catheter hubs)**



**3M™ CuROS Tips™ Disinfecting Cap
for male luers**



Male luers