

3M

Curoso™

Disinfecting Port
Protectors

How to use

3M™ Curoso™


Disinfecting Cap for Needleless Connectors

Disinfects in one minute – protects needleless connectors for up to seven days.



Peel

the foil from the 3M™ Curoso™ Disinfecting Cap.

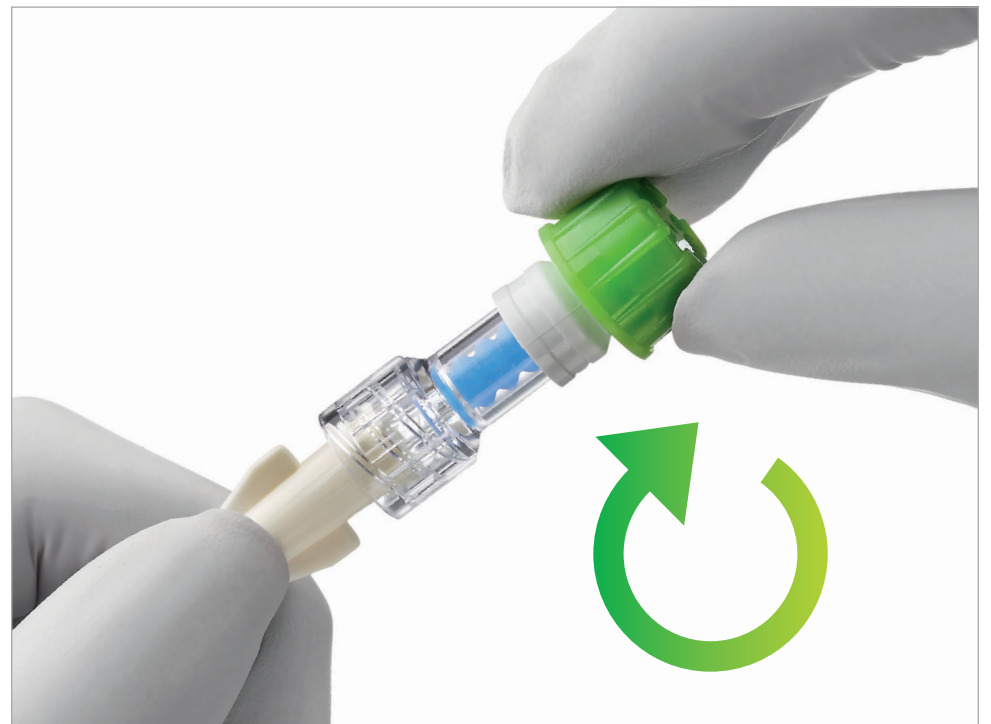

Push and twist

the Curoso disinfecting cap clockwise onto the end of the needleless connector.

- ▶ If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab before using cap
- ▶ Do not use cap if seal is broken, torn, punctured, or sponge appears dry
- ▶ Cap must remain on needleless connector for a minimum of one minute and may remain on for seven days if not removed
- ▶ The needleless connector can be accessed immediately after removing the Curoso disinfecting cap that has been in place for a minimum of one minute
- ▶ Always place a new Curoso disinfecting cap on needleless connector after each use
- ▶ Dispose of the Curoso disinfecting cap after every use
- ▶ Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses

Warning: to avoid potential for injury, use only on needleless connectors.

Caution: potential choking hazard.

3M United Kingdom PLC
Charnwood Campus
10 Bakewell Road
Loughborough
LE11 5RB
01509 611611

3M Ireland Limited
The Iveagh Building
Carrickmines Park
Carrickmines
Dublin 18

[3M.co.uk/Medical](https://www.3m.co.uk/Medical)

Active ingredient: 70% isopropyl alcohol (IPA). Single use only.
This device is not made with natural rubber latex or plasticiser
Diethylhexylphthalate (DEHP). Refer to current device labeling
for indications for use, warnings and product details.

GF531

3M and Curoso are trademarks of 3M Company.
© 3M 2020. All rights reserved.
OMG79664.