

3M

Curoso™

Disinfecting Port
Protectors

How to use

3M™ Curoso™ for Tego®

Disinfecting Cap for Tego® Hemodialysis Connectors

Disinfects in one minute – protects for up to seven days.



Peel

the foil from the 3M™ Curoso™ Disinfecting Cap for Tego® Hemodialysis Connector.



Push and twist

the Curoso disinfecting cap for Tego® hemodialysis connector clockwise onto the end of the hemodialysis connector.

- ▶ If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab before using cap
- ▶ Do not use cap if seal is broken, torn, punctured, or sponge appears dry
- ▶ Cap must remain on needleless connector for a minimum of one minute and may remain on for seven days if not removed
- ▶ The needleless connector can be accessed immediately after removing the Curoso disinfecting cap for Tego® hemodialysis connector that has been in place for a minimum of one minute
- ▶ Always place a new Curoso disinfecting cap for Tego® hemodialysis connector on hemodialysis connector after each use
- ▶ Dispose of the Curoso disinfecting cap for Tego® hemodialysis connector after every use
- ▶ Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses

Warning: to avoid potential for injury, use only on Tego® Hemodialysis Connectors.

Caution: potential choking hazard.

3M United Kingdom PLC
Charnwood Campus
10 Bakewell Road
Loughborough
LE11 5RB
01509 611611

3M Ireland Limited
The Iveagh Building
Carrickmines Park
Carrickmines
Dublin 18

3M.co.uk/Medical

Active ingredient: 70% isopropyl alcohol (IPA). Single use only.
Rx Only. This device is not made with natural rubber latex or
plasticiser Diethylhexylphthalate (DEHP). Refer to current device
labeling for indications for use, warnings and product details.

GF532

Tego® is a trademark of ICU Medical, Inc.
3M and Curoso are trademarks of 3M Company.
© 3M 2020. All rights reserved.
OMG79664.