



# CuroS™

Disinfecting Port Protectors

## HOW TO USE

# CuroS™ for Tego®

Disinfecting Cap for Tego® Hemodialysis Connectors

**Disinfects in 1 minute — protects ports for up to 7 days.**



**Peel**  
the foil from the 3M™ CuroS™ Disinfecting Cap for Tego® Hemodialysis Connector.



**Push & Twist**  
the CuroS disinfecting cap for Tego® hemodialysis connector clockwise onto the end of the hemodialysis connector.

- If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab prior to using cap.
- Do not use cap if seal is broken, torn, punctured, or sponge appears dry.
- Cap must remain on needleless connector for minimum of one (1) minute and may remain on for seven (7) days if not removed.
- The needleless connector can be accessed immediately after removing the CuroS disinfecting cap for Tego® hemodialysis connector that has been in place for a minimum of one (1) minute.
- Always place a new CuroS disinfecting cap for Tego® hemodialysis connector on hemodialysis connector after each use.
- Dispose of the CuroS disinfecting cap for Tego® hemodialysis connector after every use.
- Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses.

**WARNING: To avoid potential for injury, use only on Tego® Hemodialysis Connectors.**

**CAUTION: Potential choking hazard.**



3M Health Care  
Medical Solutions Division  
3M Gulf Ltd.  
Dubai Internet City  
P.O. Box 20191, Dubai, U.A.E.  
Tel: +971 4 3670 777  
Fax: +971 4 3670 998  
www.3Mae.ae/Medical

Please recycle this poster. Printed in U.S.A.  
© 3M 2021. All rights reserved.  
3M and CuroS are trademarks of 3M.  
Tego is a registered trademark of ICU Medical.  
70-2011-5799-0

STERILE R

Active Ingredient: 70% isopropyl alcohol (IPA).  
Single Use Only. Rx Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.