

V.A.C. VERAFLO CLEANSE CHOICE™ Dressing System Application Summary

This Application Summary should be used only in conjunction with the safety and application information provided in the instructions for use provided in the V.A.C. VERAFLO CLEANSE CHOICE™ Dressing cartons.

1. Wound Preparation and Application of 3M™ Cavilon™ No Sting Barrier Film

Ensure debridement of all necrotic, non-viable tissue, including bone, eschar, or hardened slough, as prescribed by treating physician.

Perform thorough wound and periwound area cleaning per physician order or institution protocol prior to each dressing application.

Skin should be clean & dry prior to application of 3M™ Cavilon™ No Sting Barrier Film

Wipe uniform coating of film over the entire area of concern
Allow to thoroughly dry before applying drape



2. Placement of Wound Contact Layer

Protect sensitive structures, vessels, and organs as needed

Assess wound dimensions and pathology, including the presence of undermining or tunnels

Cut the Thin or Thick Cover layer

of foam to fill explored tunnels and undermining. Be sure foam in tunnels makes contact with cover layer

Cut Wound Contact Layer as needed to cover the entire wound bed and gently place in wound



3. Filling the Wound With The Cover Layer

Cut Thick or Thin Cover Layer as needed to fill the wound depth and gently place in the wound on top of the wound Contact Layer



4. V.A.C.® Advanced Drape Application

Trim the drape to cover the foam and at least an additional 3-5 cm border of intact periwound tissue

Remove Layer #1 to expose the adhesive and apply to cover layer and intact skin while ensuring skin wrinkles and folds are soothed
Remove Layer #2 and pat the drape to ensure an occlusive seal
Remove the Ruler/Handling Bars



5. V.A.C. VERAT.R.A.C.™ Pad Placement

Choose pad application site. Give particular consideration to tubing positioning to allow for optimal flow, and avoid placement over bony prominences or within creases in the tissue

Cut an approximately 2.5cm hole through the drape to affix V.A.C. VERAT.R.A.C.™ Pad

Remove both backing layers 1 and 2 to expose adhesive

Position and apply V.A.C. VERAT.R.A.C.™ Pad over the 2.5cm hole
Pull back on blue tab to remove pad stabilization layer



NOTE:

Dressings should be changed every 48-72-hours, but no less than three times per week, with frequency adjusted by the clinician as appropriate


Do not cut dressing over wound bed

Do not pack foam into wound

Do not use the wound contact layer with holes in tunnels or undermined areas

V.A.C. VERAFLU CLEANSE CHOICE™ Dressing: A Simple Solution For Complex Wounds

Thin Cover Layer
Thickness (0.8cm)

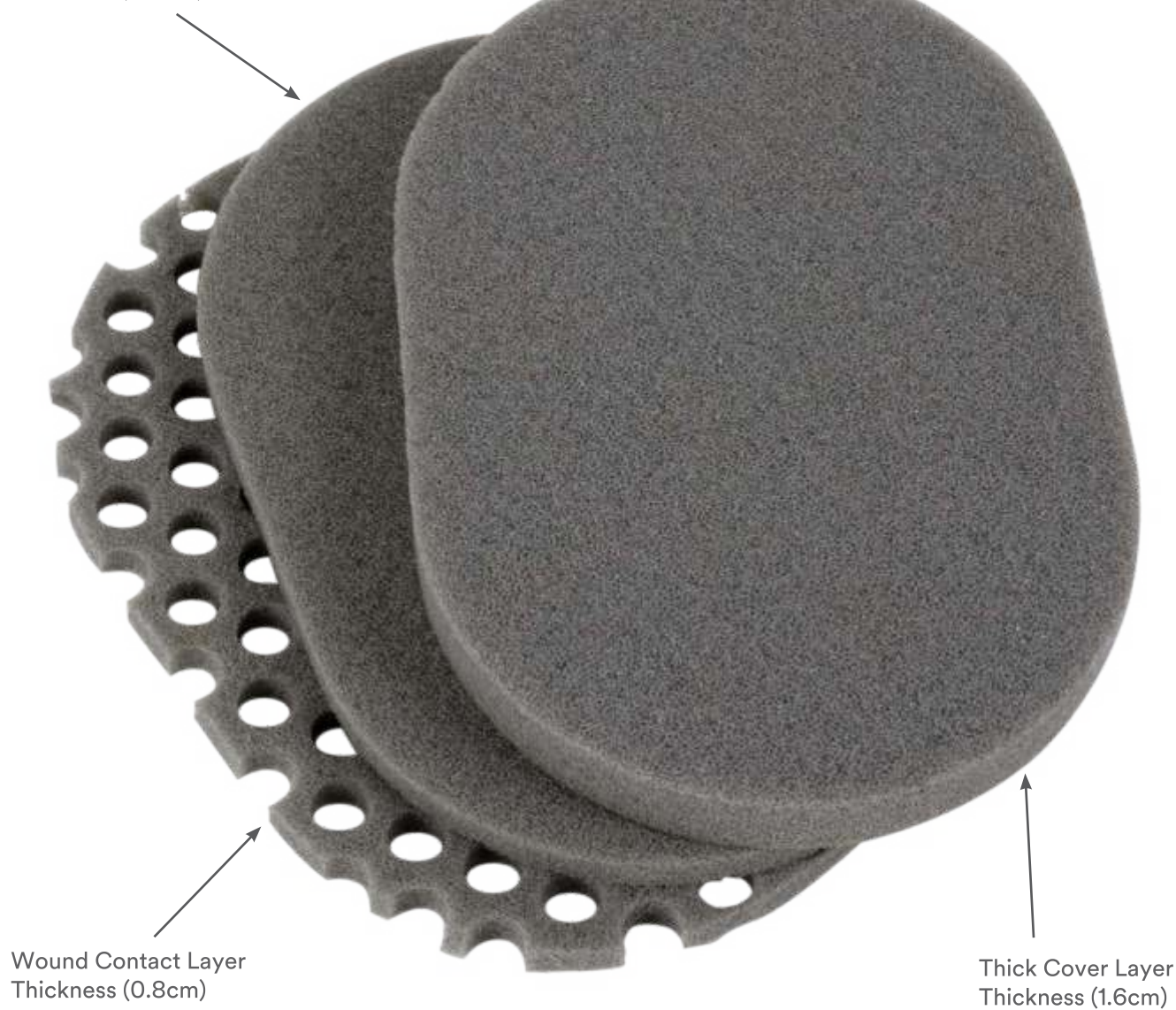

Wound Contact Layer
Thickness (0.8cm)

Thick Cover Layer
Thickness (1.6cm)

For more information, call 1800 103 8118 or visit [www. Kci-medical.in](http://www.Kci-medical.in)

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.



© 2020 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited.
PRA-PM-IN-00117(11/20)