



# Tegaderm™



IV Care Solutions

## 3M™ Tegaderm™ Antimicrobial I.V. Advanced Securement Dressing 9132

Integrated design combines antimicrobial protection with site visibility, catheter securement and consistent application for peripheral IVs.

### Antimicrobial protection

Suppresses the regrowth of normal skin flora on prepped skin better than non-antimicrobial dressings.<sup>1</sup>

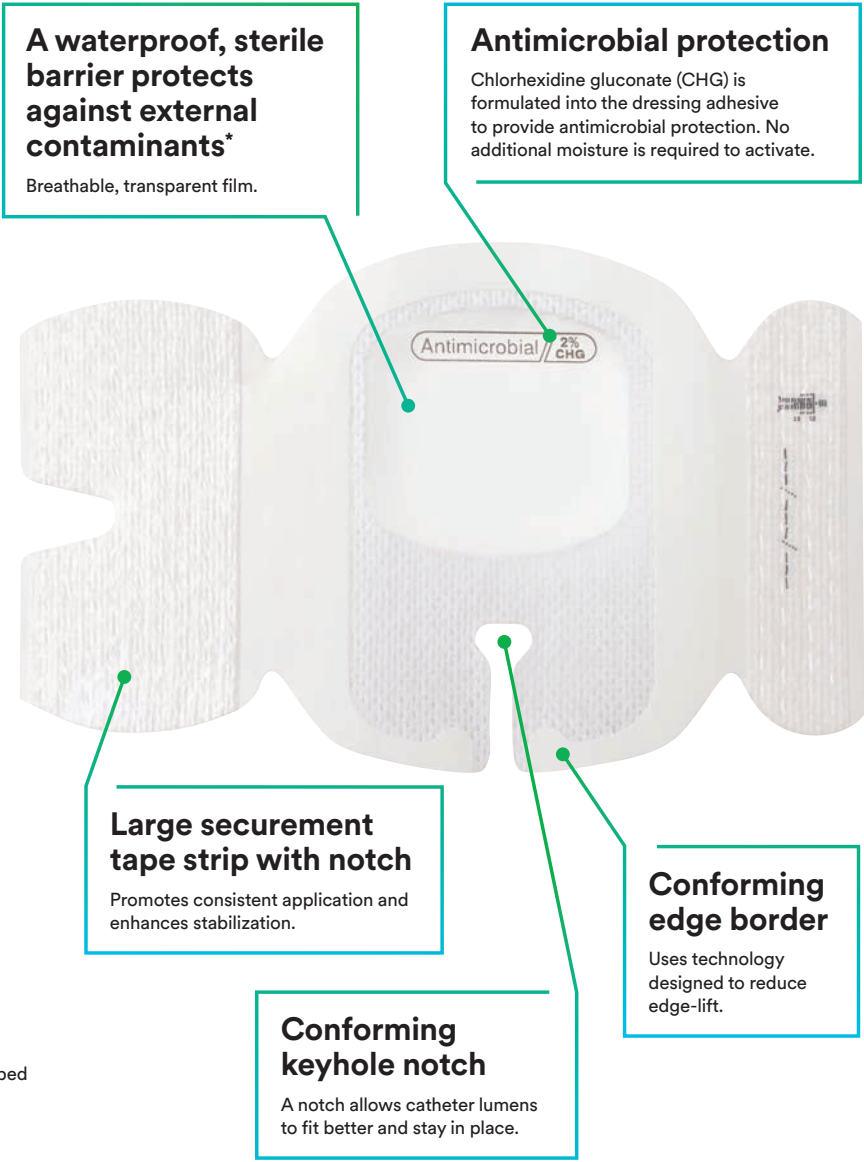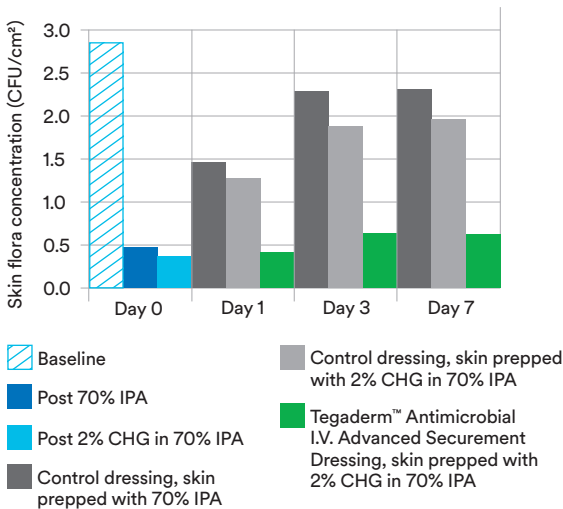

### Antimicrobial protection

Proven to sustain the suppression of regrowth of normal skin flora on prepped skin for up to 7 days.

### Site visibility

Transparent dressing enables early identification of complications at the insertion site.


### Catheter securement

Designed to minimize catheter movement and dislodgement.

### Ease of use

The dressing with CHG adhesive is designed to ensure standardized, correct application.

\*In vitro testing shows that the film provides a barrier against viruses 27 nm in diameter or larger while the dressing remains intact without leakage. 1. 3M data on file. EM-05-305455

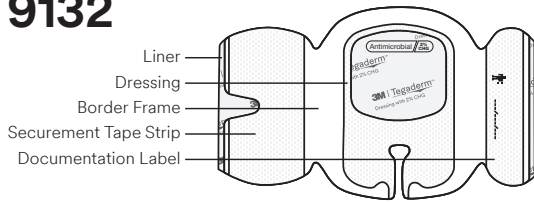

**Prepare**




Prepare site according to facility protocol. Ensure site is completely dry.

**Press**



Do not stretch the dressing at placement. Peel the liner from the dressing and place the dressing window over insertion site, aligning extension tubing with securement notch. Apply firm pressure to entire dressing with one hand for optimal adhesion, while removing border frame with opposite hand.

**Secure**

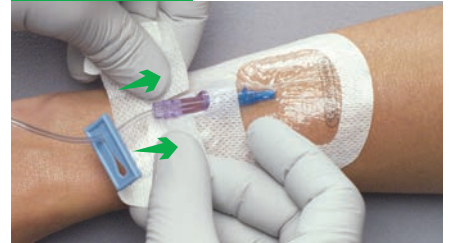

Apply securement tape strip under the blood collection hub and overlap back onto the dressing. Remove adhesive-free tabs.

**Secure**



Document the dressing change information on the label strip. Apply label strip on top of dressing, over catheter lumen(s). Remove adhesive-free tabs. Secure tubing with tape.

**Monitor**



Change the dressing:

- ▶ At least every 7 days
- ▶ If the dressing becomes loose, soiled or damp
- ▶ According to facility protocol

**Removal**



Remove tape strips. Using a low and slow removal technique, start removing the dressing from where the catheter or tubing exits the dressing toward the catheter insertion site. Avoid skin trauma by peeling the dressing back rather than pulling it up from the skin.

**Ordering information**

Product	Product number	Overall dressing size	Dressings/Box	Boxes/Case
	9132	7cm x 8.5cm	25	4

Refer to product Instructions for Use for other important information.

Visit [3M.com/IVtraining](http://3M.com/IVtraining) to find product training resources for all 3M IV care solutions.