

Quick Start Guide.

3M™ Scotchbond™ Universal Plus Adhesive

- Self-etch, selective-etch and total-etch adhesive
- MDP primer for zirconia, metal, composite
- Silane primer for glass ceramics, fiber reinforced posts

3M™ RelyX™ Universal Resin Cement

- Dual-cure two paste resin cement
- Adhesive and self-adhesive indications
- Four shades (TR, A1, A3O, WO)



Before using the products described, please refer to the instructions for use.

1. Select restoration material and pre-treat as described


Fiber-reinforced Composite Post*



Zirconia**, Alumina**, Metal**



Composite/Hybrids

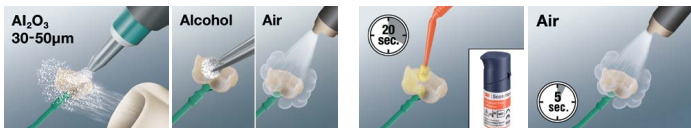

Glass-Ceramic***



2. Select tooth pre-treatment by indication

Option 1: Self-adhesive. Recommended for:

- Post
- Crown
- Bridge



Adhesive not required on tooth

Option 2: (Selective-etch) adhesive. Recommended for:

- Inlay
- Onlay






Optional for cut enamel

Option 3: Total-etch adhesive. To be used for:

- Table Top
- Veneer
- Adhesive Bridge



3. Seat restoration following steps A-D

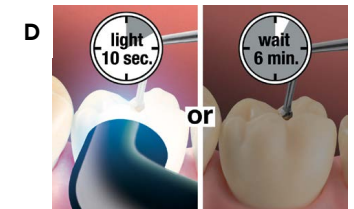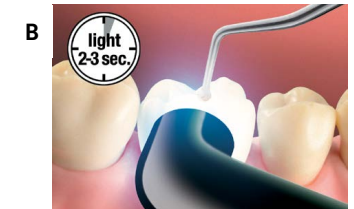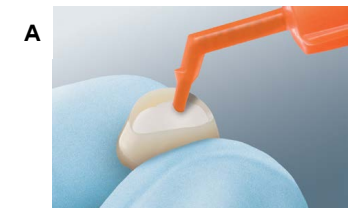

*Priming not required for 3M™ RelyX™ Fiber Posts. **Priming optional, recommended for veneer, table top, adhesive bridges. ***Application of an additional silane (e.g. 3M™ RelyX™ Ceramic Primer) is not required, but is safe to use if preferred.

Cement and Adhesive Compatibility.



	3M™ RelyX™ Universal Resin Cement (in self-adhesive mode)	3M™ RelyX™ Universal Resin Cement (in adhesive mode)	3M™ RelyX™ Ultimate Adhesive Resin Cement	Other dual cure adhesive resin cements
3M™ Scotchbond™ Universal Plus Adhesive	No adhesive required	✓✓ Light cure adhesive and cement together, 10s per surface	✓ MUST light cure adhesive first	✓ MUST light cure adhesive first
3M™ Scotchbond™ Universal Adhesive	No adhesive required	✗ Not indicated	✓ Light cure adhesive and cement together, 10s per surface	✓ MUST use DCA* MUST light cure adhesive first
Other adhesives	No adhesive required	✗ Not indicated	✗ Not indicated	—

✓✓ Recommended ✓ Indicated ✗ Not indicated

*DCA = 3M™ Scotchbond™ Dual Cure Activator

3M™ RelyX™ Universal and 3M™ Scotchbond™ Universal Plus are a fully aligned system: the adhesive is initiated and cured by RelyX Universal Resin Cement, saving the light-cure step.