





### 3M™ Attest™ Vaporized Hydrogen Peroxide Tri-Metric Chemical Indicator, Type 4


#### Unprocessed

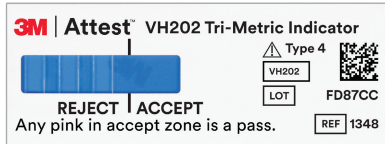

#### Processed

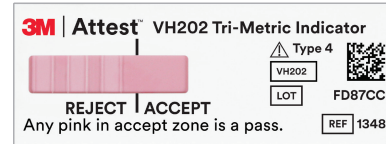

**ACCEPT**

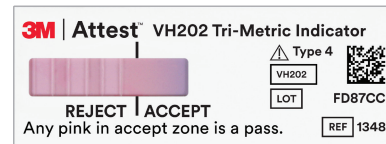

**ACCEPT**


A color change toward pink has reached the **ACCEPT** zone in the two samples shown above, verifying that the stated values for the three critical parameters of exposure time, temperature and concentration of vaporized hydrogen peroxide have been achieved.

**If a color change toward pink has entered the ACCEPT zone it is considered a pass.**

#### Processed



**REJECT**

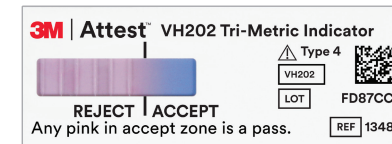

**REJECT**

A color change toward pink has not entered the **ACCEPT** zone in the two samples shown above, a **REJECT** result is indicated, the items in the pack, container system or tray were not exposed to sufficient vaporized hydrogen peroxide sterilization conditions.

The pack should be reprocessed and the cause of the failure should be investigated.

**If a color change toward pink has not entered the ACCEPT zone, it is considered a fail.**

3M™ Attest™ Vaporized Hydrogen Peroxide Tri-Metric Chemical Indicators, Type 4 comply with ISO 11140-1:2014.

© 2020 3M. All rights reserved. 3M and Attest are marks and/or registered marks of 3M. Unauthorized use prohibited. Used under license in Canada.