



**Veraflo™**  
Therapy




# Start smart with 3M™ Veraflo™ Therapy

Now backed by meta-analysis clinical  
and health economic data

# Key reasons to start with 3M™ Veraflo™ Therapy: The sooner, the better<sup>1</sup>

A meta-analysis of 13 comparative studies comprising 720 patients with a wide variety of wound types showed a positive effect when using NPWTi-d versus the control.<sup>1</sup>

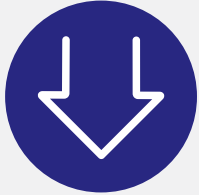

**>30% fewer** surgical debridements<sup>1,2</sup>

(1.77 debridements vs 2.69,  $p=0.008$ )



Wounds were **2.39 times more likely to close<sup>1</sup>**

( $p=0.01$ )



**>50% reduced** length of therapy<sup>1,2</sup>

(9.88 days vs 21.8 days,  $p=0.02$ )



**Reduced** bacterial count from baseline<sup>1</sup>

(Odds were 4.4 times greater,  $p=0.003$ )



Ready for closure almost **twice as fast<sup>1,2</sup>**  
(7.88 days vs 14.36 days,  $p=0.003$ )

## Global panel recommendations

Based on the 2020 NPWT with instillation consensus document by Paul Kim, DPM, MS et al. in the publication *International Wound Journal*, Veraflo Therapy should be considered early in treatment based on data and experience.<sup>3</sup>

## Recommended therapy settings

### Therapy settings:


- NPWT Time: 2-3 hours
- Dwell Time: 10 minutes
- NPWT Setting: -125mmHg
- Solution: Saline (or other compatible solutions)
- Dressing: 3M™ V.A.C. Veraflo™ Dressing unless non-viable tissue is present. Consider 3M™ V.A.C. Veraflo Cleanse Choice™ Dressing as an advanced wound cleansing option before surgical debridement or if debridement is delayed or not possible

Discontinue therapy when clinical goals are met.


# For potentially significant savings



Based upon the meta-analysis by Allen Gabriel, MD et al. an economic model was developed to compare the cost of using Veraflo Therapy to traditional wound care options, including 3M™ V.A.C.® Therapy in three countries.<sup>2</sup>

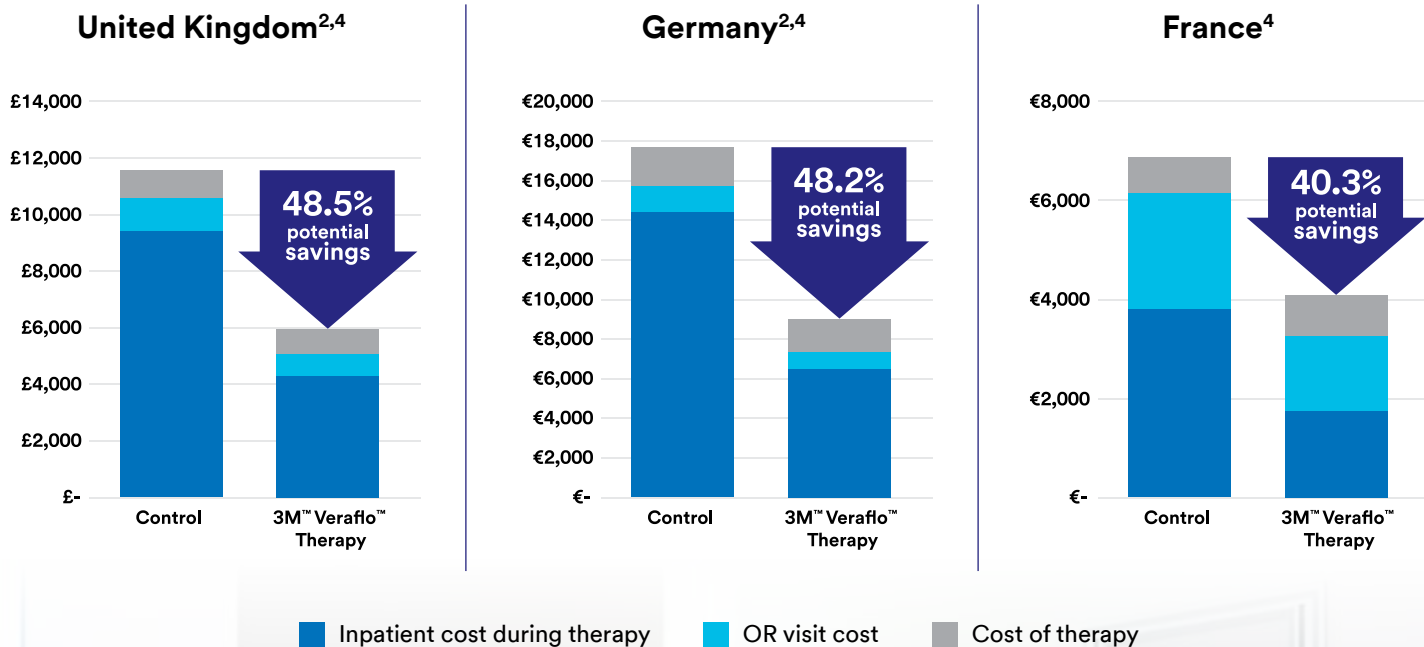

Improved outcomes can result in health economic benefits

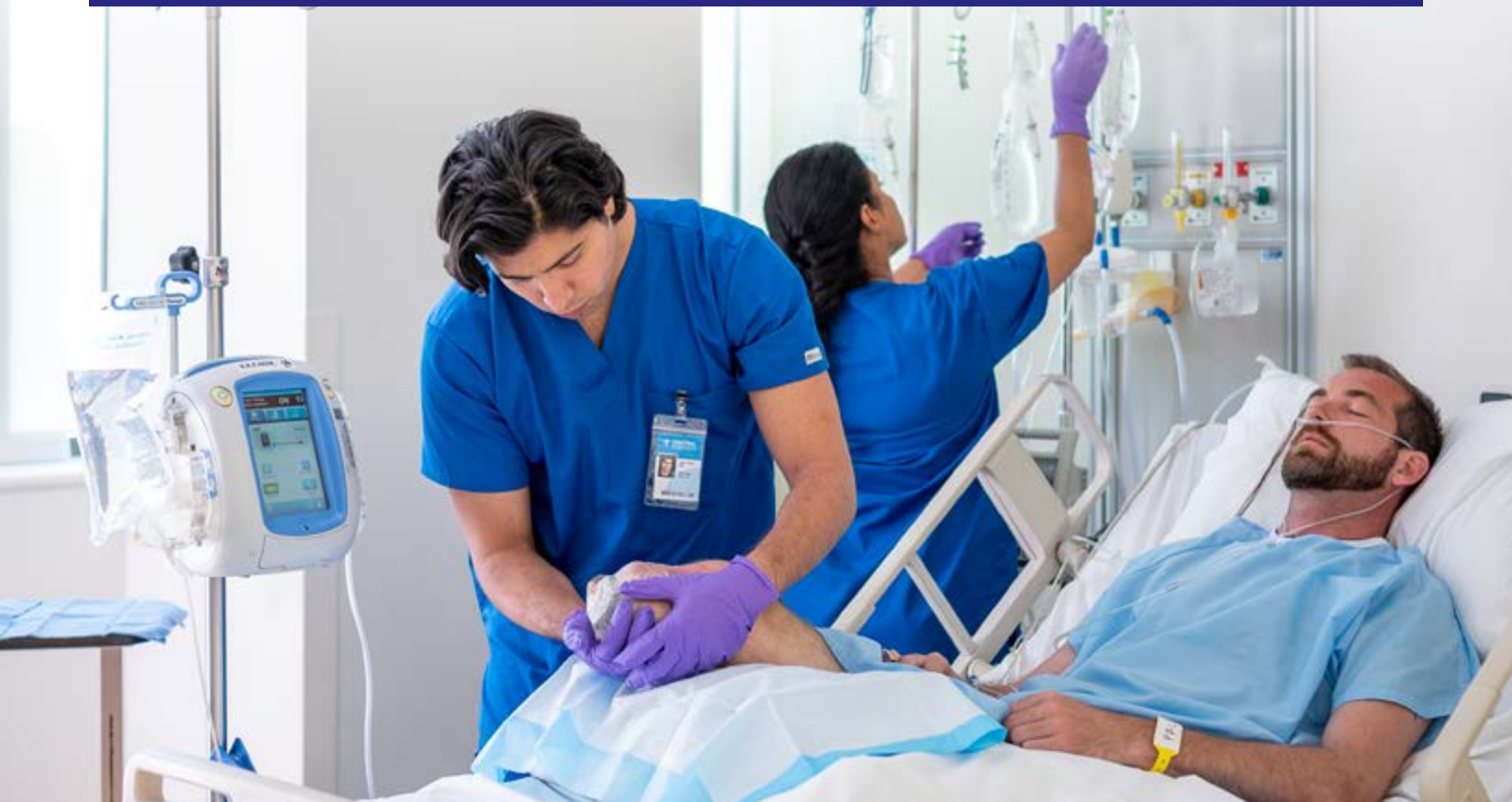

Backed by a meta-analysis of 720 patients<sup>1</sup> and demonstrated potential health economic savings<sup>2,4</sup>, see for yourself what Veraflo Therapy can do for your patients and practice. **Start smart.**



## Ordering information for Veraflo Therapy

3M™ V.A.C.® Ulta Therapy Unit	Country Specific SKU
3M™ V.A.C. Veraflo™ Dressing, small, 5-pack	ULTVFL05SM*
3M™ V.A.C. Veraflo™ Dressing, medium, 5-pack	ULTVFL05MD*
3M™ V.A.C. Veraflo™ Dressing, large, 5-Pack	ULTVFL05LG*
3M™ V.A.C. Veraflo Cleanse™ Dressing, medium, 5-pack	ULTVCL05MD*
3M™ V.A.C. Veraflo Cleanse Choice™ Dressing, medium, 5-pack	ULTVCC05MD*
3M™ V.A.C. Veraflo Cleanse Choice™ Dressing, large, 5-Pack	ULTVCC05LG*
3M™ V.A.C. Veralink™ Cassette, 5-pack	ULTLNK0500
3M™ V.A.C. VeraT.R.A.C. Duo™ Tube Set, 5-pack	ULTDUO0500
500mL Canister with gel for use with 3M™ V.A.C.® Ulta Therapy System, case of 5	M8275063/5
500mL Canister with gel for use with 3M™ V.A.C.® Ulta Therapy System, case of 10	M8275063/10
1000mL Canister with gel for use with 3M™ V.A.C.® Ulta Therapy System, case of 5	M8275093/5

\*For Nordic countries, add /NSP to end of the part number.

### References:

1. Gabriel A, Camardo M, O'Rorke E, Gold R, Kim PJ. Effects of Negative-Pressure Wound Therapy With Instillation versus Standard of Care in Multiple Wound Types: Systematic Literature Review and Meta-Analysis. *Plast Reconstr Surg.* 2021 Jan 1;147(1S-1):68S-76S. doi: 10.1097/PRS.00000000000007614. PMID: 33347065.
2. Kim PJ, Lookess S, Bongards C, Griffin LP, Gabriel A. Economic model to estimate cost of negative pressure wound therapy with instillation vs control therapies for hospitalised patients in the United States, Germany, and United Kingdom. *Int Wound J.* 2021;1-7. doi:10.1111/iwj.13689.
3. Kim PJ, Attinger CE, Constantine T, et al. Negative pressure wound therapy with instillation: international consensus guidelines update. *Int Wound J.* 2020 Feb;17(1):174-186.
4. Hargreaves J, Lookess S, Bongards C, Siabro V, Topachevskiy O, Berndt K, Palka-Santini M, Potential Budget Impact of Negative Pressure Wound Therapy with Instillation and Dwell Versus Control Therapy Across Multiple Geographies, poster presentation at Virtual ISPOR Europe, Nov 30-Dec 3, 2021.

**NOTE: Specific indications, contraindications, warnings, precautions, and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals. Rx Only.**



© 2022 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. 3M marks used under license in Canada. 70-2013-1008-6