

RelyX™ Luting Cement



Indications:

- For permanent cementing of:
 - Porcelain-fused-to-metal crowns and bridges
 - Metal inlays, onlays, crowns and bridges
 - Crowns made with all-alumina or all-zirconia cores such as Procera® AllCeram
 - Pre-fabricated and cast endodontic posts
- Cementing orthodontic bands

Tooth Preparation:


- Remove provisional restoration and all temporary cement.
- Clean tooth with oil-free pumice paste, rinse, and dry.
- Do not desiccate the tooth.



1

Mixing:


- Use an equal number of powder scoops and liquid drops.
- Fluff the powder. Dispense 3 level scoops for crown, 6 for bridge.
- Hold liquid vial vertically and dispense 3 drops for crown, 6 for bridge. Keep the vial tip clean.
- Mix all the powder into the liquid, about 30 seconds.

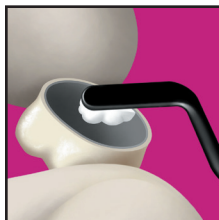

2

Load the crown/bridge:

- Spread the cement over all the internal surfaces of the crown/bridge.
- Working time is 2.5 minutes.

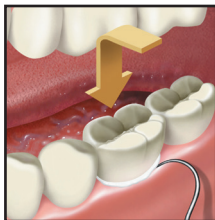RelyX™ Luting Cement**Placement:**

- Seat the crown/bridge.
- Wait at least 3 minutes after placement.
- Remove excess cement with an appropriate instrument.

3

3M ESPE Customer Care/MSDS Information:**U.S.A. 1-800-634-2249 and Canada 1-888-363-3685**

3M, ESPE, RelyX and Procera are trademarks of 3M or 3M Deutschland GmbH.

Used under license in Canada. © 3M 2014. All rights reserved.

3M ESPEDental Products, 2510 Conway Ave, MN 55144-1000 USA • www.3MESPE.com

44-0007-4669-1-A