





3M™ Cavilon™ Advanced Skin Protectant

Application around a stoma or fistula*

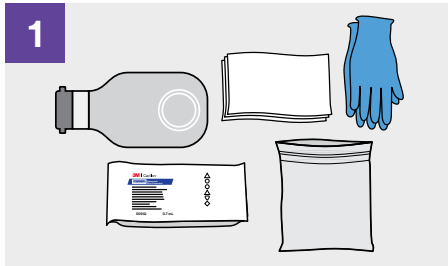

Set up your supplies on a clean surface with convenient access to the patient.


- ▶ Prepare ostomy products as needed
 - Exam gloves
 - Ostomy pouch, other ostomy supplies (e.g. ostomy paste, barrier strips) as needed
 - Plastic bag or container for waste disposal
- Non-linting soft cloths/paper towels moistened with warm water; extra cloths as needed to wick stool/urine
- 3M™ Cavilon™ Advanced Skin Protectant 0.7ml applicator

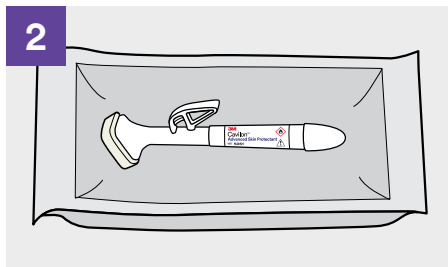

Open the package of Cavilon Advanced Skin Protectant and leave the applicator in the plastic pouch.



Position the patient and remove the ostomy pouch/flange.

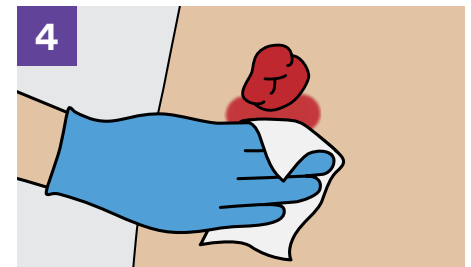

Gently cleanse the skin to remove stool/urine or residual ostomy paste or barrier material. Gently dry if needed.

- ▶ If denudement (superficial skin loss) is present, it is not necessary to apply ostomy powder prior to applying Cavilon Advanced Skin Protectant.
- ▶ If protecting skin around a tube, drain or device, cleanse the skin to remove any residual ointment/cream/paste.



Change your gloves.



Grasp the applicator and place your thumb at the end of the lever.



Aim the applicator sponge downward and firmly depress the lever to break the internal ampule.


- ▶ You will hear a snapping or popping noise when the ampule breaks.

* The instructions in this guide may also be followed when applying Cavilon Advanced Skin Protectant for skin protection around a tube, drain or device.




Keep the applicator pointed in a downward position for approximately 10 seconds to allow fluid into the foam sponge.


- ▶ The fluid flows by gravity so you don't need to maintain pressure on the lever.
- ▶ Fluid will not completely saturate the sponge to the edges.

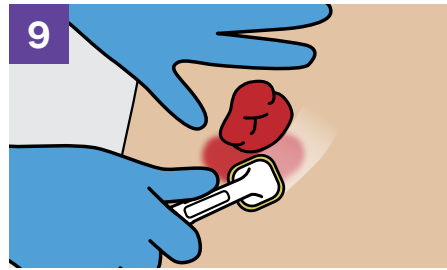

Press the foam sponge lightly against the skin and apply to any area that is denuded and/or may be exposed to stool/urine during pouch wear time. Sweep the foam sponge across the affected area to create a thin, even coating with minimal overlap. For small areas, you may apply the product with gentle blotting.

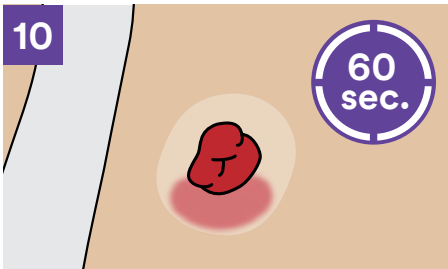

Allow the product to dry for a minimum of 60 seconds before applying the ostomy pouch or flange.

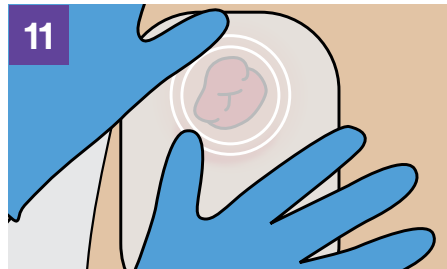

Apply the pouch or flange per manufacturer's instructions.

Note: Do not press down on the lever during application. Doing so can result in pooling of fluid and excessive stickiness.

- ▶ Continue application until the entire area of concern has been covered.
- ▶ Do not paint back and forth over same area during application.

Note: Additional instructions if using the large applicator (2.7mL) for areas less than 100 sq. inches:

- ▶ After activating the applicator, press out some of the fluid onto the back of the white top portion of the package.
- ▶ Apply the product to the patient's skin.

For skin protection around stomas or fistulas, reapply Cavilon Advanced Skin Protectant with every pouch change

- ▶ If affected area is not pouched, reapply 2-3x per week.
- ▶ At subsequent pouch changes, use an adhesive remover (containing hexamethyldisiloxane (HMDS) or a siloxane) anytime you remove an adhesive product from skin that has been coated with Cavilon Advanced Skin Protectant.

For skin protection around tubes, drains or devices, reapply 2-3x per week.

Ordering Information

3M™ Cavilon™ Advanced Skin Protectant

Cat. No	Size	Items/Box
5050	2.7 mL applicator	20
5051	0.7 mL applicator	20

If you have any questions regarding the application of 3M™ Cavilon™ Advanced Skin Protectant, please contact your local 3M representative or visit 3M.com/medical.



3M Company
2510 Conway Ave.
St. Paul, MN 55144 U.S.A.

Phone 800-228-3957
Web 3M.com/medical

Note: Specific indications, limitations, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. Rx only. Federal Law (U.S.A.) restricts the device to sale by or on the order of a health care professional.

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. Used under license in Canada. GL_70-2011-8202-2