



Impregum™ Penta™ Soft Impression Material combines the convenience of the Pentamix™ 2 Mixing Unit and the new *soft technology*.

Introducing Impregum™ Penta™ Soft Medium-Bodied Consistency Impression Material and Impregum™ Penta™ DuoSoft Heavy-Bodied/Light-Bodied Consistency Impression Material from 3M ESPE. The "Soft" family of Impregum™ Impression Materials provides unmatched accuracy and initial hydrophilicity with:

- Optimal rigidity for easier removal from the mouth and model.
- Neutral taste for increased patient comfort.

Three consistencies allow you to use whatever impression technique you prefer for virtually every clinical indication.

- Impregum Penta Soft medium-bodied consistency impression material for the one-step/monophase technique.
- Impregum Penta Soft heavy-bodied/light-bodied consistency impression material for the one step/two-viscosity technique.

Mixing is easy with the Pentamix™ 2 Mixing Unit and PentaMatic™ AutoOpen foil bags from 3M ESPE.

Automatically mix and dispense perfectly homogeneous and void-free impression material at the touch of a button. The Pentamix™ 2 Mixing Unit is:

- *Economical:* Use only the exact amount of material you need.
- *Reliable:* Perfectly mixed material assures consistent quality.
- *Hygienic:* Direct filling of tray and syringe reduces the risk of cross-contamination.
- *Efficient:* Push button activation makes it fast and easy to handle.



Impregum™ Penta™ Soft Medium-Bodied Consistency Impression Material.



Impregum™ Penta™ DuoSoft Heavy-Bodied/Light-Bodied Consistency Impression Material.



Impregum™ Light-Bodied Consistency Impression Material is available for use in both the Pentamix™ Mixing Unit and the Garant™ Automix Delivery System.



One-step tray/syringe Technique



Step 1. For initial use of a new cartridge, extrude a small amount of material and discard it.



Step 2. Begin filling the tray.



Step 3. Simultaneously start syringing impression material around the teeth.



Step 4. Finish filling tray.



Step 5. Insert the impression tray in the mouth.



Step 6. Finished impression.



1. Insert the Pentamix™ Foil Bag into the Pentamix™ Cartridge.



2. Place the cartridge into the Pentamix™ 2 mixing unit.



3. Dispense automatically into tray and Pentamix™ Syringe.

Packaging

Impregum™ Pentamix™ Soft Medium-Bodied Consistency Impression Material

31734 – Introductory Kit

- 1 Base paste - 300ml
- 1 Catalyst - 60ml
- 1 Pentamix cartridge
- 10 Pentamix mixing tips
- 1 Pentamix Elastomeric Syringe
- 1 Polyether Tray Adhesive

31730 – Refill Pack

- 2 base paste - 300ml
- 2 catalyst - 60ml

77787 – Pentamix Cartridge

- 1 Impregum Pentamix Soft Cartridge

Impregum™ Pentamix™ H DuoSoft Heavy-Bodied Consistency Impression Material

31738 – Introductory Kit

- 1 Base paste - 300ml
- 1 Catalyst - 60ml
- 1 Pentamix cartridge
- 10 Pentamix mixing tips
- 2 Garant 2 cartridges - 50ml
- 10 Garant 2 mix tips, white
- 10 Garant 2 application tips, white
- 1 Garant 2 dispenser 1:1/2:1
- 1 Polyether Tray Adhesive

31740 - Refill Pack

- 2 base paste - 300ml
- 2 catalyst - 60ml

77793 - Pentamix Cartridge

- 1 Impregum Pentamix H DuoSoft Cartridge

Impregum™ Pentamix™ L DuoSoft Light-Bodied Consistency Impression Material

31744 - Introductory Kit

- 1 Base paste - 300ml
- 1 Catalyst - 60ml
- 1 Pentamix cartridge
- 20 Pentamix mixing tips
- 1 Pentamix Elastomeric Syringe

31745 - Refill Pack

- 2 base paste - 300ml
- 2 catalyst - 60ml

77795 - Pentamix Cartridge

- 1 Impregum Pentamix L DuoSoft Cartridge

Impregum™ Garant™ L DuoSoft Light-Bodied Consistency Impression Material

31751 - Refill Pack

- 4 Garant 2 cartridges - 50ml
- 20 Garant 2 mixing tips, white

Pentamix™ Accessories

71510 - Pentamix Mixing Tips Refill Pack

- 50 Pentamix mixing tips

71210 - Pentamix Elastomeric Syringe

- 1 syringe (incl. replacement body, tips, cleaning brush)

77600 - Pentamix Wall Mounting Unit

Garant™ Accessories

71451 - 50 Garant Mixing Tips, white

71461 - 50 Garant Intraoral Tips, white

77580 - 1 Garant Dispenser 1:1/ 2:1

77590 - 1 Garant Plunger 1:1/ 2:1

3M ESPE

Dental Products

3M Center
Building 275-2SE-03
St. Paul, MN 55144-1000

www.3MESPE.com/LAdental



Recycled paper

3M, ESPE, Impregum, Pentamix and Garant are trademarks of 3M or 3M ESPE AG. Printed in U.S.A.
© 3M 2002
All rights reserved.