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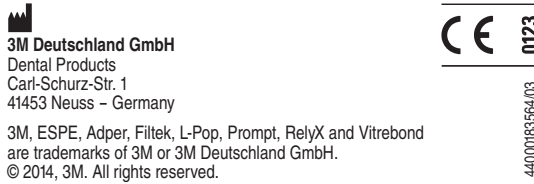
3M Deutschland GmbH
Dentale producten
Caril-Schicht-SiC
14153 Neuss - Germany

3M, ESPE, Adper, Filtek, L-Pop, Prompt, RelyX and Vitrebond are trademarks of 3M or 3M Deutschland GmbH. © 2014, 3M. All rights reserved.

ENGLISH

Product Description

Adper Prompt L-Pop is a self-etching adhesive in L-Pop blisters for one-time use. Both components in the blister package are activated, making the solution ready to use. Etching, priming, and bonding take place in one step while it is being brushed on. The adhesive is polymerized with halogen, laser, LED, or plasma lights.

⚠ Instructions for Use should not be discarded for the duration of product use.

Indications

- Bonding between dentin/enamel and light-curing composite filling materials
- Bonding between dentin/enamel and light-curing compomer filling materials
- Desensitization of exposed root surfaces
- Bonding of light-curing pit and fissure sealants
- Bonding between RelyX® Fiber Post root posts and light-curing composite core build-ups materials, e.g. Filtek™ Z250.

Contraindications

- Because of the high bond to enamel and the attendant risk of enamel damage upon bracket removal, the use of Adper Prompt L-Pop for orthodontic applications is contraindicated.
- Adper Prompt L-Pop is suited for purely light-curing materials, but not for dual-curing or self-curing materials.

Precautions

For Patients and Dental Personnel
Contact with eyes can cause severe eye damage. Wear eye protection to prevent injury. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

For Patients

- This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies.
- If prolonged contact with oral tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use.

For Dental Personnel

- This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water.
- The use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If the product contacts the glove, remove and discard the glove, wash the hands immediately with soap and water and then re-glove.
- If allergic reaction occurs, seek medical attention as needed.

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Precautions During Processing

- Use a rubber dam if necessary. Superficial changes of the gingivae, which may occur in some cases, are reversible.
- Eugenol-free products should be used, as relining materials that contain eugenol, can adversely affect the polymerization of the adhesive.

Preparation

- Remove loose preparation debris by spraying with water.
- Use 2-3 brief blows of air to dry the cavity. The cavity should only be dried to the extent that there is no water visible on the surface. **Over-drying may result in post operative sensitivity.**

- Clean untreated tooth structure with a rubber cup and an oil and fluoride-free cleaning paste (e.g. pumice paste); rinse thoroughly with water spray and dry.

Pulp Protection

To prevent pulp irritation, areas of deep dentin should be covered with calcium hydroxide preparation. The calcium hydroxide preparation should be covered with a relining material, e.g. Vitrebond®.

- Mixing Adper Prompt L-Pop**
- Attention:** Do not activate Adper Prompt L-Pop without a disposable applicator.
 - Hold the shaft of the disposable applicator with one hand.
 - With the thumb and index finger of the other hand, squeeze the red fluid reservoir, starting with the outer end, in the direction of the disposable applicator.

- When the reservoir has been fully squeezed, the yellow blister area will be markedly convex toward the outside. This clearly visible convexity indicates that the blister has been properly activated.

- Squeeze the red reservoir until it is completely empty and carefully fold it at the connection to the yellow reservoir.
- To prevent the fluid from flowing back, keep the red reservoir squeezed together while folding.**

- The red and yellow blister sections are now on top of each other (only the green end of the blister is now visible).

- Beginning at the outer end again, squeeze all fluid from the yellow section into the green section of the blister. Use measured pressure and cover the blister opening at the shaft of the applicator with your thumb. This will prevent the contents from spraying out and coming into contact with skin and clothing.

- As soon as the fluid has flowed into the green area of the blister, turn the disposable applicator back and forth in the fluid. This will mix the fluid again and cover the applicator completely.

- For easier application in the cavity, the applicator can be folded while it is still in the blister. In order to do this, only pull the disposable applicator out of the blister far enough that the intended folding area (narrow section of the applicator shaft) can be seen at the edge of the foil, and fold the applicator at this location over your thumbal.

- When the fluid is optimally mixed, it will have a homogeneous, yellowish color without streaks. Treat each tooth individually and quickly, as described under Application.

- During treatment, move the fluid into the open end by fully squeezing out the green blister area. This allows the applicator to be recoiled without covering the applicator shaft with fluid. However, if the shaft does become covered with fluid, be sure that the fluid-covered applicator shaft does not come into contact with the mucosa.

Application

Instructions for direct light-curing restorations in enamel and dentin:
Brush the adhesive onto the entire cavity surface, and, for a treatment surface of approximately 5 x 5 mm (25 mm²), **massage it in for 15 sec, applying pressure; longer massaging times are required for larger surfaces.**

- This time cannot be shortened. It is not enough just to brush it on and distribute it!**
- The adhesive must be massaged in over the entire surface, as this is the step in which the cavity surface is also etched, conditioned, and covered with adhesive. The more thoroughly the adhesive is massaged the more effective it will be.
- Protect the disposable applicator from light during treatment.

- If the applicator is contaminated with blood or saliva, use a new Adper Prompt L-Pop or a new disposable applicator. Disposable applicators are available separately in sizes M (medium) and XS (fine).

- Use a gentle stream of air to **thoroughly** dry the adhesive to a thin film.
- Rewet the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Again, use a gentle stream of air to thoroughly dry the adhesive to a thin film.
- Harden the adhesive with a halogen or LED light for 10 sec. Harden for only 3 sec if using a laser or plasma light.
- Follow the manufacturer’s instructions for use for the filling material to be used.

Instructions for desensitization of exposed root surfaces

- Clean the surface to be treated with a rubber cup and cleaning paste free of fluoride and oil; rinse thoroughly with water spray and lightly dry.
- Brush the adhesive onto the surface to be treated and **massage it in for 15 seconds, applying pressure.**
 - This time cannot be shortened! It is not enough just to brush it on and distribute it!**
- If the applicator is contaminated with blood or saliva, use a new Adper Prompt L-Pop or a new disposable applicator. Disposable applicators are available separately in sizes M (medium) and XS (fine).
- Use a gentle stream of air to **thoroughly** dry the adhesive to a thin film.
- Rewet the brush tip with adhesive and apply a second coat. This coat does not need to be massaged into the surface.
- Again, use a gentle stream of air to thoroughly dry the adhesive to a thin film. **Pooling of the bonding material during application is to be avoided.**
- Harden the adhesive with a halogen or LED light for 10 sec. Harden for only 3 sec if using a laser or plasma light.
- Remove the inhibition layer with a moistened gauze.

Instructions for bonding light-cured dental sealants:

Brush the adhesive onto the surface to be sealed, and massage it in for 15 seconds, **applying pressure; longer massaging times are required for larger surfaces.**

- Do not shorten the time. It is not enough just to brush it on and distribute it!**
- The adhesive must be massaged in over the entire surface, as this is the step in which the cavity surface is also etched, conditioned, and covered with adhesive. The more thoroughly the adhesive is massaged in, the more effective it will be.

- Protect the disposable applicator from light during treatment.
- If the applicator is contaminated with blood or saliva, use a new Adper Prompt L-Pop or a new disposable applicator.

- Use a gentle stream of air to **thoroughly** dry the adhesive to a thin film. A second layer is not required for bonding sealants.
- Apply a light-cured sealant.
- Harden the sealant and adhesive together with a halogen, LED or plasma light for the amount of time specified to cure the sealant.

Instructions for core build-ups on RelyX Fiber Post root posts.

- Cement the RelyX Fiber Post root post according to the RelyX Fiber Post Instructions for Use.
- Apply the adhesive on the RelyX Fiber Post root post surface and the tooth structure as described for direct light-curing restorations. Start the application on the root post and proceed with the tooth structure.
- With root posts the adhesive needs not to be massaged in with pressure.
- Proceed according to the Instructions for Use of the respective core build-up material.

Notes

It is extremely important for adhesion that both adhesive components be completely mixed. Using incompletely activated Adper Prompt L-Pop blisters can cause the adhesion effect to be eliminated. Massaging the adhesive only partially or shortening the massage time also compromises adhesion.

Disposal

Only dispose of completely empty packages. The residue of the etching fluid – if heavily diluted with water or neutralized – can be poured down the drain.

Storage and Shelf Life

Adper Prompt L-Pop blisters should be protected from light. Store the product at 15–25°C/59–77°F.

Do not use after the expiration date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH’s sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Information as of December 2011

RUS: Authorised representative of the manufacturer in RF: ZAO “3M Russia”, Russia 121614, Moscow, Krylatskaya str. bldg. 173 Т. +74957847474

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