3M™ Tegaderm™ Transparent Film Dressing Frame Style

1626, 1627, 1628, 1629, 1630, 1634, 1622W, 1624W, 1626W, 9505W, 9506W

3M™ Tegaderm™ HP Transparent Film Dressing Frame Style

9519HP, 9534HP, 9536HP, 9543HP, 9545HP, 9546HP, 9548HP

3M™ Tegaderm™ Transparent Film Dressing with Border

1614, 1616

3M™ Tegaderm™ I.V. Transparent Film Dressing with Border

1610, 1650, 1655, 9525HP, 9634HP

3M™ Tegaderm™ Transparent Film Dressing First Aid Style

1620, 1621

Product Description Tegaderm™ Film consists of a thin film backing with a non-latex, hypoallergenic adhesive. Tegaderm™ Film with Border is notched and reinforced with soft cloth tape to provide a better seal around catheters and other devices. The dressing is breathable, allowing good oxygen and moisture vapor exchange. It is waterproof and impermeable to liquids, bacteria, and viruses. * An intact dressing protects the site from outside contamination. *In vitro testing shows that the film of Tegaderm™ and Tegaderm™ HP dressings provides a viral barrier from viruses 27 mm in diameter or larger; the dressing remains intact without leakage.	®
--	----------

Indications for Use Tegaderm™ Film can be used to cover and protect catheter sites and wounds, to maintain a moist environment for wound healing or to facilitate autolytic debridement, as a secondary dressing, or around catheters and other devices. The dressing is breathable, allowing good oxygen and moisture vapor exchange. It is waterproof and impermeable to liquids, bacteria, and viruses. * An intact dressing protects the site from outside contamination. *In vitro testing shows that the film of Tegaderm™ and Tegaderm™ HP dressings provides a viral barrier from viruses 27 mm in diameter or larger; the dressing remains intact without leakage.	®
--	----------

Warnings

Using the Tegaderm dressing to cover a drug delivery patch could lead to faster absorption, increasing the risk for overdoses.
Caution, see instructions for use.
Do not stretch the dressing during application as tension can cause skin trauma.
Make sure the skin is clean, free of soap residue and lotion and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion.
The dressing may be used on an infected site, only when under the care of a health care professional.
Antimicrobial ointments containing polyethylene glycols may compromise the strength of the Tegaderm™ HP Transparent Film Dressings.

Shelf Life:

For further information outside of the United States, contact your local 3M representative or contact us at 3M.com and select your country.

Caution, see instructions for use Do not use if package is damaged Do not reuse	Use by date Batch code Manufacturer Date of manufacture Sterilized using irradiation
--	---

Not Made With Natural Rubber Latex Caution, see instructions for use Do not use if package is damaged Do not reuse	Chargenbezeichnung Hersteller Herstellungsdatum Sterilisation durch Bestrahlung
--	---

Verwendbar bis	STERILE 1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22 23 24 25 26 27 28 29 30 31 32 33 34 35 36 37 38 39 40 41 42 43 44 45 46 47 48 49 50 51 52 53 54 55 56 57 58 59 60 61 62 63 64 65 66 67 68 69 70 71 72 73 74 75 76 77 78 79 80 81 82 83 84 85 86 87 88 89 90 91 92 93 94 95 96 97 98 99 100
-----------------------	--

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