



Prevena[™]
Incision Therapy

PROMISES study data suggests 3M[™] Prevena[™] Therapy can help advance the standard of care

Promising new data from a randomised controlled trial further affirms that Prevena Therapy significantly reduces the risk of 90-day surgical site complications (SSCs) and postop readmissions



The PROMISES (post-market, randomised, open-label, multicentre study to evaluate effectiveness) trial

The effectiveness of closed incision negative pressure therapy versus silver-impregnated dressings in mitigating surgical site complications in high-risk patients after revision knee arthroplasty.

Higuera-Rueda CA, Emara AK, Nieves-Malloure Y, Klika AK, Cooper HJ, Cross MB, Guild GN, Nam D, Nett MP, Scuderi GR, Cushner FD, Piuze NS, Silverman RP. J. Arthroplasty 2021; doi: 10.1016/j.arth.2021.02.076

Study design

Post-market, randomised, open-label, multicentre study.

Study purpose

Evaluate the effectiveness of closed incision negative pressure therapy (ciNPT) versus standard of care (SOC) dressings in reducing surgical site complications (SSCs).

Methods

- A total of 294 revision total knee arthroplasty (rTKA) patients (15 centres) at high-risk for wound complications were randomised to ciNPT or SOC (n=146 each) and stratified by revision type (aseptic vs. septic). Demographics, comorbidities, causes of revision and duration of treatment were similar between cohorts (p>0.05).
- 242 patients with incisions completed follow-up, including 124 patients treated with 3M™ Prevena™ Therapy (ciNPT) and 118 patients treated with an antimicrobial silver-impregnated dressing (SOC).
- Primary outcome was the 90-day incidence of SSCs with stratification in accordance with revision type. Secondary outcomes were the 90-day health care utilisation parameters (readmission, reoperation, dressing changes, and visits) and patient-reported outcomes (PRO). Treatment-related adverse events were compared and stratified as severe and non-severe.

Results

Compared to SOC, patients in the Prevena Therapy group demonstrated:

- Significantly decreased rates of surgical site complications (ciNPT 3.4% vs. SOC 14.3%, **p=0.0013***)
- Significantly lower readmission rates (ciNPT 3.4% vs. SOC 10.2%, **p=0.0208***)
- Reduced dressing changes (ciNPT 1.1+0.29 vs. SOC 1.3 +0.96, **p=0.0003***)

Conclusions

Prevena Therapy significantly mitigated 90-day surgical site complications, readmission rates, and reduced frequency of dressing changes compared with the standard of care among high-risk rTKA patients.

- Treatment-related adverse effects were similar between both cohorts.
- The benefit of ciNPT on specific SSCs and post-rTKA patient-reported outcomes (PRO) was not established and further studies are warranted.

*Calculation(s) are derived based on relative patient group incidence rate reported in this study. *Statistically significant (p<0.05)

Patients treated with 3M™ Prevena™ Therapy were:

4x less likely
to experience a post-operative 90-day surgical site complication.

3x less likely
to be readmitted compared to the standard of care group.

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

3M

3M United Kingdom PLC
Charnwood Campus
10 Bakewell Road
Loughborough
LE11 5RB
01509 611 611

3M Ireland Ltd
The Iveagh Building
The Park, Carrickmines
Dublin
D18 X015
Ireland
1 800 320 500

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited.
PRA-PM-EU-00616 (12/21)