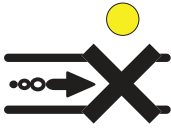
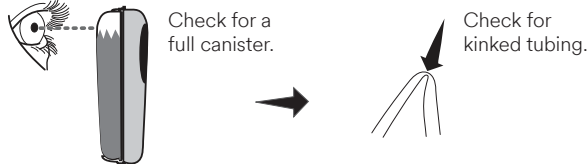

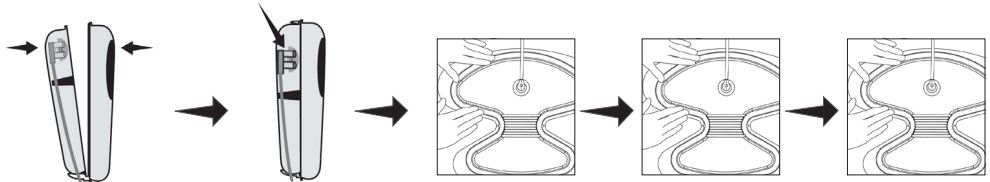

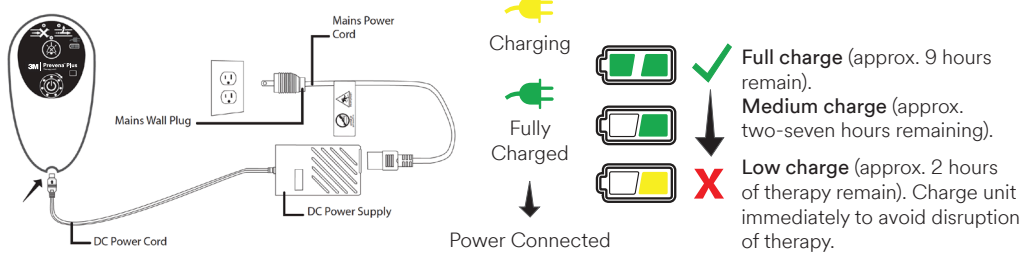

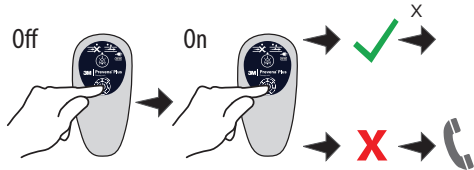






Prevena Restor™

Postoperative Recovery

3M™ Prevena™ Plus 125 Therapy Unit — 14 Day Indicators and Alerts

Alert type	How to fix
<p>Blockage Alert</p>  <p>Audible blockage alert will sound two beeps repeating every 15 seconds.</p>	 <p>Check for a full canister.</p> <p>Check for kinked tubing.</p>
<p>Leak Alert</p>  <p>Leak Alert will sound two beeps repeating every 15 seconds.</p>	 <p>Ensure canister is security locked onto the therapy unit. When canister is installed, a distinct click will be heard.</p> <p>Check dressing tubing connector at canister.</p> <p>Press down through drape onto the adhesive seal all the way around to ensure seal.</p> <p>Use fingers to smooth out any creases or wrinkles.</p> <p>Use 3M™ Prevena™ Patch Strips or excess drape along the outer edge of drape to seal leaks.</p>
<p>Low Battery Alert</p>  <p>Alert will sound two beeps repeating every four minutes.</p>	 <p>Mains Wall Plug, Mains Power Cord, DC Power Supply, DC Power Cord</p> <p>Charging (Yellow plug icon)</p> <p>Fully Charged (Green plug icon)</p> <p>Power Connected</p> <p>Full charge (approx. 9 hours remain). Medium charge (approx. two-seven hours remaining). Low charge (approx. 2 hours of therapy remain). Charge unit immediately to avoid disruption of therapy.</p>
<p>System Fault Alert</p>  <p>All LEDs flashing and two beeps every 15 seconds.</p>	<p>Turn therapy unit off then on.</p>  <p>Continue therapy</p> <p>Call 1-800-668-5403</p>
<p>Alert Mute Button</p> 	<p>Press and hold centre of the alert mute button for three seconds during an alert condition to silence the audible alert for two minutes. When pressed, the alert mute button will illuminate to indicate mute has been selected. The alert will re-occur after two minutes unless the alert condition has been corrected.</p>
<p>Therapy Ended</p>  <p>When therapy begins, all seven green LEDs are illuminated. During the last six days of therapy, after each 24-hour period, an indicator will turn off.</p>	<ul style="list-style-type: none"> • A solid yellow LED at the top of the Therapy Life Indicator will turn on • The therapy unit will sound eight beeps, followed by a continuous beep for five seconds, then the therapy unit will turn off • Notify the treating physician or caregiver: If the therapy unit has completed the 14 day therapy and has timed out, and an attempt is made to turn the therapy on, the therapy unit will sound an alert for three seconds then shut off



Available in Canada from your authorized 3M-KCI distributors.
 KCI USA, Inc., a 3M Company
 KCI owned and operated by 3M Company

KCI Medical Canada Inc.
 75 Courtneypark Dr W, Unit 4
 Mississauga, ON
 L5W 0E3

KCI USA, INC.
 2930 IH 10 West
 San Antonio, TX
 78249

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product Instructions For Use prior to application. This material is intended for healthcare professionals

3M and Prevena are trademarks of 3M. Used under license in Canada. © 2021, 3M. All rights reserved. Other marks shown are marks and/or registered marks. Unauthorized use prohibited. PRA-PM-CA-00289 (12/21) 2112-22645-E