


# 3M™ Kerramax Care™ Super-Absorbent Dressing

## 3M™ Kerragami™

Kerramax Care Dressings are highly conformable and can be folded to suit a variety of different wounds.

### In-between toes/fingers

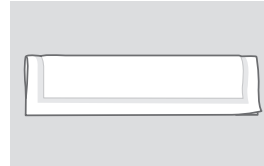


**1** Fold 10cm x 22cm dressing in half lengthways




**2** Weave the dressing in between toes or fingers


### Heel/elbow

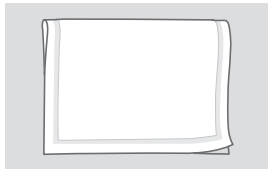


**1** Fold 10cm x 22cm dressing in half

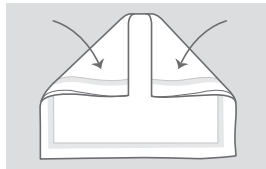


**2** Fold each corner inwards

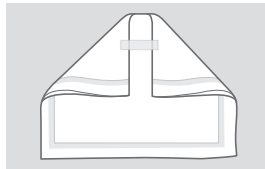

**3** Place tape to hold folded corners in place

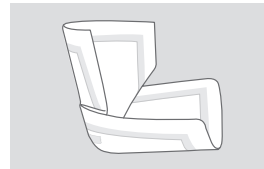

**4** Open out folded dressing

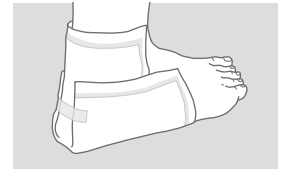

**5** Place on heel or elbow


### Leg/arm



**1** Place 20cm x 50cm dressing on the base of the leg or arm



**2** Wrap the dressing around the leg or arm

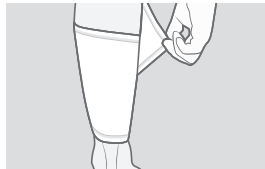


**3** Overlap the dressing to ensure there are no gaps



**4** Secure dressing with retention or compression bandaging

### How much fluid can Kerramax Care Dressings absorb?

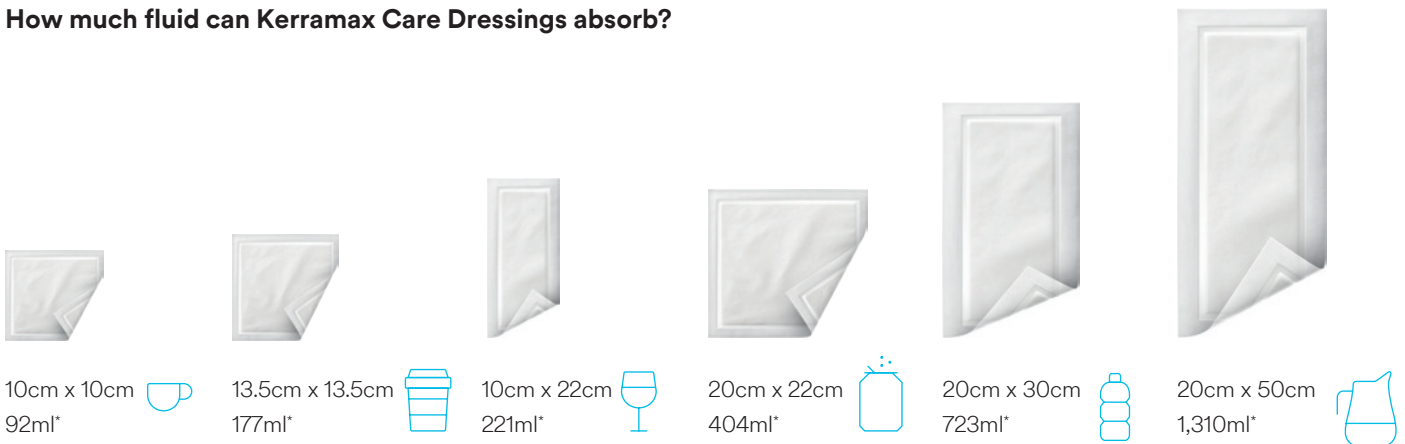

**To learn more about Kerramax Care Dressings contact your local 3M sales representative.**

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies.

3M United Kingdom PLC · Charnwood Campus · 10 Bakewell Road · Loughborough · LE11 5RB · 01509 611611

Please consult a healthcare provider and product instructions for use prior to application. This material is intended for healthcare professionals.

\*Lodhi, N. & Warde, D. Free-swell absorbency and absorbency under compression testing of Kerramax Care (all SKUs) – internal report CHC R839. Crawford Healthcare Ltd. Knutsford, UK: 2019. 3M, Kerramax Care and Kerragami are trademarks of 3M Company. © 3M 2022. All rights reserved. OMG284924.