



Prevena™
Incision Therapy

3M™ Prevena™ 125 Therapy Unit indicators and alerts



Caution Indicator

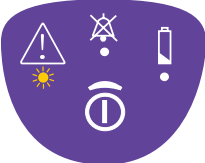


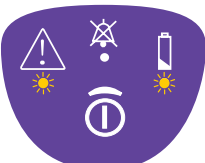
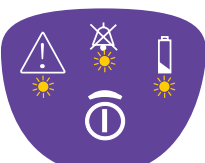


Audible Alarm (muted)

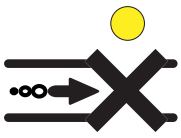
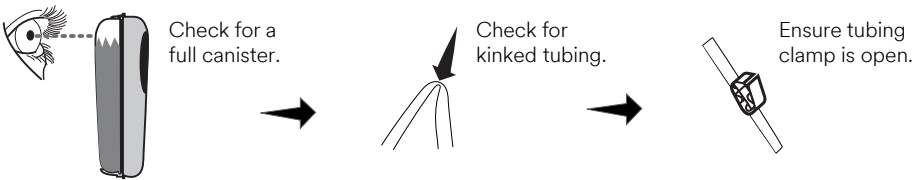
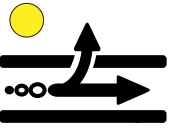
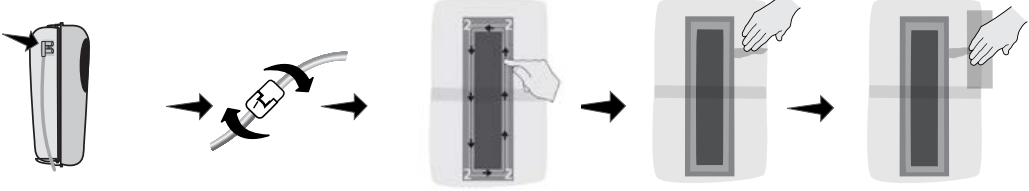
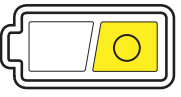
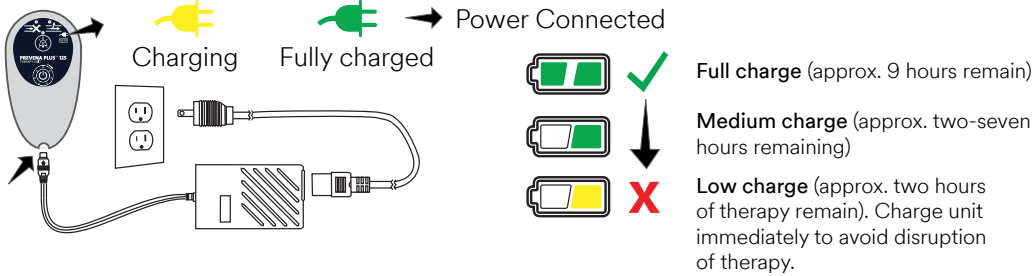

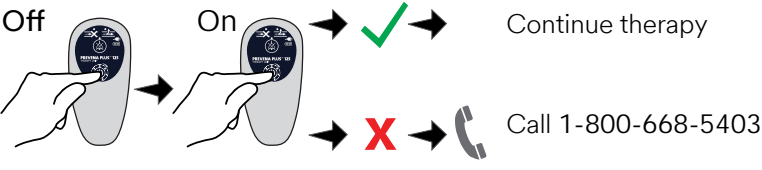




Battery Level

This is a quick guide, further details are in the 3M™ Prevena™ Therapy Clinician Guide.

Alert type	How to fix
<p>Leak Alert One beep, one solid yellow light.</p> 	<p>To ensure proper application of the 3M™ Prevena™ Incision Dressing, the Prevena 125 Therapy Unit provides the Visicheck™ Feature. Double press ON/OFF button.</p> <ul style="list-style-type: none"> - One light: Best - Two lights: Good - Three lights: Marginal <p>If the Visicheck™ Feature indicates a “Marginal” leak rate condition, follow these steps:</p> <ul style="list-style-type: none"> - Ensure canister is locked into the unit - Check the tubing connectors to make sure they are fully engaged and locked - Use finger to smooth out any creases in the adhesive - Use Prevena Incision Dressing Patch Strips (located in the dressing package) to help seal leaks - Slowly press firmly around dressing edges
<p>Canister Full Alert Two beeps, one solid yellow light.</p> 	<p>Check fluid level in the canister</p> <ol style="list-style-type: none"> 1. If the canister isn't full check for kinks or pinched tubing 2. If the canister is full then turn the unit off 3. Call 1-800-668-5403
<p>Low Battery Alert Low - One slow beep, one solid yellow light. Critical - One beep, repeating rapidly, increasing volume, two solid yellow lights.</p> 	<p>This therapy unit comes with three AA batteries</p> <ol style="list-style-type: none"> 1. Hold five seconds to turn off 2. Press and push the panel on the back of the unit to open 3. Insert three AA batteries (lithium batteries recommended) 4. Press and push to close panel 5. Hold two seconds to turn on
<p>System Error Alert One Beep, repeating rapidly, increasing in volume, two solid yellow lights.</p> 	<ol style="list-style-type: none"> 1. Hold for five seconds to turn off 2. Hold two seconds to turn back on 3. If the therapy is still not running call 1-800-668-5403
<p>Device Lifecycle Expired Three lights show life cycle is up. There will be a repeated beep for 15 seconds and then the unit will shut off.</p> 	<p>Call 1-800-668-5403</p> <p>Life cycle=192 hours (eight days)</p>

3M™ Prevena™ Plus 125 Therapy Unit indicators and alerts

Alert type	How to fix
<p>Blockage Alert</p>  <p>Audible blockage alert will sound two beeps repeating every 15 seconds.</p>	 <p>Check for a full canister.</p> <p>Check for kinked tubing.</p> <p>Ensure tubing clamp is open.</p>
<p>Leak Alert</p>  <p>Leak Alert will sound two beeps repeating every 15 seconds.</p>	 <p>Ensure canister is security locked onto the therapy unit. When canister is installed, a distinct click will be heard.</p> <p>Check tubing connectors to ensure they are fully engaged and locked.</p> <p>Press down through drape onto the adhesive seal all the way around to ensure seal.</p> <p>Use fingers to smooth out any creases or wrinkles.</p> <p>Use 3M™ Prevena™ Patch Strips or excess drape along the outer edge of drape to seal leaks.</p>
<p>Low Battery Alert</p>  <p>Alert will sound two beeps repeating every four minutes.</p>	 <p>Charging</p> <p>Fully charged</p> <p>Power Connected</p> <p>Full charge (approx. 9 hours remain)</p> <p>Medium charge (approx. two-seven hours remaining)</p> <p>Low charge (approx. two hours of therapy remain). Charge unit immediately to avoid disruption of therapy.</p>
<p>System Fault Alert</p>  <p>All LEDs flashing and two beeps every 15 seconds.</p>	<p>Turn therapy unit off then on.</p>  <p>Off</p> <p>On</p> <p>Continue therapy</p> <p>Call 1-800-668-5403</p>
<p>Alert Mute Button</p> 	<p>Press and hold center of the alert mute button for three seconds during an alert condition to silence the audible alert for two minutes. When pressed, the alert mute button will illuminate to indicate mute has been selected. The alert will re-occur after two minutes unless the alert condition has been corrected.</p>
<p>Therapy Ended</p> 	<ul style="list-style-type: none"> • A solid yellow LED at the top of the Therapy Life Indicator will turn on • The therapy unit will sound eight beeps, followed by a continuous beep for five seconds, then the therapy unit will turn off • Notify the treating physician or caregiver: If the therapy unit has completed the seven day therapy and has timed out, and an attempt is made to turn the therapy on, the therapy unit will sound an alert for three seconds then shut off



Available in Canada from your authorized 3M-KCI distributors.
KCI USA, Inc., a 3M Company
KCI owned and operated by 3M Company

KCI Medical Canada Inc.
75 Courtneypark Dr W, Unit 4
Mississauga, ON
L5W 0E3

KCI USA, INC.
12930 IH 10
West San
Antonio, TX
78249

NOTE: Specific indications, limitations, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product Instructions For Use prior to application. This material is intended for healthcare professionals.

3M and Prevena are trademarks of 3M. Used under license in Canada. © 2022, 3M. All rights reserved. Other marks shown are marks and/or registered marks of their respective owners. Unauthorized use prohibited. 2204-23874 E.