

**3M**  
**Nexcare™**  
**PRODUCTS**  
**TEGADERM™**  
**+PAD**  
**Transparent Dressing**

If you have any questions or comments, call 1-800-537-2191.

If you've enjoyed this product, please visit [www.nexcare.com](http://www.nexcare.com) or scan the QR Code and leave a review.



**Product Description**

Nexcare™ Tegaderm™ +Pad Transparent Dressing is a waterproof, germ barrier dressing. The dressing consists of a non-adherent, absorbent pad bonded to a larger thin film backing with a non-latex, hypoallergenic adhesive.

**Precautions:**

- The dressing is NOT to be used on dirty, contaminated, infected, or punctured wounds. Consult your medical provider if you notice any itching, burning, rash, or signs of infection: fever, pain, redness, or swelling.
- Make sure the skin is clean, free of soap residue and lotion and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion.
- Do not stretch the dressing during application as tension can cause skin trauma.
- Stop any bleeding at the site before applying the dressing.

**Warnings:**

- The dressing should not be used in place of stitches or to hold wound edges together.

**Dressing Care**

The site should be observed for signs of infection or other complications. If infection is suspected, remove the dressing, assess the site directly, and determine appropriate medical intervention. Infection may be signaled by fever, pain, redness, swelling or an unusual odor or discharge.

**Change the dressing if:**

- It begins to leak or is no longer sticking to the skin
- There are holes or tears in the dressing
- As recommended by your doctor

**Alternate Method for Removal (Peel Method):**

- Gently grasp an edge and slowly peel the dressing back upon itself in the direction of hair growth. Do not pull it up from the skin.

**Storage & Shelf Life**

- For best results, store at room temperature. Avoid excess heat and humidity. Refer to the expiration date on the package.

Sterility guaranteed unless individual wrapper is opened or damaged.

3M Company  
2510 Conway Avenue  
St. Paul, MN 55144 USA  
[www.nexcare.com](http://www.nexcare.com)

Questions? 1-800-537-2191  
3M, Nexcare and Tegaderm are  
trademarks of 3M.

Made in U.S.A with  
globally sourced materials.  
© 2022 3M.  
All rights reserved.  
34-8728-1736-5