Adhesive Wound Dressing

- (b) Adhesive Wound Dressing
- (c) Pansement adhésif stérile
- (d) Sterile Wundverband
- (e) Medicazione adesiva sterile
- (f) Apósito adhesivo estéril
- (g) Zelfklevend wondverband
- (h) Häftende Verbundbinde mit dyna
- (i) Selvklebende sårbindning
- (j) Selvklebende sårbandasje
- (k) Līmītājēhaasīvās
- (l) Pienso adhesivo compresa
- (m) Aurokōtoōlo Emphōto Tρoupiotou me Tropou
- (n) Jalopatrunek samoprylepny
- (o) Ontpadded Fedóköté
- (p) Adhezivni kryti ran
- (q) Náplast na krytie rán
- (r) Oblik za rane
- (s) Liimuv haavaplaaster
- (t) Pasipons bruci parisenanais materials
- (u) Lipus zaidzio tvartis
- (v) Pansament adesivo sterli
- (w) Повязка для закрытия ран
- (x) Адгезивна повязка для закриття ран
- (y) Ljepljivi povoj za rane
- (z) Адгезивна прев'язка за рани
- (aa) Hirunki flaster sa jastucem
- (bb) Emci Pedli Steril Hazir Yara Ortusu

Product Description

The 3M™ Medipore™ + Pad Adhesive Wound Dressing consists of a non-adherent absorbent pad bonded to a larger soft cloth backing coated with a border of water-resistant adhesive. The dressing is presented sterile.

Intended use:

The 3M™ Medipore™ + Pad adhesive wound dressing is designed to be used as a mechanical barrier and for absorption of exudates from acute wounds.

Intended Users:

Intended users are health care professionals and lay persons following the direction of the professional and instruction for use.

Warnings and Precautions:

- The 3M™ Medipore™ + Pad adhesive dressing is not intended to replace sutures or other primary wound closure methods.
- The dressing should be changed if it becomes compromised or exudate penetrates through the soft cloth backing.
- The site should be observed frequently for signs of infection. If infection occurs (which may be signaled by fever, pain, redness, swelling, or an unusual odor or discharge), remove the dressing and take standard precautions.
- Follow the directions provided by your doctor and instruction for use. If the wound does not begin to show signs of healing or if any other unexpected symptoms occur, consult a health care professional.
- Haemostasis of the wound should be achieved before applying the dressing.
- The skin should be dry and free of detergent residue to ensure good adhesion.
- Mechanical skin trauma may result if it is applied.
- When applied on areas of high mobility or considerable tissue swelling is anticipated, tension-related skin injuries may occur.
- Do not reuse. Reuse may lead to infection or other illness/injury.
- If the sterile packaging is damaged or unintentionally opened, discard the product; do not use.

Directions for Use:

Follow your "gauze and tape" protocol for use of this dressing.

Preparation: In accordance with standard hospital protocol, prepare the site for wound dressing. To ensure best dressing adhesion, clipping of excess hair at the site may be necessary. Shaving is not recommended. Apply to a clean, dry site. For best results, the dressing should be large enough to have a margin which adheres to dry, skinny skin around the wound site.

1. Open package and remove dressing.

2. Peel the paper liner from the dressing, exposing the adhesive surface.

3. Position the dressing over the wound site and apply dressing. To avoid skin tension, flex joints before application.

4. Smooth down the dressing edges. Seal securely around wound site. Firmly smooth adhesive border to the skin.

5. Check tension on the dressing regularly, particularly if the wound is subject to swelling.

6. Change the dressing should exudate penetrate through the soft cloth backing.

7. To remove the dressing, gently grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

Storage / Shelf Life / Disposal:

For best results, store 3M™ Medipore™ + Pad adhesive dressing in normal room temperatures. Avoid excessive heat and humidity. For shelf life, refer to the expiration date which is printed on each package. Do not use if expired or if expiration date is missing or illegible. Discard according to local country legislation.

How Supplied and Ordering Information:

Supplied in boxes of individually packaged sterile dressings. Dressing is sterile unless package is unintentionally opened or damaged.

Customer Information:

Please report a serious incident that has occurred in relation to the device to 3M and the local competent authority.

For further information, please contact your local 3M representative or contact us at 3M.com and select your country.

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