### 3M™ Tegaderm™ Transparent Film Dressing Frame Style 3M™ Tegaderm™ IV Transparent Film Dressing with Border

**Product Description**  
Tegaderm™ Film consists of a thin film backing with a non-latex, hypoallergenic adhesive. Tegaderm™ Film with Border is notched and reinforced with soft cloth tape to provide a better seal around catheters and other devices. The dressing is breathable, allowing good oxygen and moisture vapor exchange. It is waterproof and impermeable to liquids, bacteria, and viruses.  
An intact dressing protects the site from outside contamination.

\*In vitro testing shows that the film of Tegaderm™ dressings provides a viral barrier from viruses 27mm in diameter or larger while the dressing remains intact without leakage.

**Indications for use**

Tegaderm™ Film can be used to cover and protect catheter sites and wounds, to maintain a moist environment for wound healing, and to facilitate autolytic debridement as a secondary dressing, as a primary dressing, or as an adjunctive device to a cast, to cover first and second degree burns and as a protective eye covering. Do not use the dressing as a replacement for sutures and other primary wound closure methods.

**Warnings**

Using the Tegaderm dressing to cover a drug delivery patch could lead to faster absorption, increasing the risk for overdose.

**Precautions**

1. Stop any bleeding at the site before applying the dressing.  
2. Do not stretch the dressing during application as tension can cause skin trauma.  
3. Make sure the skin is clean, free of soap residue and lotion and allowed to dry thoroughly before application to prevent irritation and ensure good adhesion.  
4. The dressing may be used on an infected site, only when under the care of a health care professional.  
5. If frequent problems occur, please seek professional advice.

**Intended users and Directions for Use**

Intended users are healthcare professionals and patients following the instructions for use and application illustrations on the back side of the dressing.

For optimal adhesion, apply 3M™ Tegaderm™ IV Transparent Film Dressing Frame Style using gentle firm pressure over entire dressing. Please report any serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

**Storage/ Shelf life/ Disposal**

For shelf life refer to the expiration date, printed on each package. If the sterile packaging is damaged or unintentionally opened before use, discard the product, do not reuse.

Note: dispose product per facility protocol. For further information, please contact your local 3M representative or contact us at 3M.com and select your country.

**Explanation of Symbols**

	Consult instructions for use
	Do not use if package is damaged or open
	Not Made With Natural Rubber Latex
	Unique Device Identifier
	Single Sterile Barrier System
	Medical Device
	Sterilized using irradiation
	1623W, 1632P-10, 1633, 1633E, 1633P, 1633KSA, 1635
	Sterilized using ethylene oxide (if part of a KIT)
	1633NS, 1635NS
	Date of issue: 2020-11
	3M and Tegaderm are trademarks of 3M Deutschland GmbH

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3M et Tegaderm sont des marques déposées de 3M Deutschland GmbH