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## 3M RelyX Universal Resin Cement

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項目內容	型號
<p><b>RelyX Universal Sample Kit TR</b>                      1x 1.2 g cement                      5 micro mixing tips                      5 elongation tips                      Scotchbond Universal Plus Vial 1.5 ml*                      Instructions for use &amp; Step by step booklet RelyX Universal                      Instructions for use &amp; Step by step Scotchbond Universal Plus</p>	56966
<p><b>RelyX Universal Sample Kit A1</b>                      1x 1.2 g cement                      5 micro mixing tips                      5 elongation tips                      Scotchbond Universal Plus Vial 1.5 ml*                      Instructions for use &amp; Step by step booklet RelyX Universal                      Instructions for use &amp; Step by step Scotchbond Universal Plus</p>	56967
<p><b>RelyX Universal Intro Kit TR, A1, A30, WO</b>                      4x 3.4 g cement Syringes Shades TR, A1, A30, WO                      60 micro mixing tips                      20 elongation tips                      Scotchbond Universal Plus Vial 5 ml*                      50 Brushes size M orange                      Scotchbond Universal Etchant 3 ml**                      25 Blue Etchant tips                      Instructions for use &amp; Step by step booklet RelyX Universal                      Instructions for use &amp; Step by step Scotchbond Universal Plus</p>	56968
<p><b>RelyX Universal Trial Kit TR</b>                      1x 3.4 g cement                      15 micro mixing tips                      5 elongation tips                      Scotchbond Universal Plus Vial 1.5 ml*                      50 Brushes size M orange                      Scotchbond Universal Etchant 3 ml**                      25 Blue Etchant tips                      Instructions for use &amp; Step by step booklet RelyX Universal                      Instructions for use &amp; Step by step Scotchbond Universal Plus</p>	56969
<p><b>RelyX Universal Trial Kit A1</b>                      1x 3.4 g cement                      15 micro mixing tips                      5 elongation tips                      Scotchbond Universal Plus Vial 1.5 ml*                      50 Brushes size M orange                      Scotchbond Universal Etchant 3 ml**                      25 Blue Etchant tips                      Instructions for use &amp; Step by step booklet RelyX Universal                      Instructions for use &amp; Step by step Scotchbond Universal Plus</p>	56970

<b>RelyX Universal Refill TR</b> 1x 3.4 g cement 15 micro mixing tips 5 elongation tips Instructions for use	56971
<b>RelyX Universal Refill A1</b> 1x 3.4 g cement 15 micro mixing tips 5 elongation tips Instructions for use	56972
<b>RelyX Universal Refill A3O</b> 1x 3.4 g cement 15 micro mixing tips 5 elongation tips Instructions for use	56973
<b>RelyX Universal Refill WO</b> 1x 3.4 g cement 15 micro mixing tips 5 elongation tips Instructions for use	56974
<b>RelyX Universal Micro Mixing Tips Refill</b> 30 micro mixing tips	56975
<b>RelyX Universal Micro Mixing Tips Refill Endo</b> 15 micro mixing tips 15 elongation tips	56976
<b>RelyX Universal Value Pack TR</b> 3x 3.4 g cement 45 micro mixing tips 15 elongation tips Instructions for use	56977
<b>RelyX Universal Value Pack A1</b> 3x 3.4 g cement 45 micro mixing tips 15 elongation tips Instructions for use	56978

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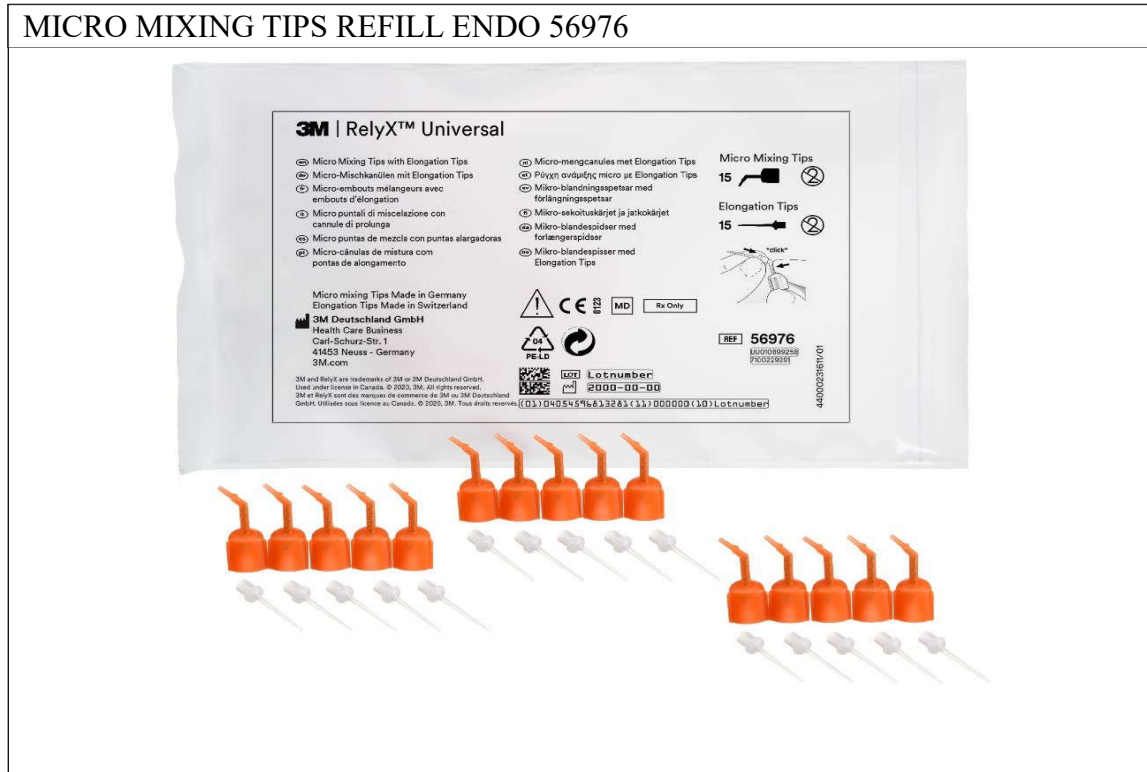
MICRO MIXING TIPS REFILL 56975

## MICRO MIXING TIPS REFILL ENDO 56976

## RelyX Universal Resin Cement Sample kit 56966 / 56967

RelyX Universal Resin Cement Value Pack 56977 / 56978

RelyX Universal Resin Cement Syringe

RelyX Universal Resin Cement Syringe with Mixing tip and Elongation tip