



# 3M™ Prevena™ 125 Therapy Unit indicators and alerts



Caution indicator





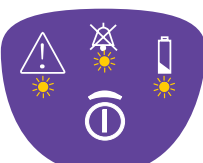


Audible alarm (muted)

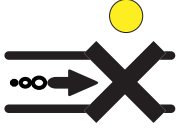
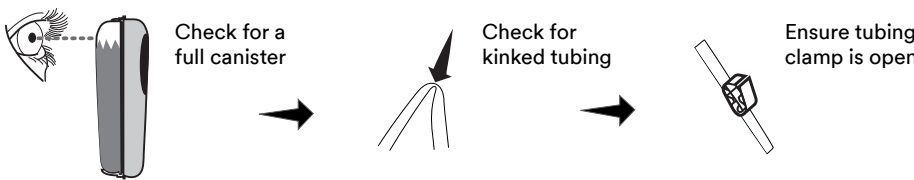

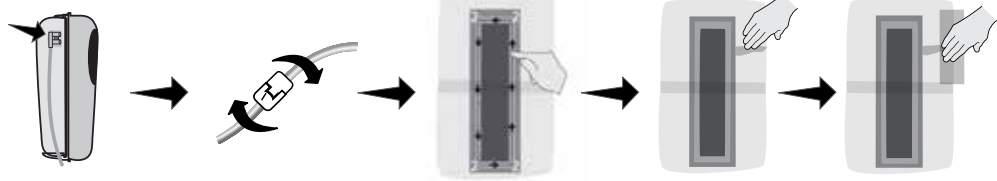

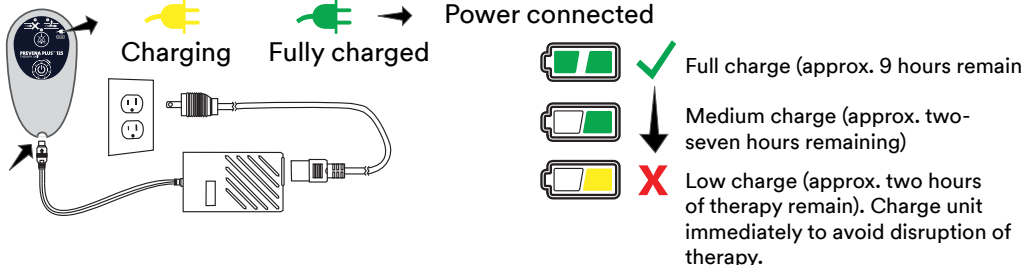
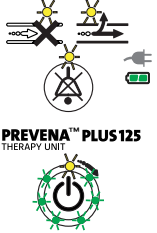
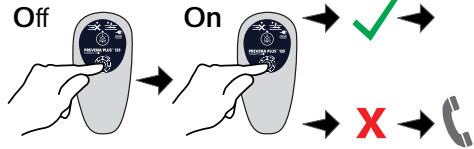




Battery level

This is a quick guide, further details are in the 3M™ Prevena™ Therapy Clinician Guide

Alert type	How to fix
<p><b>Leak Alert</b> One beep, one solid yellow light</p> 	<p>To ensure proper application of the 3M™ Prevena™ Incision Dressing, the Prevena 125 Therapy Unit provides the Visicheck™ Feature. Double press ON/OFF button.</p> <ul style="list-style-type: none"> <li>- One light: Best</li> <li>- Two lights: Good</li> <li>- Three lights: Marginal</li> </ul> <p>If the Visicheck™ Feature indicates a “Marginal” leak rate condition, follow these steps:</p> <ul style="list-style-type: none"> <li>- Ensure canister is locked into the unit</li> <li>- Check the tubing connectors to make sure they are fully engaged and locked</li> <li>- Use finger to smooth out any creases in the adhesive</li> <li>- Use Prevena Incision Dressing Patch Strips (located in the dressing package) to help seal leaks.</li> <li>- Slowly press firmly around dressing edges</li> </ul>
<p><b>Canister Full Alert</b> Two beeps, one solid yellow light</p> 	<p>Check fluid level in the canister</p> <ol style="list-style-type: none"> <li>1. If the canister is full then turn the unit off and replace canister</li> <li>2. If the canister isn't full check for kinks or pinched tubing</li> <li>3. If therapy is still alerting, call your 3M representative</li> </ol>
<p><b>Low Battery Alert</b> Low- one slow beep, one solid yellow light Critical- one beep, repeating rapidly, increasing volume, two solid yellow lights</p> 	<p>This therapy unit comes with three AA batteries</p> <ol style="list-style-type: none"> <li>1. Hold five second to turn off</li> <li>2. Press and push the panel on the back of the unit to open</li> <li>3. Insert three AA batteries (lithium batteries recommended)</li> <li>4. Press and push to close panel</li> <li>5. Hold two seconds to turn on</li> </ol>
<p><b>System Error Alert</b> One beep, repeating rapidly, increasing in volume, two solid yellow lights</p> 	<ol style="list-style-type: none"> <li>1. Hold for five seconds to turn off</li> <li>2. Hold two seconds to turn back on</li> <li>3. If the therapy is still not running call your 3M representative</li> </ol>
<p><b>Device Lifecycle Expired</b> Three lights show life cycle is up. There will be a repeated beep for 15 seconds and then the unit will shut off.</p> 	<p>The life cycle of the Prevena 125 Therapy unit is 192 hours (eight days)</p> <p>Notify the treating clinician or caregiver on completion of therapy</p>

# 3M™ Prevena™ Plus 125 Therapy Unit indicators and alerts

Alert type	How to fix
<p><b>Blockage Alert</b></p>  <p>Audible blockage alert will sound two beeps repeating every 15 seconds</p>	 <p>Check for a full canister</p> <p>Check for kinked tubing</p> <p>Ensure tubing clamp is open</p>
<p><b>Leak Alert</b></p>  <p>Leak Alert will sound two beeps repeating every 15 seconds</p>	 <p>Ensure canister is securely locked onto the therapy unit. When canister is installed, a distinct click will be heard.</p> <p>Check tubing connectors to ensure they are fully engaged and locked.</p> <p>Press down through drape onto the adhesive seal all the way around to ensure seal</p> <p>Use fingers to smooth out any creases or wrinkles</p> <p>Use 3M™ Prevena™ Patch Strips or excess drape along the outer edge of drape to seal leaks</p>
<p><b>Low Battery Alert</b></p>  <p>Alert will sound two beeps repeating every four minutes</p>	 <p>Power connected</p> <p>Charging</p> <p>Fully charged</p> <p>Full charge (approx. 9 hours remain)</p> <p>Medium charge (approx. two-seven hours remaining)</p> <p>Low charge (approx. two hours of therapy remain). Charge unit immediately to avoid disruption of therapy.</p>
<p><b>System Fault Alert</b></p>  <p>All LEDs flashing and two beeps every 15 seconds</p>	<p>Turn therapy unit off then on</p>  <p>Off</p> <p>On</p> <p>Continue therapy</p> <p>Call your 3M representative</p>
<p><b>Alert Mute Button</b></p> 	<ul style="list-style-type: none"> <li>• Press and hold center of the alert mute button for three seconds during an alert condition to silence the audible alert for two minutes.</li> <li>• When pressed, the alert mute button will illuminate to indicate mute has been selected.</li> <li>• The alert will re-occur after two minutes unless the alert condition has been corrected</li> </ul>
<p><b>Therapy Ended</b></p> 	<ul style="list-style-type: none"> <li>• A solid yellow LED at the top of the Therapy Life Indicator will turn on</li> <li>• The therapy unit will sound eight beeps, followed by a continuous beep for five seconds, then the therapy unit will turn off</li> <li>• If the therapy unit has completed its cycle and an attempt is made to turn the therapy on, the therapy unit will sound an alert for three seconds then shut off</li> <li>• Notify the treating clinician or caregiver on completion of therapy</li> </ul>

KCI Medical Australia Pty Ltd  
 Building A, 1 Rivett Road  
 North Ryde, NSW 2113  
 Ph: 1300 524 822

InterMed Medical Limited  
 71 Apollo Drive  
 Albany Auckland 0632  
 Ph: 0800 524 222

**Note:** Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

© 2023 3M. All Rights Reserved. 3M and Prevena are trademarks of 3M. All other trademarks are property of their respective owners.