
3M™ Coheban™ Self-Adherent Wrap Nexcare™ Athletic Wrap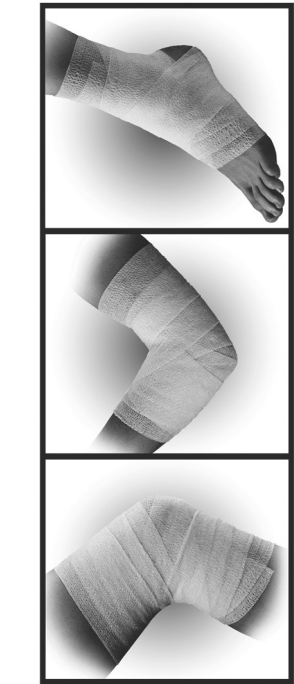

Made in Germany
3M Deutschland GmbH
Health Care Business
 Carl-Schurz-Str. 1
 41453 Neuss, Germany

© 2022, 3M. All rights reserved. Unauthorized use prohibited.
 3M, Coheban and Nexcare are trademarks of 3M.
 3M.com

Date of issue 2022-10
 DV-0163-0381-5/1022 3100041857/1022

3M™ Coheban™ Nexcare™ Self-Adherent Wrap Athletic Wrap

Product Description:

3M™ Coheban™ Self-Adherent Wrap and Nexcare™ Athletic Wrap is a laminate of nonwoven material and elastic fibers placed lengthwise to provide elasticity. The elastic wrap contains a cohesive material that makes it stick to itself but not to other materials or skin.

Intended Use:

3M™ Coheban™ Self-Adherent Wrap / Nexcare™ Athletic Wrap is intended for use as an elastic wrap to provide compression or support, or to secure dressings or devices. This product is not designed, sold or intended for use except as indicated.

Intended User:

Intended users are health care professionals and lay persons following the directions of the professional and the instruction for use.

Warnings:


- Wrap will not slip or loosen after application. DO NOT wrap too tightly to avoid impairing circulation and causing serious injury. The risk of impaired circulation is higher for children.
- Monitor area frequently (no less than every 24 hours) for signs of swelling, discoloration, pain, numbness, tingling, or other changes in sensation. If these symptoms occur, remove wrap immediately and contact your health care provider.
- DO NOT place the Wrap in direct contact with an open wound because it may lead to infection.
- This product contains natural rubber latex which may cause allergic reactions.

Precautions:

- 3M™ Coheban™ Self-Adherent Wrap / Nexcare™ Athletic Wrap does not slip or loosen after application. It should be applied with only the appropriate amount of compression.
- When used as a component in a compression wrap system, it is important to ensure adequate arterial blood flow to avoid injury secondary to decreased circulation. 3M™ Coheban™ Self-Adherent Wrap / Nexcare™ Athletic Wrap should be used as a component of a compression wrap system only under the supervision of a wound care specialist.
- Do Not Reuse. Reuse may result in compromising product integrity and / or lead to device failure.

Directions for Use:

- Unwind approximately 12 inches (30 cm) of 3M™ Coheban™ Self-Adherent Wrap / Nexcare™ Athletic Wrap from the roll and allow the wrap to relax.
- Without stretching, apply one full wrap and overlap. Press overlapped area lightly to keep the end in place. See Figure 1.
- Proceed with wrapping, applying tension to the wrap only to obtain desired compression. Always unwind more 3M™ Coheban™ Self-Adherent Wrap / Nexcare™ Athletic Wrap from the roll before making a wrap so that the product is not applied with too much tension. See Figure 2.
- When application is complete, cut off excess 3M™ Coheban™ Self-Adherent Wrap / Nexcare™ Athletic Wrap. Secure the end of the application by gently pressing.

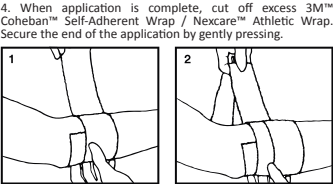Storage/ Shelf Life/Disposal:

This product should be stored at room temperature. Avoid excessive heat and humidity. For shelf life, refer to the date printed on each package. After the shelf life has expired, the device shall not be used but disposed as indicated.

How supplied / Ordering information:

3M™ Coheban™ Self-Adherent Wrap / Nexcare™ Athletic Wrap are available in multiple sizes and packaging sizes with a variety of different designs . Please contact your local 3M subsidiary for details about offerings in your country.

Note: Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

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