3M

Nexcare™

Blood Stop en

Product Description
Blood Stop plasters are disposable, first aid adhesive strips. Blood Stop plasters consist on non-woven dressing pad fixed on an adhesive conformable, non-woven backing.
Blood Stop Bandages contain micro-dispersed oxidized cellulose (a well-known, tolerated haemostatic agent), that reduces blood loss and stops bleeding faster than a traditional dressing (no m•doc™) through oxidized cellulose powder local activity.

Intended Use / Intended Purpose
The first aid dressing consisting of the wound pad with m•doc™ is dedicated for covering and protecting minor wounds (cuts, scrapes, abrasions) while the wound heals.

Intended users
The products are intended for the general consumer, lay persons following the instructions for use.

Warnings
In case of profuse bleeding or signs of infection, a physician should be consulted.
Signs of infection may include: pain, redness, swelling, unusual odor or drainage, absent or delayed healing, or fever.
If the wound does not begin to show signs of healing or if any other unexpected symptoms occur, consult a health care professional.

Precautions
The wound should be cleaned before the dressing is applied. Do not reuse. This is a single use device. Reuse may result in compromising product integrity and lead to device failure.

Directions for Use

1. Open package and remove dressing
2. Remove the paper liner to expose the adhesive surface
3. Firmly smooth down adhesive border on to the skin
4. Apply to clean and dry skin to ensure good adhesion

Apply a new Blood stop dressing when the dressing has reached its absorbent capacity.

Storage/ Shelf Life/Disposal
For best results store at normal room temperature. Avoid exposure to heat and humidity.
For shelf life, refer to the expiration date, which is printed on the base of the back.
Dispose of product in regular household waste.

Customer Information
Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.
For more information see, HCBRegulatory.3M.com
For further information, please contact your local 3M representative or contact us at 3M.com and select your country.

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© 2022, 3M. All rights reserved. 3M and Nexcare are trademarks of the 3M. m.doc® - is a trademark of HemCon Medical Technologies Europe Limited
3M Deutschland GmbH