
Nexcare™

ColdHot Therapy Pack Happy Kids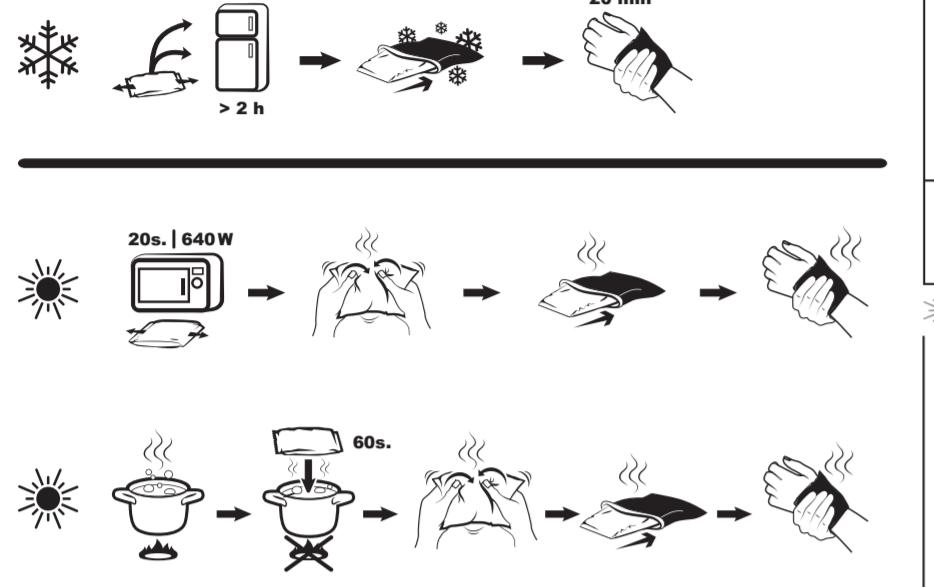

3M Deutschland GmbH Health Care Business
Carl-Schurz-Str.1, 41453 Neuss, Germany
Made in Germany

© 2022 3M. All rights reserved.
3M and Nexcare are trademarks of 3M.
3M.com
DW-2663-0068-5/0922

en Reusable cold and hot gel pack

The 3M™ Nexcare™ ColdHot Therapy Pack Happy Kids pack can be used cold or hot and is a convenient solution for those who prefer drug-free therapy. A protective cover for the gel pack is included, allowing for easy and comfortable application. The gel is non-toxic and safe for whole family, even accidental ingestion of a small amount of gel will not be harmful.

Cold therapy

Cold helps to bring relief in swelling, inflammation and pain.
When to use ColdHot Happy Kids pack

- Post injection and insect bite swelling
- Acute injuries (bruises, sprains, minor bumps)
- Nosebleeds

Preparation - cold

Refrigerator or freezer
Keep the gel pack in the refrigerator or freezer for minimum 2h before use. The gel pack will be always ready to use if it is stored constantly in the refrigerator or freezer.

Always insert gel pack in the supplied cover. Use the non-white side of the provided cover or clean towel, when applying to the skin. Apply for no longer than 20 minutes. Then wait another 20 minutes before reapplying the cold pack again.

Hot therapy

Heat helps to reduce muscle spasm and to decrease pain, soothes and relaxes.
When to use ColdHot Happy Kids pack

- Stomach cramps (abdominal muscle cramping)
- Hand warming

Preparation - hot

Hot water
Boil water and remove the pan from the heat. Immerse the gel pack just into boiled water (taken off the heat) for 60 sec. Remove gel pack carefully from hot water. Knead to distribute heat evenly!

Always insert gel pack in the supplied cover. Use the non-white side of the provided cover or clean towel, when applying to the skin. Apply to the affected area for as long as it is comfortable. Re-warm gel pack once it reaches room temperature.

Microwave
Microwave
Lay gel pack flat in microwave, set to 640W and heat for 20 sec. Remove gel pack carefully from microwave. Knead to distribute heat evenly!

Disposal
Dispose of product in regular household waste.

Please report a serious incident occurring in relation to the use of this product to 3M and the local competent authority (EU) or local regulatory authority.

Date of issue of the instruction of use 2019-10

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