
Nexcare™

ColdHot Therapy Pack Back and Abdomen Belt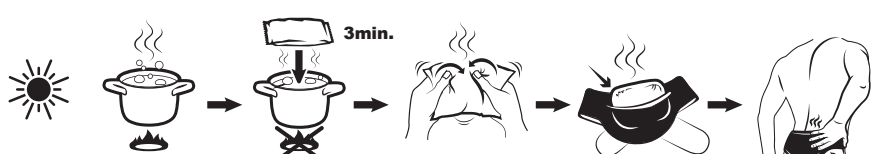

3M Deutschland GmbH Health Care Business
Carl-Schurz-Str. 1, 41453 Neuss, Germany

© 2022 3M. All rights reserved.
3M and Nexcare are trademarks of 3M.

3M.com
01-2663-0013-1/0022

en Reusable cold and hot gel pack

The 3M™ Nexcare™ ColdHot Therapy Pack Back and Abdomen Belt can be used cold or hot and is a convenient solution for those who prefer drug-free therapy. A protective cover for the gel pack is included, allowing for easy and comfortable application. The gel is non-toxic and safe for whole family, even accidental ingestion of a small amount of gel will not be harmful.

Cold therapy

Cold helps to bring relief in swelling, inflammation and pain.

When to use ColdHot Back and Abdomen Belt pack

- Acute injuries (bruises, sprains, minor bumps)

* Preoatration - cold

Refrigerator or freezer
Keep the gel pack in in the refrigerator or freezer for minimum 2h before use. The gel pack will be always ready to use if it is stored constantly in the refrigerator or freezer.

Always insert gel pack in the supplied cover or clean towel, when applying to the skin. Apply for no longer than 20 minutes. Then wait another 20 minutes before reapplying the cold pack again.

Hot therapy

Heat helps to reduce muscle spasm and to decrease pain, soothes and relaxes.

When to use ColdHot Back and Abdomen Belt pack

- Lower back pain
- Period pain
- Stomachache (alleviate abdominal muscle cramping)

* Preoatration - hot

Hot water
Boil water and **remove the pan from the heat**. Immerse the gel compress into just boiled water (taken off the heat) for 3 minutes.

Microwave
Lay gel pack flat in microwave, for 30 seconds at 640W and heat for 3 x 30 sec. Remove gel pack carefully from microwave. Knead to distribute heat evenly!

Always insert gel compress in the supplied cover or clean towel, when applying to the skin. Apply to the affected area for as long as it is comfortable. Re-warm gel compress once it reaches room temperature.

Disposal
Dispose of product in regular household waste.

Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

Date of issue of the instruction of use 2019-10

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