
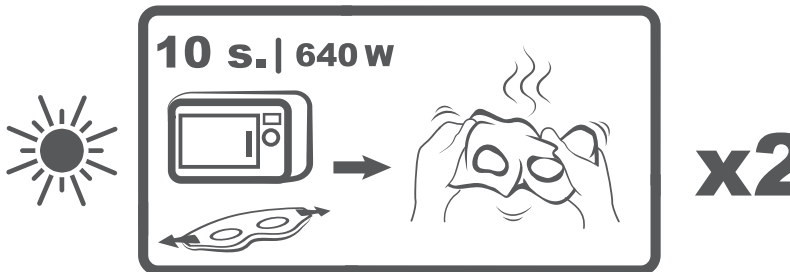
Nexcare™

ColdHot Therapy Pack Mask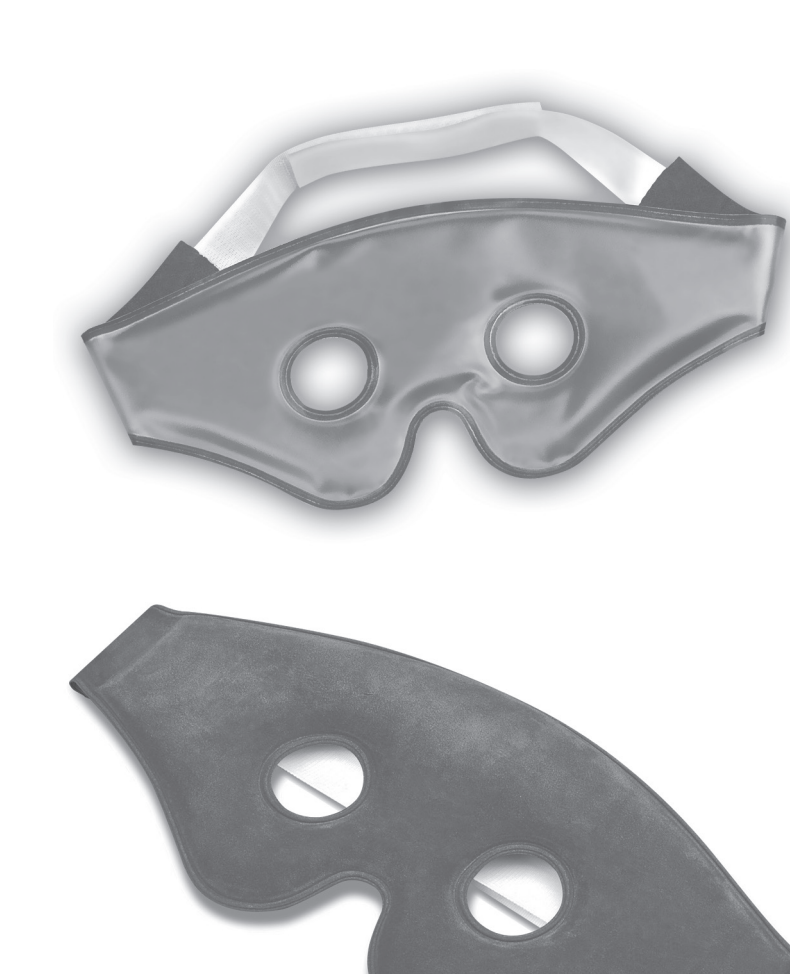

en Reusable cold and hot gel pack

The 3M™ Nexcare™ ColdHot Therapy Pack Mask can be used cold or hot during facial pain and to support cosmetic treatments in a drug-free way. It has an effective ergonomic design and velvety skin friendly surface. The blue gel is non-toxic and safe for the whole family.

* Cold therapy

Cold helps to reduce swelling, inflammation and pain.

When to use ColdHot Mask

- Tired Headache
- Tired eyes
- Swollen eyes
- Digital eye strain

* Preparation - Cold

Refrigerator

Keep the gel pack in in the refrigerator for minimum 2h before use. The gel pack will be always ready to use if it is stored constantly in the refrigerator.

Apply for no longer than 20 minutes with the velvety part to the skin. Then wait another 20 minutes before reapplying the cold pack again.

* Hot therapy

Heat helps to reduce muscle spasm and to decrease pain, soothes and relaxes.

When to use ColdHot Mask

- Promotes relaxation

* Preparation - Hot

Hot water

Not suitable for heating in water due to the velvety fabric cover.

Microwave

Lay pack flat in microwave set to 640W and heat 2 x 10 sec. Remove gel pack carefully from microwave. After each heating knead pack to distribute heat evenly!

Apply to the affected area for as long as is comfortable. Re-warm pack once it reaches room temperature.

Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

Warnings and Precautions

- For external use only.
- Do not use on broken or sensitive skin.
- People with known or suspected nerve or circulatory problems should not use this product unless prescribed by a physician.
- Supervise usage by children or people with cognitive disabilities.
- Pregnant women should consult a midwife or gynecologist for advice before use.
- Never heat the gel pack when folded, creased or frozen. Leakage may occur.
- Gel pack will be damaged by over heating.
- Do not heat in an oven or grill.
- Gel pack is not dedicated for constant heating.
- Check for leaks before use.
- Never use a damaged gel pack. Throw away if leaking.
- Do not use when material degradation occurs: - gel becomes fluid - foil is ruptured causing leakage of gel.
- For unknown microwave oven power, please use the lowest power setting (defrost setting) and warm up the product in 10 seconds steps.
- Be careful when removing hot gel pack with bare hands from microwave.
- If the gel pack is too hot, wait for it to cool down before applying to body.
- Never use foil directly on skin. Apply the velvety side to skin only.
- Do not sit or lean on the gel pack or apply excessive pressure because it may cause the gel pack to rupture and/or leak. Do not use while sleeping.
- Care must be taken to avoid frostbites and superficial nerve injury.
- Over heating of gel pack may cause burns.
- Heat may be sufficiently analgesic to produce local burns.
- Stop application in case of any discomfort.
- If condition persists, seek medical advice.
- In case of any doubts consult with a physician and/or pharmacist.

Gel pack care instructions

The mask can be washed by hand with a mild detergent. Do not use alcohol or other solvents to clean the pack. Keep the Nexcare ColdHot Mask in a refrigerator to extend useful lifetime.

Disposal

Disposal of product in regular household waste.

Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

Date of issue of the instruction for use 2019-10

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