
# 3M Nexcare™ ColdHot Therapy Pack Maxi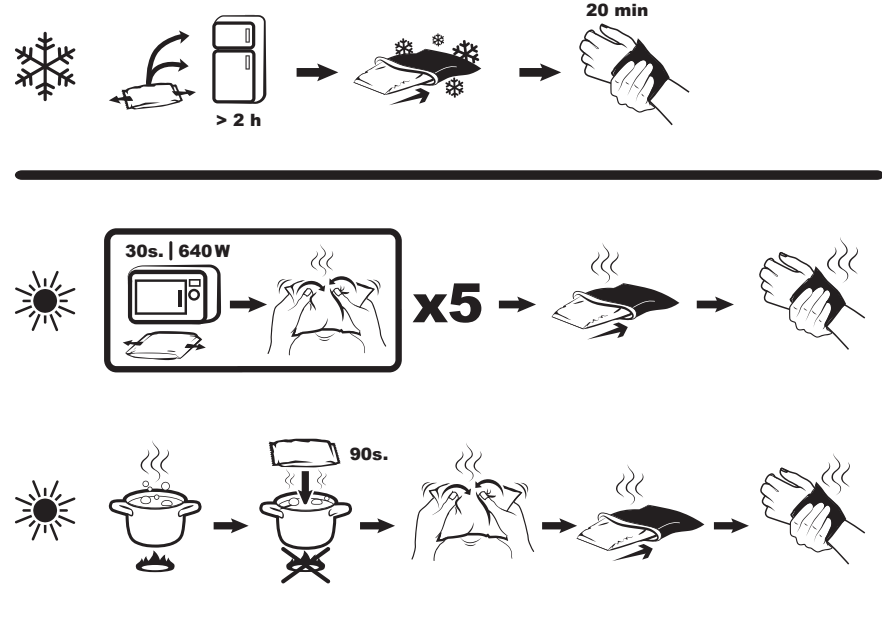

3M Deutschland GmbH Health Care Business  
Carl-Schurz-Str. 1, 41453 Neuss, Germany  
MD CE UK CA

© 2022 3M. All rights reserved.  
3M and Nexcare are trademarks of 3M.  
08X.com  
DN-2663-0011-50092

## en Reusable cold and hot gel pack

The 3M™ Nexcare™ ColdHot Therapy Pack Maxi can be used cold or hot and is a convenient solution for those who prefer, drug-free therapy. A protective cover for the gel pack is included, allowing for easy and comfortable application. The gel is non-toxic and safe for whole family, even accidental ingestion of a small amount of gel will not be harmful.

Warnings and Precautions

- For external use only.
- Do not use on broken or sensitive skin.
- People with known or suspected nerve or circulatory problems should not use this product unless prescribed by a physician.
- Supervise usage by children or people with cognitive disabilities
- Pregnant women should consult a midwife or gynecologist for advice before use.
- Never heat the gel pack when folded, creased or frozen. Leakage may result.
- Gel pack will be damaged by over heating.
- Do not heat in an oven or grill.
- Gel pack is not dedicated for continuous heating.
- Check for leaks before use.
- Never use a damaged gel pack. Throw away if leaking.
- Do not use when material degradation occurs.
- Acute injuries (bruises, sprains, minor bumps)

- Preparation - cold**
- Refrigerator or freezer**
- Keep the gel pack in in the refrigerator or freezer for minimum 2h before use. The gel pack will be always ready to use if it is stored constantly in the refrigerator or freezer.

Always insert gel pack in the supplied cover or clean towel, when applying to the skin. Apply for no longer than 20 minutes. Then wait another 20 minutes before reapplying the cold pack again.

- Hot therapy**
- Heat helps to reduce muscle spasm and to decrease pain, soothes and relaxes.
- When to use ColdHot Maxi pack**
- Lower back pain
- Period pain
- Stomachache (alleviate abdominal muscle cramping)
- Chronic musculoskeletal pain resulting from past contusions
- General feeling of cold (use as hot water bottle)

- Gel pack care instructions**
- The ColdHot Maxi gel pack can be washed by hand in a mild detergent.
- Do not use alcohol or other solvents to clean the gel pack. Store in a dark place.

- Disposal**
- Dispose of product in regular household waste.
- Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

- Preparation - hot**
- Hot water**
- Boil water and remove the pan from the heat. Immerse the gel pack into just boiled water (taken off the heat) for 90 sec.
- Remove gel pack carefully from hot water.
- Knead to distribute heat evenly!

Always insert gel compress in the supplied cover or clean towel, when applying to the skin. Apply to the affected area for as long as it is comfortable. Re-warm gel compress once it reaches room temperature.

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**Gel pack care instructions**

The ColdHot Therapy Pack Maxi can be washed by hand in a mild detergent.

Do not use alcohol or other solvents to clean the gel pack. Store in a dark place.

**Disposal**

Dispose of product in regular household waste.

Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

Date of issue of the instruction of use 2019-10

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