3M Nexcare™ Cold/Hot Therapy Pack Mini

3M Deutschland GmbH Health Care Business
Carl-Schurz-Str. 1, 41453 Neuss, Germany
Made in Germany

© 2022 3M. All rights reserved.
3M and Nexcare are trademarks of 3M.
3M.com
09-2683-0008-1/0922

en Reusable cold and hot gel pack

The 3M™ Nexcare™ Cold/Hot Therapy Pack Mini can be used cold or hot and is a convenient solution for those who prefer, drug-free therapy. A protective cover for the gel pack is included, allowing for easy and comfortable application. The gel is non-toxic and safe for whole family, even accidental ingestion of a small amount of gel will not be harmful.

Cold therapy

Cold helps to bring relief in swelling, inflammation and pain.

When to use Cold/Hot Mini pack

- Tooth ache
- Acute injuries (bruises, sprains, minor bumps)
- Nosebleeds

* Preparation - cold

Refrigerator or freezer
Keep the gel pack in the refrigerator or freezer for minimum 2h before use. The gel pack will be always ready to use if it is stored constantly in the refrigerator or freezer.

Always insert gel pack in the supplied cover or clean towel, when applying to the skin. Apply for no longer than 20 minutes. Then wait another 20 minutes before reapplying the cold pack again.

Hot therapy

Heat helps to reduce muscle spasm and to decrease pain, soothes and relaxes.

When to use Cold/Hot Mini pack

- Wrist arthritis as an adjunct therapy for rheumatoid arthritis (without acute inflammation)
- Hand warming

* Preparation - hot

Hot water
Boil water and remove the pan from the heat. Immerse the gel pack into just boiled water (taken off the heat) for 60 sec. Remove gel pack carefully from hot water. Knead to distribute heat evenly!

Microwave
Lay gel pack flat in microwave, set to 640W and heat for 20 sec. Remove gel pack carefully from microwave. Knead to distribute heat evenly!

Always insert gel pack in the supplied cover or clean towel, when applying to the skin. Apply to the affected area for as long as it is comfortable. Re-warm gel pack once it reaches room temperature.

- For external use only.
- Do not use on broken or sensitive skin.
- People with known or suspected nerve or circulatory problems should not use this product unless prescribed by a physician.
- Supervise usage by children or people with cognitive disabilities.
- Pregnant women should consult a midwife or gynecologist for advice before use.
- Never heat the gel pack when folded, creased or frozen.
- Leakage may result.
- Gel pack will be damaged by over heating.
- Do not heat in an oven or grill.
- Gel pack is not dedicated for continuous heating.
- Check for leaks before use.
- Never use a damaged gel pack. Throw away if leaking.
- Do not use when material degradation occurs:
 - gel becomes fluid
 - full is ruptured causing leakage of gel.
- For unknown microwave oven power, please use the lowest power setting (defrost setting) and warm up the product in 20 seconds steps.
- Be careful when removing hot gel pack with bare hands from hot water / microwave.
- If the gel pack is too hot, wait for it to cool down before applying to body.
- Never use gel directly on skin. Use the provided cover or wrap in a clean cotton towel.
- Do not sit or lean on the gel pack or apply excessive pressure because it may cause the gel pack to rupture and/or leak. Do not use while sleeping.
- Care must be taken to avoid frostbites and superficial nerve injury.
- Over heating of gel pack may cause burns.
- Heat may be sufficiently analgesic to produce local burns.
- Stop application in case of any discomfort.
- If condition persists, seek medical advice.
- In case of any doubts consult with a physician and/or pharmacist.

Gel pack care instructions

The Cold/Hot Mini gel pack can be washed by hand in a mild detergent.
Do not use alcohol or other solvents to clean the gel pack. Store in a dark place.

Disposal

Dispose of product in regular household waste.

Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

Date of issue of the instruction of use 2019-10

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